

NYOrtho Smoker's Aprons cover a seated smoker and help prevent accidental burns to the skin and/or clothing. Hot ashes and lit cigarettes will not burn through the fabric, however, they may scorch the fabrics or leave a burn mark on the surface. These Smoker's Aprons can be used in a variety of applications with individuals in wheelchairs, gerichairs, dining room chairs, etc. The aprons are reversible, with no front or back for extended life.

- Certified flame-retardant fabric meets NFPA 701
- Lightweight, fire-proof apron cannot be ignited by a cigarette, cigar or burning match
- Available oversized for Geri-chair (9531)
- Special design prevents cigarettes and cigars from falling between resident and the wheelchair
- Available in an extra-durable & fire resistant material (9532)

#### Economy Smoker's Apron

- 9530 White, 30" x 34"
- 9530-3 3-Pack: White, 30" x 34"
- 9530-4 4-Pack: White, 30" x 34"
- 9531 Oversized, White, 30" x 44"

#### Deluxe Smoker's Apron

- 9532 Deluxe Smoker's Apron (Grey), 30" x 34"
- 9532-3 3-Pack: Deluxe Smoker's Apron (Grey), 30" x 34"
- 9532-4 4-Pack: Deluxe Smoker's Apron (Grey), 30" x 34"

#### CLEANING INSTRUCTIONS:

- DO NOT LAUNDER. Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure product is completely dry before use.

#### APPLICATION INSTRUCTIONS:

- 1) Place the apron over the resident and adjust the neck strap positioning at the top of the apron above the chest. Apply hook to loop.
- 2) Utilizing the hook and loop, attach the upper straps around the left and right bars behind the armrest pads.
- 3) Wrap the lower straps in the same fashion around the left and right bars below the front of the wheelchair seat.



#### STORAGE & HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

#### INSPECTION

Inspect before each use: check for broken stitches or parts; torn, cut or frayed material; or buckles that do not hold securely. DO NOT use soiled or damaged products.

#### DISPOSAL

Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

#### WARNING

Make sure the patient cannot slide down or fall off the chair seat and become entrapped by the Smoker's Apron. If their body becomes suspended by the apron, suffocation could result.

This apron is not a substitute for proper supervision and all patients or residents should always be closely supervised while smoking. Peripheral materials used in the construction of the apron such as the threading, straps and labels, represent less than 1% of the product. They do not offer the same level of flame resistance and pose only a remote risk of fire hazard.