

Lifetime Limited Warranty

This ConvaQuip product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to our customers for years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This Warranty provides only for replacement of defective parts and does not cover shipping or labor charges.

This warranty does not cover product failures due to normal wear and tear or device misuse. This warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and require periodic replacement.

This warranty shall not apply to serial numbered products if the serial number has been removed or defaced, or to defects or damages caused by unauthorized parts, attachments or repairs, negligence, accident, circumstances beyond our control, or improper operation, maintenance or storage.

ConvaQuip SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES WHATSOEVER. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

For additional information on this ConvaQuip product or this warranty, please contact an authorized ConvaQuip Products Dealer.



Toll Free: 800-637-8436



Model 1701B Bariatric Shower Chair with Back

INSTRUCTIONS
& WARRANTY



Capacity 500 lbs.

Hospital Quality

Height Adjustable



WEIGHT CAPACITY : 500 LBS

Assembly Instructions

SET-UP

1. Remove seat frame, back support and legs from box.
2. Slide back support through channel guides until back support tube buttons protrude fully through channel guide holes and lock into place.
3. Adjust seat height as directed in height adjustment section.

HEIGHT ADJUSTMENT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
2. Repeat for other three legs.
3. To be sure legs are locked into place by pushing down on or by pulling legs.

MAINTENANCE

1. Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that shower chair is well rinsed before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of shower chair.
3. Check fit and tightness of all nuts, bolts and knobs weekly to ensure that the shower chair is safe to use.



WEIGHT CAPACITY : 500 LBS

Assembly Instructions

SET-UP

1. Remove seat frame, back support and legs from box.
2. Slide back support through channel guides until back support tube buttons protrude fully through channel guide holes and lock into place.
3. Adjust seat height as directed in height adjustment section.

HEIGHT ADJUSTMENT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
2. Repeat for other three legs.
3. To be sure legs are locked into place by pushing down on or by pulling legs.

MAINTENANCE

1. Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that shower chair is well rinsed before using.
2. Never use organic solvents, abrasive cleaners or scouring pads on any part of shower chair.
3. Check fit and tightness of all nuts, bolts and knobs weekly to ensure that the shower chair is safe to use.

ConvaQuip 
BARIATRIC EQUIPMENT

Toll Free: 800-637-8436