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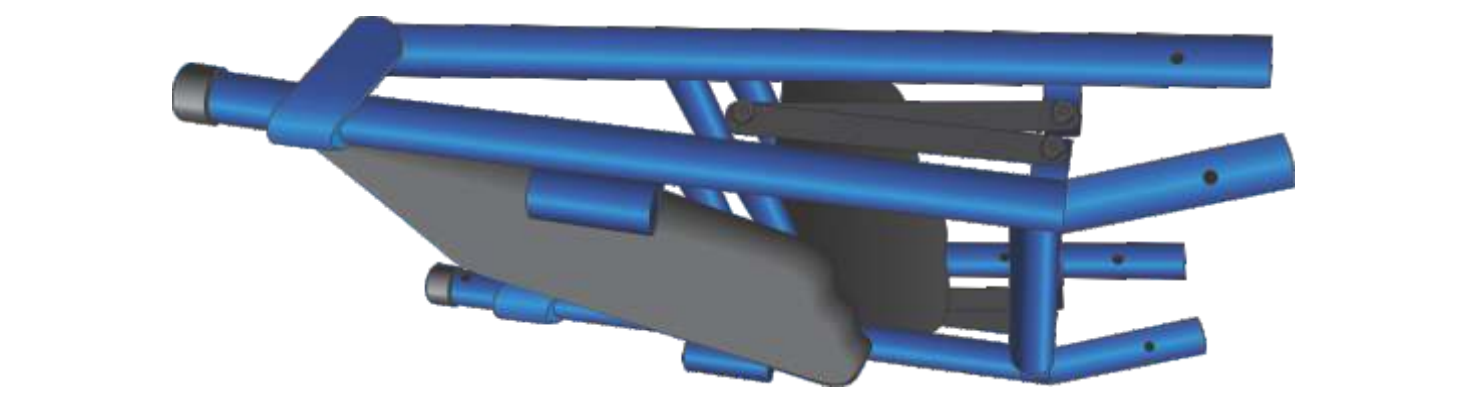
LIMITED WARRANTY • GARANTIE LIMITÉE • GARANTÍA LIMITADA

(EN) AMG Medical Inc. warrants the frame of this Hugo® mobility aid to be free from defects in materials and workmanship for three (3) years. All other parts and components are warranted for a period of one (1) year. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

For warranty service, you have two options:
1. Return your Elite™ to the vendor where the unit was bought.
2. In Canada & the U.S. call AMG's Consumer Info Line at 1-800-363-2381 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of Purchase and Serial Number (on bar underneath the seat).
Our customer service team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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- (EN)**
1. Allow time before walking to perform some simple stretches as this will help you walk further and avoid injury.
 2. Bring a bottle of water with you to stay hydrated.
 3. Wear walking shoes that are well cushioned and snug in the heel, yet allow room to wiggle your toes.
 4. When possible, walk on soft ground that is more joint-friendly such as grass or dirt paths and when walking near a road wear bright clothing that is easily visible.

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- (EN)**
(F)
(E)

NOTE

If you are 5'3" or shorter, first remove all 4 spacer tubes (G and M) from the wheel shafts before proceeding with the steps below, unless you prefer sitting at an elevated position.

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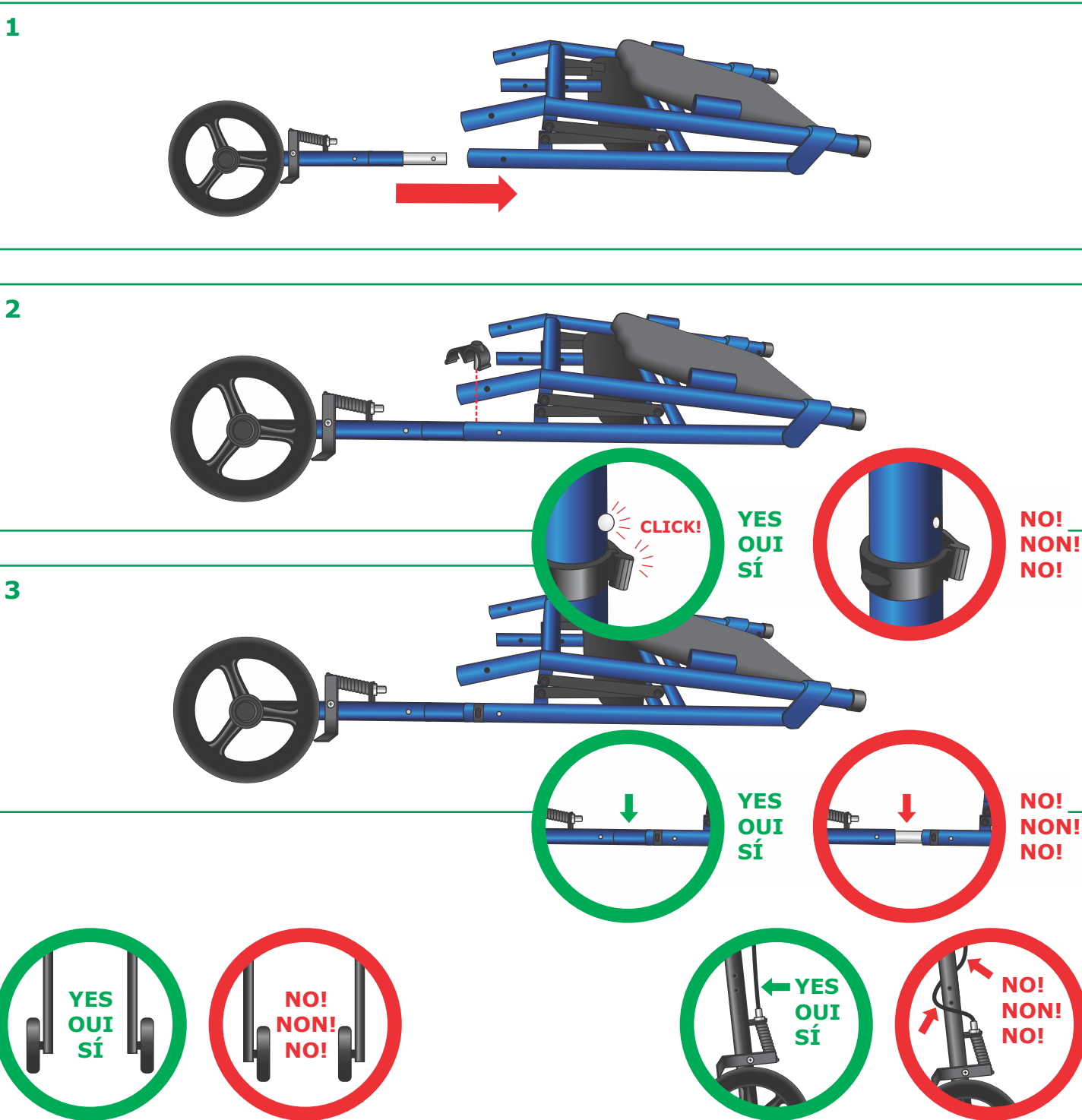
EN
Lay the walker frame down so that the Seat (J) is facing up and remove the black plastic plugs from all 4 leg tubings.

Insert the Rear Wheels (H) into the tubing as shown, noting that the wheels are OUTSIDE the frame and that the brake lever is facing towards the back of the unit. Ensure both Push Buttons click into place.

Secure by inserting an E-clip (L) into the hole in the frame, just below the Push Buttons and snap closed. Repeat this operation for the other Rear Wheel.

Note: Ensure that the wheels are OUTSIDE of the frame and that the brake cables are not wrapped around the tubing or frame.

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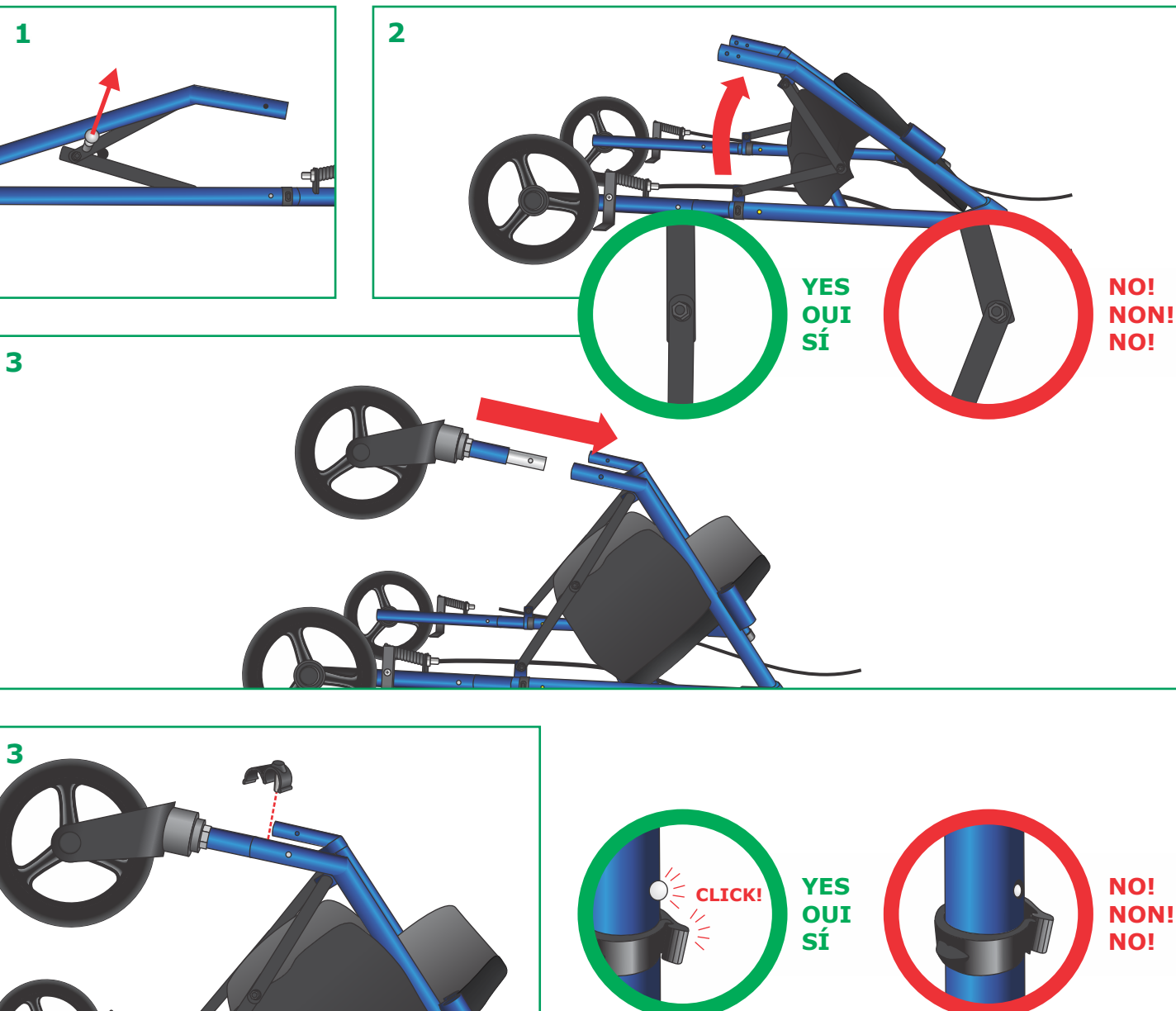
EN
Keeping the walker laying down flat, open the walker frame. To do so, pull out the Safety Latch (O) at the center of the black Folding Bracket (E), located at the side of the frame.

Insert Front Wheel (N) into frame until both Push Buttons click into place.

Insert E-Clip into hole in frame just below Push Buttons and snap closed. Repeat this operation for the other front wheel.

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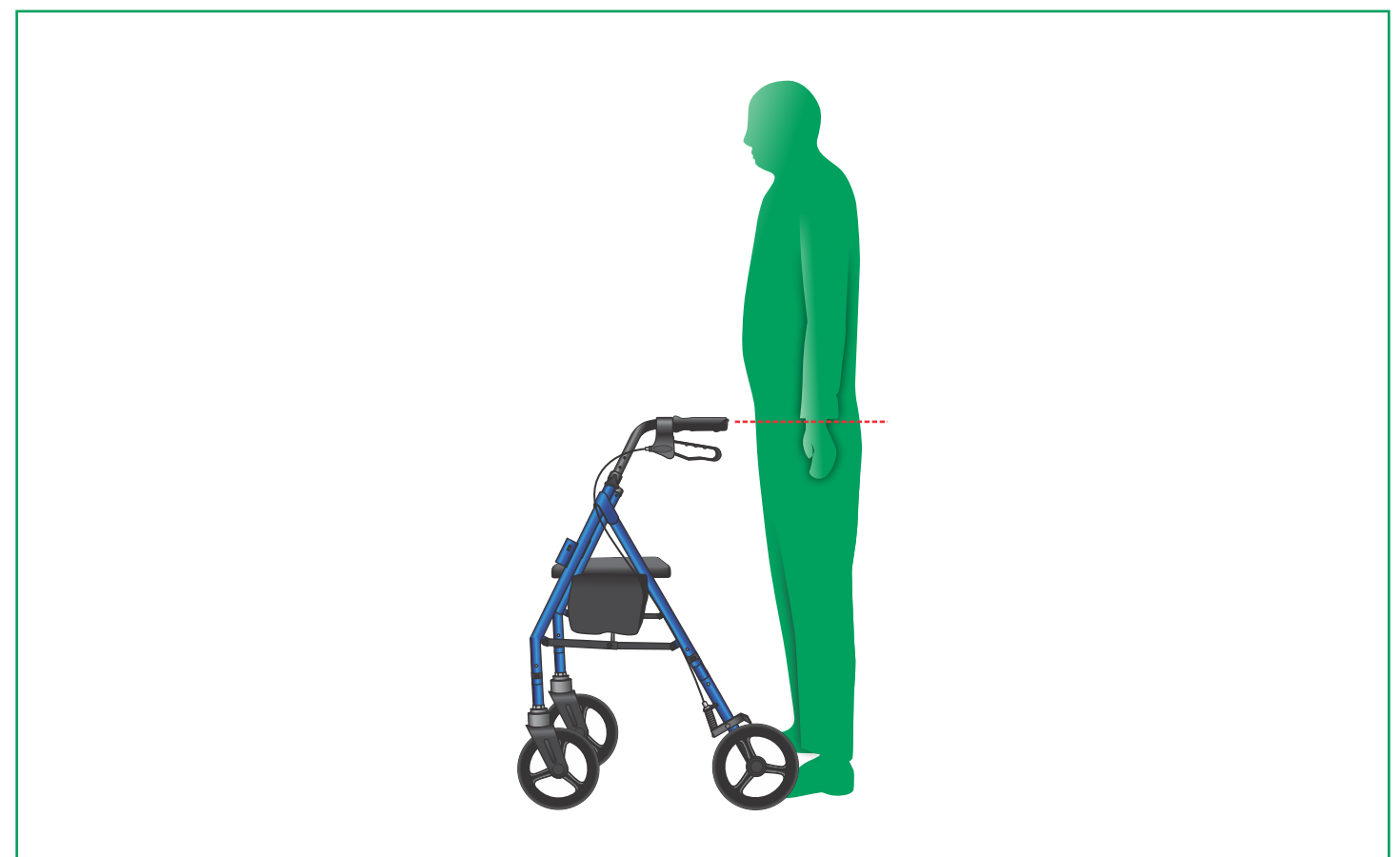
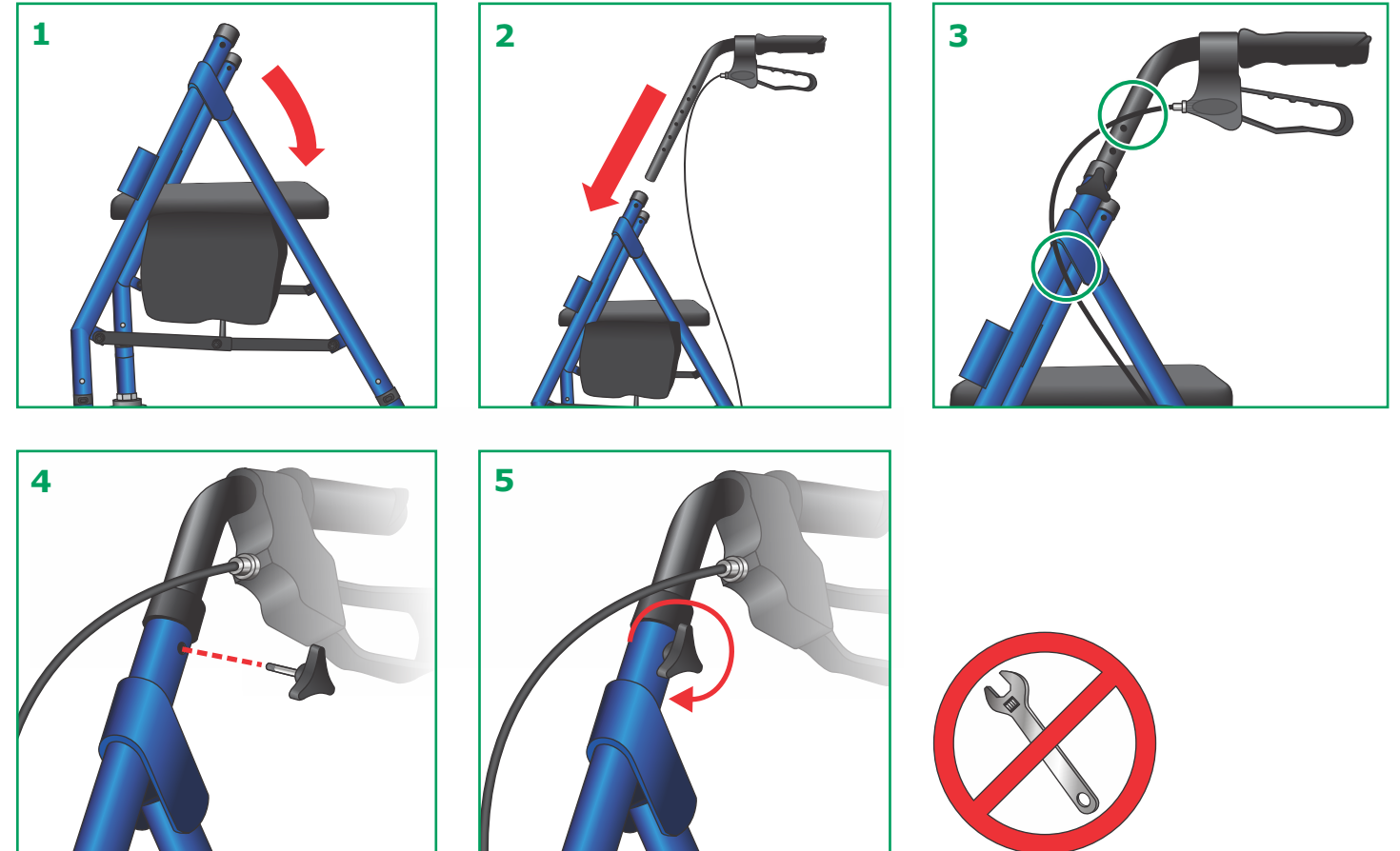
EN
Stand the walker upright. Open the frame by pushing the Seat (J) down into place. To ensure the frame is completely open, push down on the Folding Brackets (E) until they are fully extended.

Insert the Handlebars (C) so that the Handgrips are aligned with the Rear Wheels. The Brake Cables should be on the outside of the Handlebars and frame.

For proper height adjustment, stand upright with shoulders relaxed and arms by your side. The Handgrips should be at a height even with your wrists.

Secure the Handlebars at the closest hole to the desired height. Insert and tighten the triangular Hand Screws (D) until secure. HAND TIGHTEN ONLY.

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
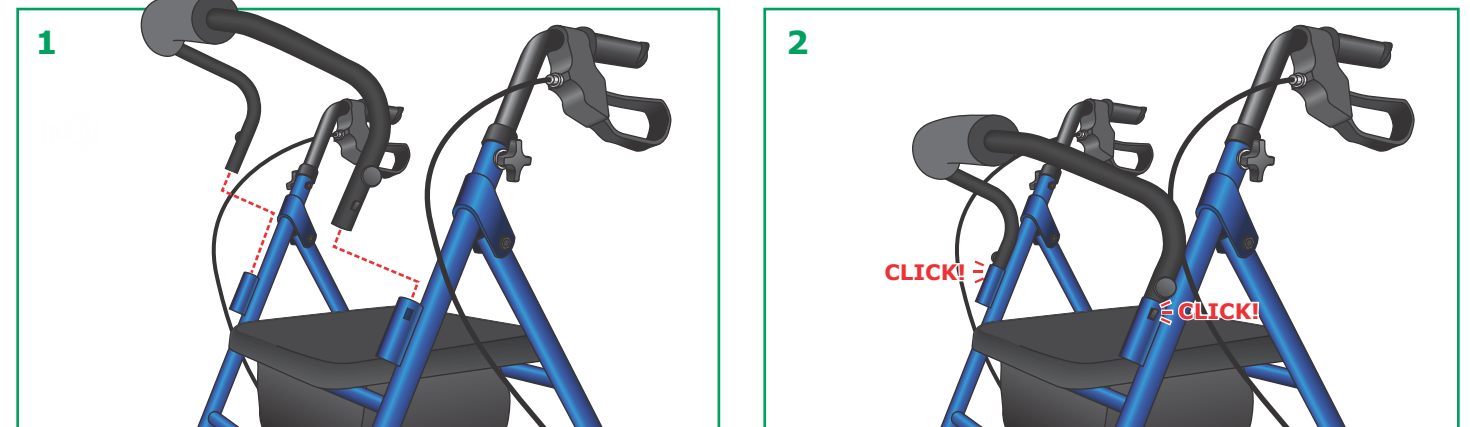
EN
Standing in front of the walker, hold the Backrest (I) so that the padded portion faces the front of the unit. Push in both Push Buttons and insert the Backrest into both sides of the Walker at the SAME TIME until it locks into place.

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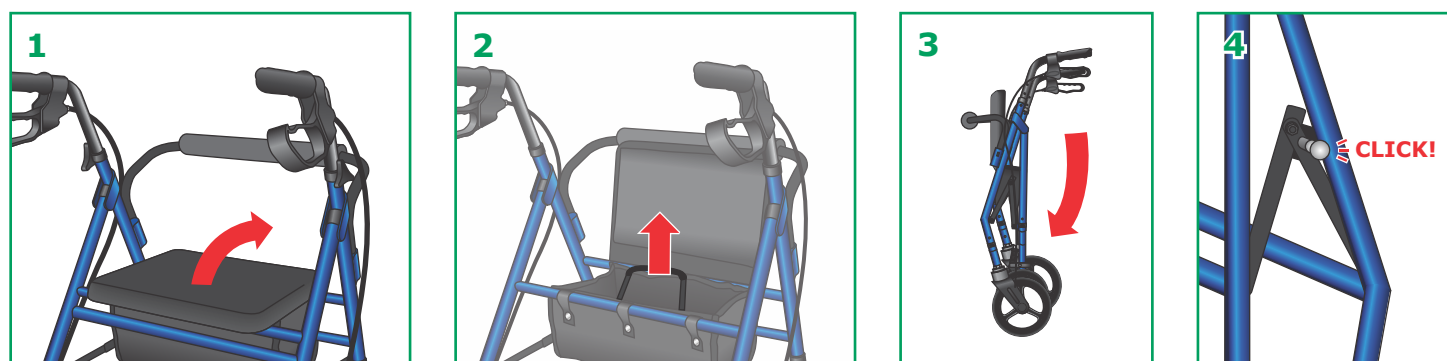7 TO FOLD PLIER PARA PLEGAR


EN
Lift the Seat up to gain access to the under seat storage bag.
As you lift up on the black strap the Folding Bracket (E) will fold and allow the Walker to close.
Continue to lift until you hear the Folding Bracket click and lock into place.

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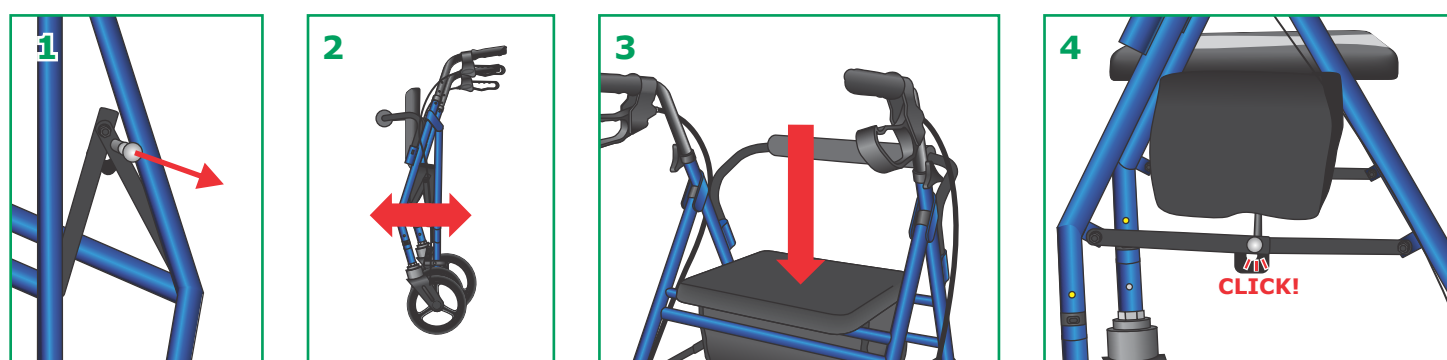8 TO UNFOLD DÉPLIER PARA DESPLEGAR



EN
Pull out the Safety Latch (O) at the center of the black Folding Bracket (E), located at the side of the frame. Open the frame completely by pushing down on the Folding Brackets (E) to spread apart the frame until fully extended.

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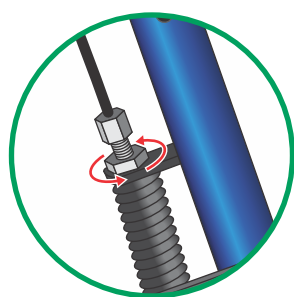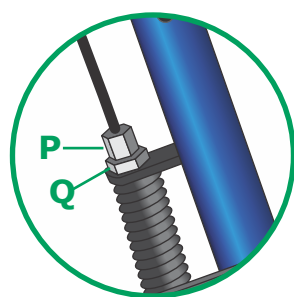EN
Over time the Brake Cables on your walker will stretch, much like those on a bicycle, and will require adjusting.
With the brakes NOT applied, hold the Adjusting Barrel (P) in place and rotate the Adjusting Nut (Q) clockwise to increase braking force.
To decrease braking force, rotate counter-clockwise.
IMPORTANT – Before using the walker again, ensure the brakes are working properly.
If the Rear Wheels of the walker move while the brakes are locked in position, DO NOT use the walker.
If you are unsure of how to adjust your brakes or feel you require further assistance please see your authorized Hugo® dealer or call our toll-free number (1-800-363-2381, Canada & USA only) for help.

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SAFETY PRECAUTIONS CONSIGNES DE SÉCURITÉ INSTRUCCIONES DE SEGURIDAD

EN
READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR HUGO® ELITE™

1. Consult your doctor, therapist or other qualified professional to determine if this Hugo® mobility aid will suit your particular needs.
2. **IMPORTANT** - Before using, make sure that all parts are secure and the frame is fully opened.
3. For proper positioning and use as a rolling walker, imagine a line drawn between the back of the two rear wheels. Part of one foot should always be slightly ahead of that line as you walk.
4. To slow down, squeeze the brake handles up. To lock the brakes, push down on the brake handles until the brakes click into locked position. If the rear wheels move while the brakes are in locked position, DO NOT use the rolling walker. Please see adjustment instructions in this manual or call our toll-free number: 1-800-363-2381 (Canada & USA only) or see your distributor. To release the brakes from locked position, pull up on the brake handles.
5. **IMPORTANT: WHEN USING THE ROLLING WALKER** - Before using the seat portion of your walker, ALWAYS lock the brakes FIRST. DO NOT navigate the rolling walker while sitting on the seat. The brakes should ALWAYS be locked when the seat is being used. Risk of fall and serious injury may result if Brakes are not locked while the user is sitting on the seat. Falls are possible if the user leans or slides backward on the seat without first installing the backrest. NEVER attempt to sit on the seat when the rolling walker is on an incline. Serious risk of falls and injury may result. **CAUTION - DO NOT** exceed the following weight limits, or serious injury could result: Rolling walker: 300 lbs. Seat: 300 lbs.

6. Store folding strap in underseat storage bag when not in use.
7. Never let anyone sit in the rolling walker while in a moving vehicle.
8. This product contains dry natural rubber.

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! NOTE

Your rolling walker should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.

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