X 4

X 2

X 2

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LIMITED WARRANTY • GARANTIE LIMITÉE • GARANTÍA LIMITADA

(EN)

AMG Medical Inc. warrants the frame of this Hugo® mobility aid to be free from defects in materials and workmanship for three (3) years. All other parts and components are warranted for a period of one (1) year. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

For warranty service, you have two options:

1. Return your walker to the Vendor where the unit was bought.
2. In Canada & the U.S. call AMG's Consumer Info Line at 1-866-518-6099 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of Purchase and Serial Number (on front crossbar underneath Hugo® logo).

Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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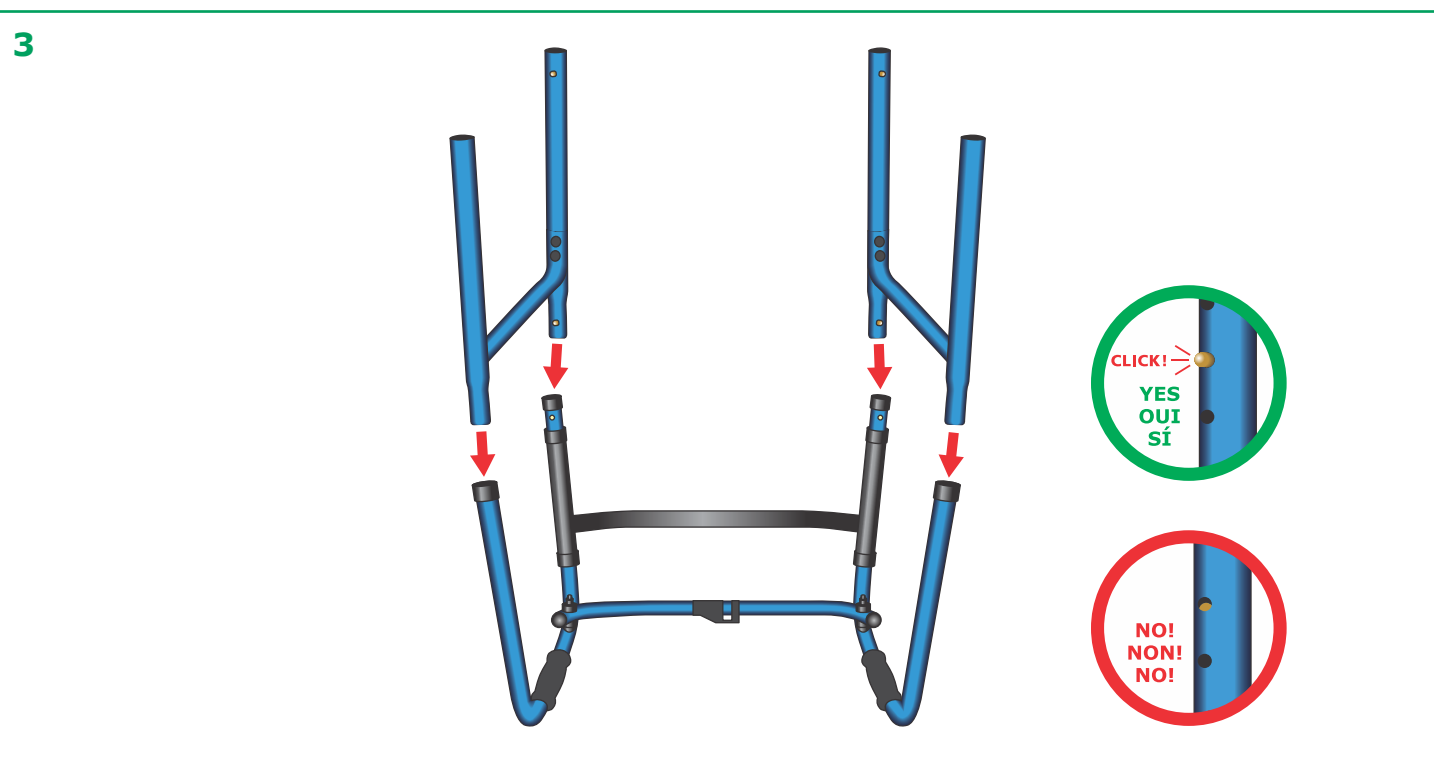
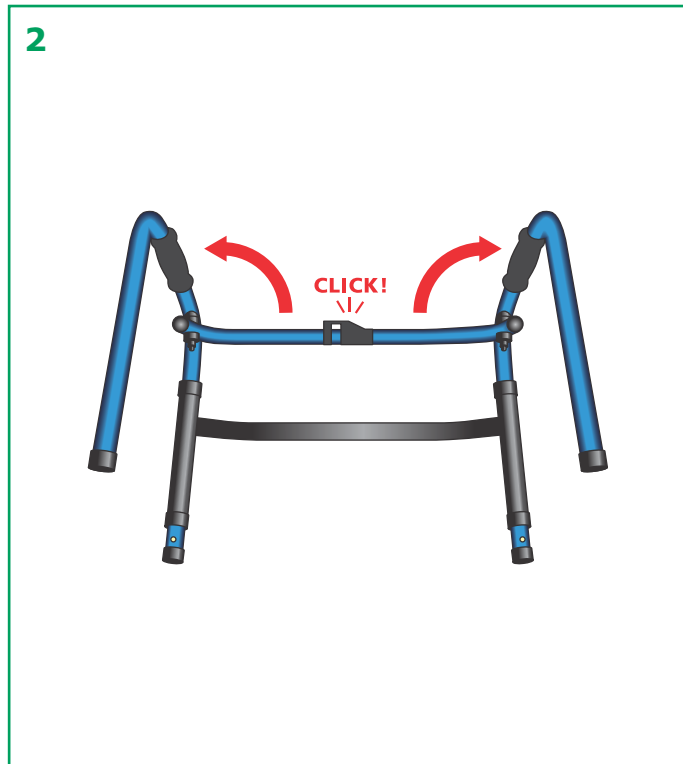
3 ASSEMBLE FRAME ASSEMBLER LE CHÂSSIS MONTAJE DEL MARCO

EN

Pull apart both sides of the Upper Frame with Handgrips (A), ensuring that the One-Touch Push-Button (B) is fully deployed and has clicked into place. Turn the walker upside down so the handgrips are on the ground and the legs are facing up.

Insert one Side Frame (C) into two of the leg openings, as illustrated. You need to push in the bronze push-buttons to guide the side frame into place until a click indicates it is locked. Repeat for the opposite side.

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EN

Insert all four bottom legs into the side frames. You will hear a click when the legs are locked into place. Ensure that each leg is adjusted to the same height adjustment.

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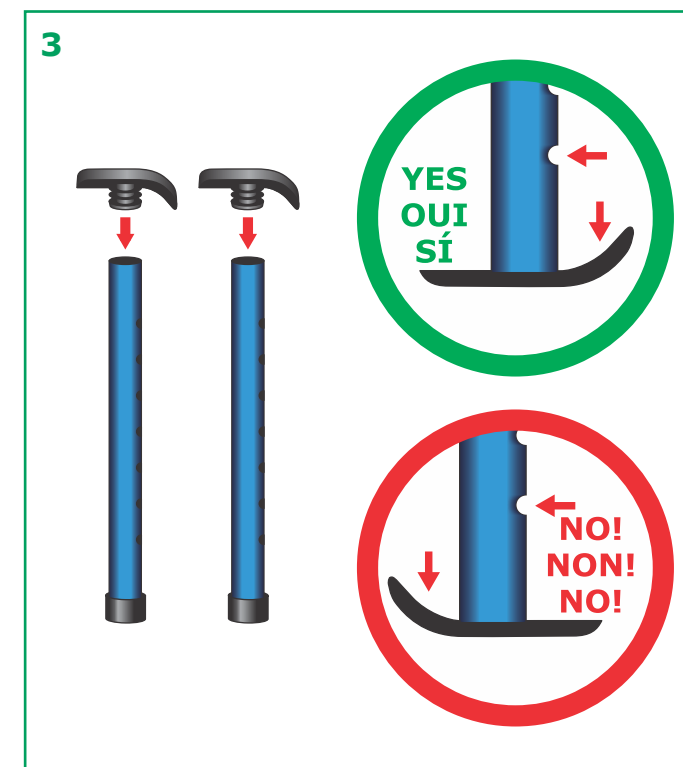
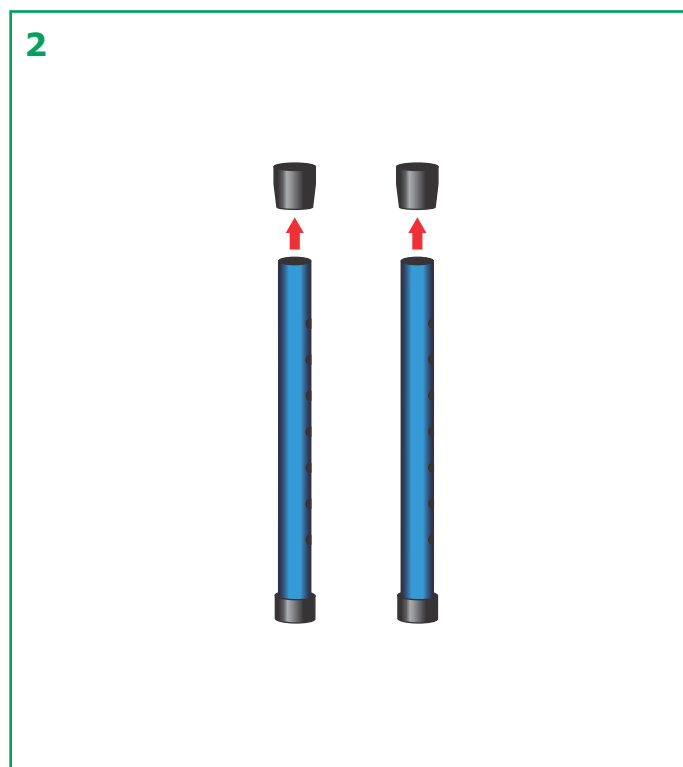
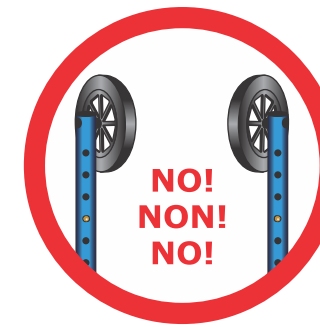
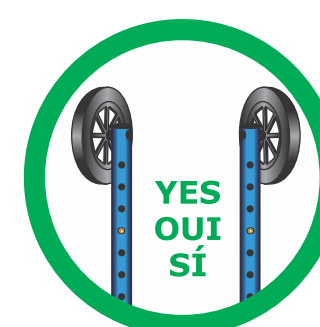
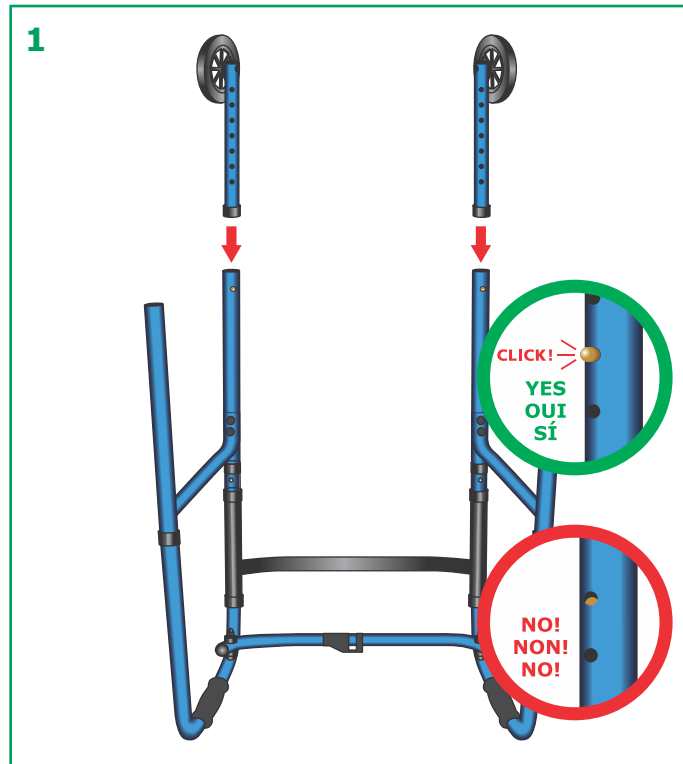
EN

Remove the Front Legs (F) and insert the two Wheels (H) onto the front of the frame until they lock into place. Ensure that each wheel is adjusted to the same height adjustment. The wheels should be installed on the FRONT LEGS ONLY, on the outside of the frame.

Remove the Rubber Tips (G) from the bottom of the two rear legs and replace with the Glides (I). Ensure that the glides are fully inserted into the tubes, with the front of the glides in line with the Adjustment Holes (D). The glides should be installed on the REAR LEGS ONLY. Turn the folding walker over, ensuring that both glides and both wheels contact the floor evenly.

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CAUTION ATTENTION PRECAUCIÓN

EN

Before using, always ensure that rubber tips are not ripped, worn or missing. Replace immediately if needed. Only use glides with front wheel attachments. DO NOT install glides on all four legs or use glides and rubber tips at the same time.

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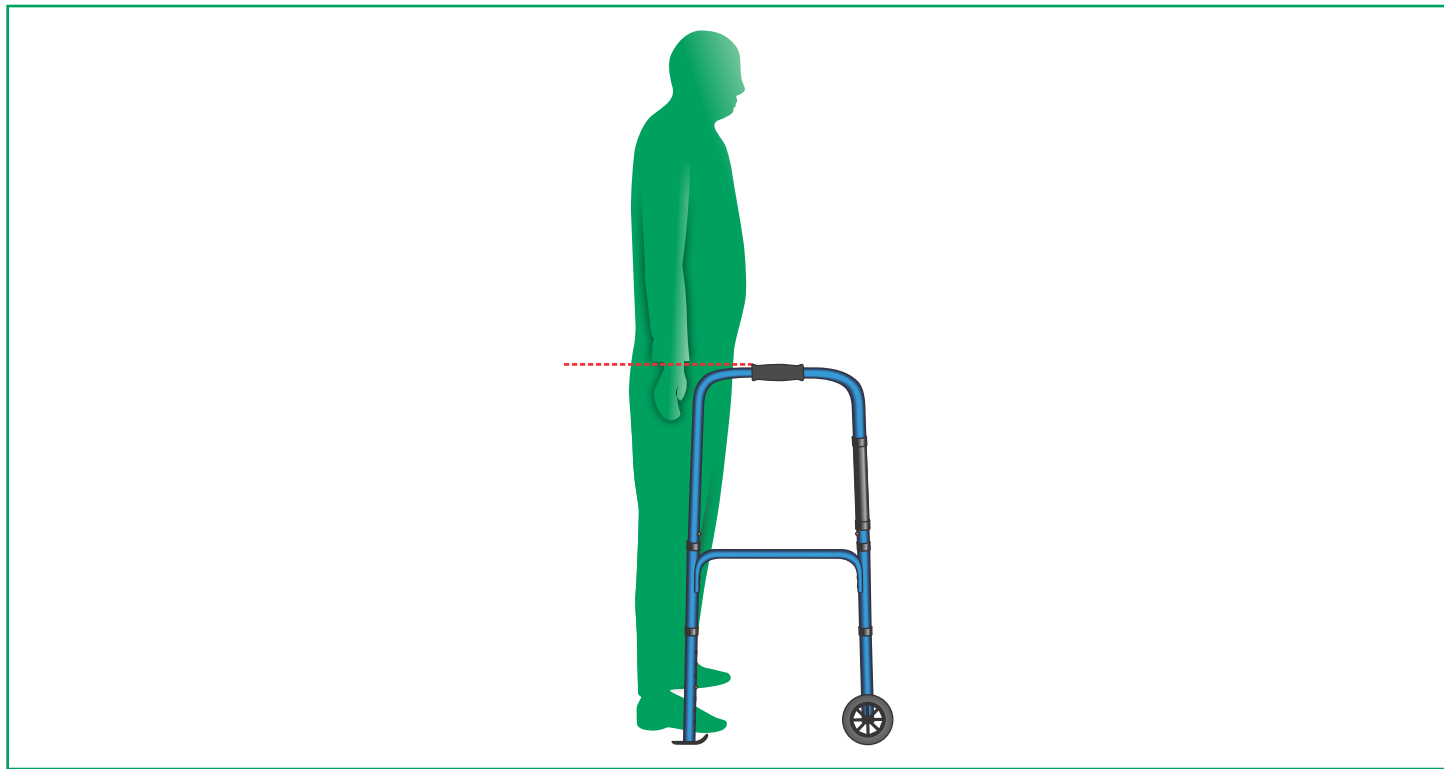
EN

Turn walker over ensuring all four legs contact the floor.

To determine the right height for you, extend your arms down to your sides. The handgrips should be at wrist height when in the proper position. Push in the bronze push-buttons to adjust all four legs to the same height.

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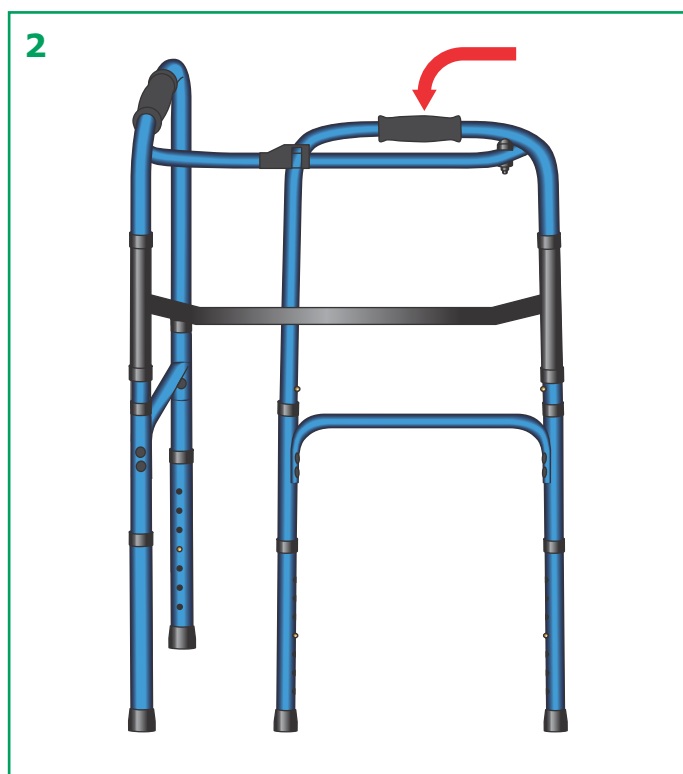
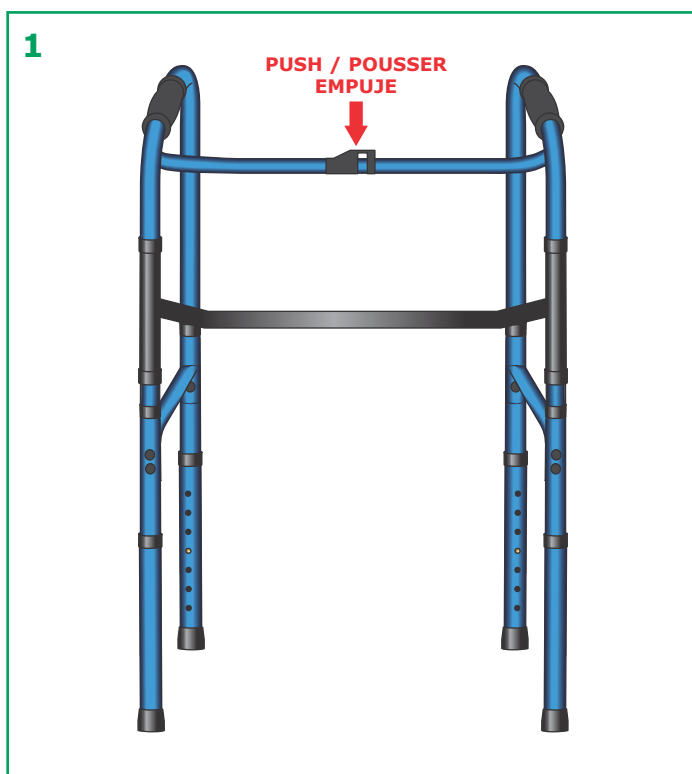
EN

Make sure you are not relying on the walker for stability while you are folding or unfolding the walker.

To fold, while pushing down on the One-Touch Push-Button (B), use your opposite hand to fold one side of the walker toward the center of the frame. Then slide in the other.

To unfold, simply grab both hand grips and open the frame. The One-Touch Push-Button (B) will click into place when fully deployed. Only use the walker when both sides have been fully deployed and locked into place.

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SAFETY PRECAUTIONS CONSIGNES DE SÉCURITÉ INSTRUCCIONES DE SEGURIDAD

EN

READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR HUGO® FOLDING WALKER

1. Consult your doctor, therapist or other qualified professional to determine if this Hugo® mobility aid will suit your particular needs.
2. Examine all parts for shipping damage. In case of shipping damage, or if anything is missing, do not use. Call our toll-free number: 1-866-518-6099 (Canada & USA only).
3. IMPORTANT - Before using, make sure that all parts are secure and the frame is fully opened.
4. Maximum weight capacity: 300 lb (136 kg).
5. The Hugo® Folding Walker provides support and stability while walking. It is not intended to support the full weight of the user.
6. Always use caution when walking on wet or icy surfaces.
7. Consult a qualified dealer or repair center for periodic inspections and repairs.
8. To clean, wipe frame using a clean soft cloth. Do not use abrasive materials. Clean wheels with warm water and mild cleanser. Dry with a clean cloth.
9. Replace any broken, damaged or worn items immediately.

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