

ENDURANCE HD[®]

BUILT TO LAST!

Heavy Duty Four Wheeled Walker



Weight Capacity: 500 lbs.

ITEM # W1802

Made in China

2011 ESSENTIAL MEDICAL SUPPLY, INC.[®]



Manufactured for
ESSENTIAL[®]
MEDICAL SUPPLY, INC.
Orlando, FL 32822

– SAVE THESE INSTRUCTIONS –

Do not attempt to adjust or operate your **ENDURANCE^{HD}** walker without carefully reading all instructions in this manual. If, after reading this manual, you have any questions, please call your health professional or place of purchase, otherwise injury or damage may occur.

SET UP INSTRUCTIONS

1. Remove all contents from carton and remove protective packaging, including the cardboard panel between the rear wheels. The carton should contain the following:



Do not use the walker if there is apparent damage and / or if parts are missing.

Please call place of purchase for repair / return information.

SET UP INSTRUCTIONS

2. Open frame by pushing down on the hinged folding mechanism so that the hinged bar is level.

Loop Brakes Handles are attached to the Frame by Cables (2)

Backrest

Backrest Receiving Tube

Height Adjustment Knobs (2) With Bolt Attached

Vinyl Tote

Serial No. Label on back
(Silver label, number starts with "RM-E")

Hinged Folding Mechanism

3. Insert the backrest, handles and tote.
 - (a) To insert the backrest, push pins on the outside of the backrest and insert backrest into the brackets on the front of the walker frame. Once properly installed, the backrest should click into place and the pins should protrude into the holes.
 - (b) Unscrew the bolts and height adjustment knobs from handles and insert handles into the frame of the walker while making sure the cables are on the outside of the frame.
 - (c) Adjust the handle height as directed by your physician, physical therapist or trained professional. Insert the bolt from inside of the frame so that the hex head fits snugly into the six sided hole.
 - (d) Firmly screw the knob until fully tightened, making sure that the bolt head is still securely recessed into the frame.
 - (e) Place the tote under the seat of the walker, between the frame tubes. Secure the tote with the snap button flaps.
4. Be sure all attached hardware is tightened securely.



USER WARNING

– Read Immediately –



DO NOT USE YOUR WALKER AS A WHEELCHAIR OR TRANSPORT CHAIR.

Use seat for stationary seating only.

Never sit and push with your feet.

Do not go up or down stairs, use on escalators or on moving walkways.

Do not allow other to push you while seated.

USE and SAFETY INFORMATION

USER WEIGHT LIMIT: 500 lbs.

- Always consult with your physician or therapist to determine proper adjustment and usage. They should assist in the height adjustment for maximum support and correct brake activation.
- Be sure that both height adjustment knobs are fully tightened and handles are at equal heights. Test unit for stability before applying full weight to hand grips.
- Be sure the unit is fully open, seat is down and the hinged folding mechanism (*see page 2*) under seat is locked in the down position.
- After initial setup, care should be taken to ensure that all hand and height adjustments are secure, and that all wheels are moving freely before using this unit.
- Test the loop brakes before operating the unit. When wheels are locked the lock bar on rear wheels should prevent wheels from turning. If the rear wheels turn while brakes are locked, please take unit back to your dealer for adjustment. When loop brakes are released, the wheels should turn freely. (*See page 5 for Brake Operation.*)
- Always observe the weight limit. Check all labels are present and legible. Removal of serial number and product labels void warranty.
- Replace tires when worn or damaged.



USER WARNING

– Read Immediately –



Check front casters weekly for signs of damage.

**DISCONTINUE USE IF FRONT CASTERS LOOSEN
AND RETURN TO PLACE OF PURCHASE OR
CONTACT ESSENTIAL MEDICAL.**

USE and SAFETY INFORMATION *(continued)*

- If loop brake handles are exposed to extreme temperatures (above 100° F or below 32° F), high humidity and / or become wet, prior to using the unit, ensure handgrips DO NOT twist on frame handle - otherwise damage or injury may occur.
- All wheels MUST be in contact with the floor at all times during use. This will ensure that unit is properly balanced.
- To prevent tipping which may result in injury or damage:
 - (a) DO NOT attempt to reach objects if you have to move forward while seated.
 - (b) DO NOT hang anything from the frame of your unit. Items should be placed in the tote designed for your unit.
 - (c) The backrest should ALWAYS be in place and is NOT designed to support your total body weight.
 - (d) When reaching or picking up items place both feet securely on the floor and reach down between your knees.
 - (e) DO NOT add to your load when you are seated.



The above situations will cause a change of weight distribution and may cause the unit to tip, resulting in injury or damage. Always use EXTREME caution when reaching for any object.


- DO NOT use seat when unit is on an incline or uneven ground.
- Be sure you get proper instruction for use from your supplier.
- The tote has a weight limit of 10 lbs (5kg). Items placed in the tote should not protrude from the tote.

HAND BRAKE OPERATION

Always operate wheel locks on both sides at the same time.


■ To Slow or Stop Unit Temporarily

Pull up on the loops with the fingers until the walker slows or stops.
(The hand brakes can then be locked if the unit remains stationary.)

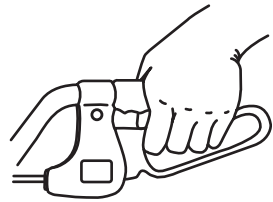

■ To Lock Wheels in Position

Stand between handles and press down on the rounded part of the loop with palm of hand. When you feel, and hear, “click” the wheels are locked.

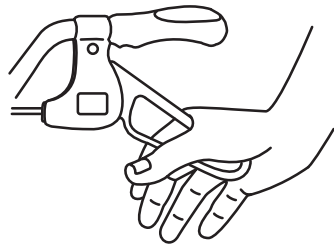

USER WARNING




Before sitting, test the walker to make sure it will not roll. Before sitting, touch the seat edge with your legs to ensure proper seating position onto the walker. Slowly lower your body on the seat.

■ When the walker is stopped, the hand brakes **MUST** be locked.

■ To Release Wheel Locks

Stand between handles and push up with the palm of hand on the bottom of loop to its original position.

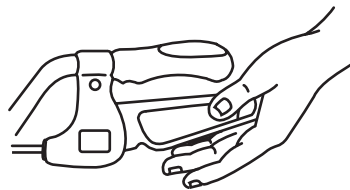

CLEANING, CARE and MAINTENANCE

Your new **ENDURANCE HD** BUILT TO LAST! is a mechanical device with moving parts and like all mechanical devices needs regularly scheduled maintenance. Much you can do easily, however, if you find problems in periodic checks, please return unit to place of purchase for service or repair.

CLEANING:

Wipe frame with a damp cloth to remove dirt and dust as needed.

STORAGE:

Keep unit indoors overnight and do not expose to salt air or harsh environments. If exposed, inspect for damage and clean before using.

PERIODIC CHECKS:

Units used on a daily basis should be checked weekly for:

- 1. Front Wheel (caster) function:**
 - (a) Turns easily.**
 - (b) Is completely secure in the frame.**
 - (c) Is free of hair, lint and other debris that could interfere with free wheel operation**
2. There are no cracks in the frame, tubing and / or welds.
3. Inspect all wheels to see they are in good condition with no significant wear, cracks or damage.
4. Verify operation of the brakes, wheel should not turn when brakes are engaged. Have them adjusted if necessary. Contact your dealer or place of purchase for adjustment.
5. If the handgrip is loose, do not use. Contact your place of purchase.
6. Replace any broken, damaged or worn items immediately.

ESSENTIAL MEDICAL SUPPLY, INC. LIMITED WARRANTY

Essential Medical Supply, Inc. (“Essential”) warrants its products free from defects in materials and workmanship (“Warranty”) for a Limited Lifetime from the date of purchase (“Warranty Period”). The Warranty extends only to the original purchaser of Essential products. If within the Warranty Period there is a breach of Warranty as to an Essential producer, such product shall be repaired or replaced at Essential’s sole discretion. This Warranty does not include any labor or shipping costs incurred in replacement, part installation, or repair of any Essential product. ESSENTIAL’S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO REPAIR AND/OR REPLACEMENT OF THE DEFECTIVE ESSENTIAL PRODUCT.

For Warranty service, please contact the dealer from whom you purchased your Essential product. In the event you do not receive satisfactory Warranty service, please write Essential directly at:

Essential Medical Supply, Inc.
Customer Service Department
6420 Halzeltine National Drive
Orlando, FL 32822-5121

Please provide the dealer’s name, address, and date of purchase, and indicate the nature of the problem and the Essential product serial number.

Upon receipt of written notification, Essential will issue a return authorization. The defective unit or parts must be returned to Essential for Warranty inspection using the serial number as identification within thirty (30) days of the date of return authorization. DO NOT return products to Essential without Essential’s prior written consent. Cash on delivery shipments will be refused and shipping charges must be prepaid.

Limitations and Exclusions: The Warranty shall not apply to products if 1) the Essential serial number has been removed or defaced; 2) the product has been damaged by negligence, accident, improper operation, maintenance or storage; and/or 3) the product has been modified without Essential’s express written consent or repaired by an unauthorized third-party (“Warranty Disqualifying Events”) include but are not limited to the addition of unauthorized parts or attachments or damage caused by circumstances beyond Essential’s control, the occurrence of which shall be determined exclusively by Essential. The Warranty also shall not apply to problems or defects arising from normal wear and tear on Essential products and parts, including but not limited to rubber tips and hand grips, and failure to adhere to user instructions and safety precautions.

THE WARRANTY STATED HEREIN IS THE SOLE AND EXCLUSIVE WARRANTY OFFERED BY ESSENTIAL. THERE ARE NO OTHER WARRANTIES RESPECTING ESSENTIAL’S PRODUCTS, EITHER EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO ANY WARRANTY OF DESIGN, MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE, EVEN IF ESSENTIAL HAS BEEN INFORMED OF SUCH PURPOSE. NO AGENT OF ESSENTIAL IS AUTHORIZED TO ALTER OR EXCEED ESSENTIAL’S WARRANTY OBLIGATIONS AS SET FORTH HEREIN. ESSENTIAL SHALL NOT BE RESPONSIBLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL, OR CONSEQUENTIAL DAMAGES ARISING OUT OF USE OF ESSENTIAL PRODUCTS.

Product Serial No. _____

NOTE: Warranty voids if Serial No. label is removed or destroyed.