


Note: The most current version of these instructions can be found online at www.grahamfield.com

- ⚠ WARNING: Important! Read and understand these instructions before installing or using the 2032A Sure-Safe Bathtub Rail. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the 2032A Sure-Safe Bathtub Rail is not properly assembled and adjusted, personal injury and damage to the Sure-Safe Bathtub Rail could result.**
- ⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Sure-Safe Bathtub Rail.**
- ⚠ WARNING: This product has a weight limit of 250 lb.**
- ⚠ WARNING: This product is NOT recommended for use on fiberglass or plastic bathtubs—do NOT over-tighten. GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

INSTALLATION

1. Ensure that the tub surface where the Bathtub Rail will be installed is smooth, clean, and dry before installing the Bathtub Rail. Remove any dust, powder, or foreign particles to ensure proper installation.
2. Use the hand knob to loosen the mounting bar until it is wide enough to fit your bathtub.
3. Place the Bathtub Rail in the desired location on the bathtub.
4. Turn the hand knob clockwise until the Bathtub Rail is secure on the bathtub.
5. Test installation security by giving a strong pull to the Bathtub Rail in a lengthwise direction to the bathtub. If the Bathtub Rail moves, reset it and tighten it more.

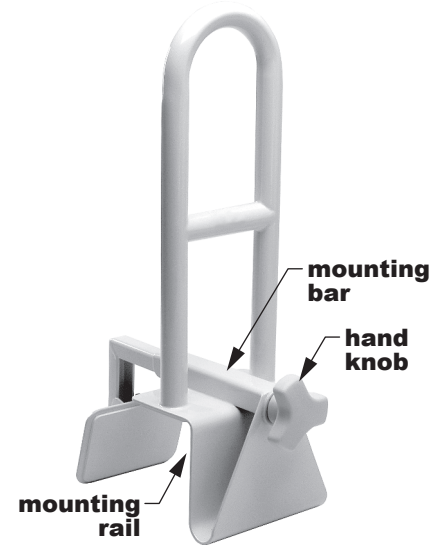

**2032A Sure-Safe[®]
Bathtub Rail**

▲ Caution: Do not over-tighten the Bathtub Rail.

- ⚠ WARNING: Ensure that the Sure-Safe Bathtub Rail is installed as described. If the Sure-Safe Bathtub Rail is not properly assembled and installed, personal injury and damage to the Sure-Safe Bathtub Rail could result.**

OPERATION

- ⚠ WARNING: Always ensure the Sure-Safe Bathtub Rail is secure before use.**
- ⚠ WARNING: Hold the Sure-Safe Bathtub Rail firmly when entering or exiting the bathtub.**

MAINTENANCE

⚠ WARNING: Check the Sure-Safe Bathtub Rail weekly for signs of damage or cracking. If damage or cracking of any component of the Sure-Safe Bathtub Rail is found, DO NOT use the Bathtub Rail. Contact your Graham-Field provider.

Inspect the Sure-Safe Bathtub Rail weekly and tighten it as needed to ensure proper installation.

To clean the Sure-Safe Bathtub Rail, wash it with soap and water or a mild detergent, then rinse. Ensure that the Sure-Safe Bathtub Rail is rinsed well before use.

WARRANTY

GF Health Products, Inc. warrants the Lumex 2032A Sure-Safe Bathtub Rail as follows:

Limited lifetime warranty against manufacturer's defects. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.



GRAHAM-FIELD.

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice.

Graham-Field, Lumex, and Sure-Safe are registered trademarks of GF Health Products, Inc.

© December 2008 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0601 • www.grahamfield.com