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LUMEX[®]
A Graham-Field Brand

700176C: FOLDAWAY ONYX WALKER ASSEMBLY AND OPERATION INSTRUCTIONS

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE

SAFETY GUIDELINES - PLEASE READ BEFORE USE

- ⚠ **WARNING: Important: Read and understand these instructions before assembling or using the Fold-Away Onyx Walker. Ensure that the Fold-Away Onyx Walker is assembled as described in the following paragraphs before use. If the Fold-Away Onyx Walker is not properly assembled, personal injury and/or property damage could result.**
- ⚠ **WARNING: Do not use this product without proper instruction from a healthcare professional.**
- ⚠ **WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**
- ⚠ **WARNING: If components are damaged or missing, contact your Graham-Field distributor immediately. DO NOT use substitute parts.**
- ⚠ **WARNING: Before using the Fold-Away Onyx Walker, ensure that the release mechanisms are locked in the open position, release levers are lowered, all legs are locked in place in walker frame, and that all leg-frame assembly and leg height adjustment buttons protrude through holes.**
- ⚠ **WARNING: Always use caution when using this product on wet or slippery surfaces.**

ASSEMBLY

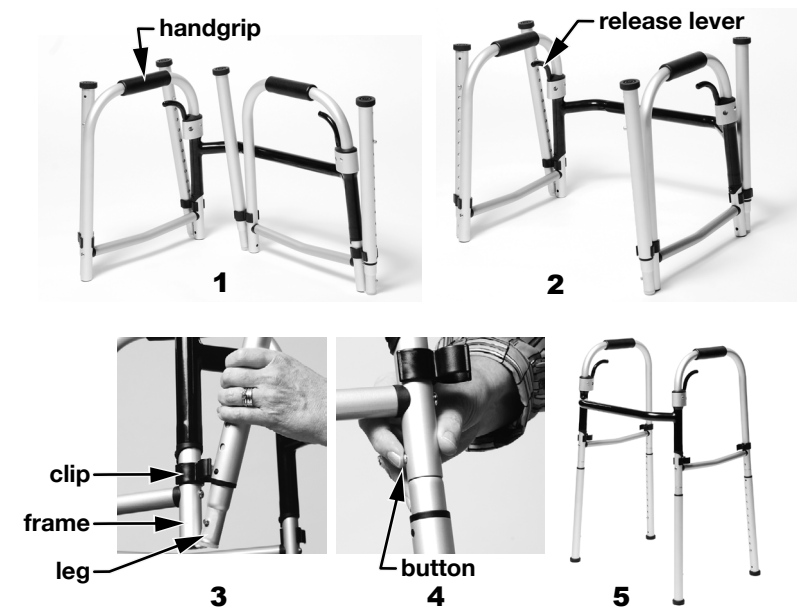
Remove walker from box. The walker is packed in collapsed position, as shown at right.

Note: Walker legs are assembled with elastic shock cord (bungee cord), so that the legs will not detach from the frame when the walker is collapsed.

Open walker

1. Grasp one handgrip and pull away from center of walker.
2. Repeat with other handgrip. Push handgrips away from each other until the release mechanisms lock into place (you will hear a click to confirm that the release mechanisms are locked into position) and the release levers are lowered.
3. Engage legs: Remove each walker leg from the built-in clip on walker frame; keep clip on walker frame.
4. Insert leg tubing into walker frame, press the push button, and secure leg in walker frame. Repeat with all legs until walker is in full assembled position.
5. Ensure that release mechanisms are locked in the open position, release levers are lowered, all legs are locked in place in walker frame, and that all four leg buttons protrude through frame holes.

Note: Adjust walker height before operation.

Adjust walker height

Each walker leg has several height adjustment positions. The picture at right shows the hole locations and the push button. To adjust leg height, depress the height adjustment button and move the leg to the desired height. Ensure that all four legs are adjusted to the same height, and that push buttons protrude fully through height adjustment holes.

OPERATION

Collapse walker for storage or transport

1. Disengage legs: Press frame leg assembly push button and pull walker leg out from frame (the elastic shock cord will prevent legs from detaching from frame). Rotate leg upward and secure in frame clip. Repeat for other three legs.
2. Pull upward on release lever release and hold. Grasp the nearest handgrip and push toward center of walker. Repeat for other handgrip. To open walker, follow instructions on previous page.

Dimensions

Width inside handgrips	Width inside rear legs	Overall width	Depth front to rear	Depth folded	Minimum height	Maximum height	Approximate user height	Maximum weight capacity	Product weight
16.65"	19.7"	22.8"	16.5"	3.75"	32"	41"	5'4" - 6'8"	300 lb	6 lb
423mm	502mm	580mm	420mm	95mm	812mm	1,040mm	163mm - 203mm	136kg	2.75kg

Replacement parts

Contact your Graham-Field distributor to order replacement parts.	Item number	Description	Unit
	700176C-C	Replacement Walker Clip	1 each
	700176C-T	Replacement Walker Tip	1 pair

MAINTENANCE

Wipe walker weekly with a damp cloth. To clean: As needed, wipe walker with a clean cloth and mild soap and water solution. DO NOT use organic solvents, such as acetone, which could damage the handgrips. Never use abrasive cleansers or scouring pads on any part of walker. Check all hardware weekly to ensure that it is securely attached. Replace walker tips as needed. To guarantee the best service from your walker, always specify Graham-Field replacement parts, available from your Graham-Field distributor.

LIMITED WARRANTY

GF Health Products, Inc. offers a Limited Lifetime Warranty for workmanship and materials on the FoldAway Onyx Walker. This warranty excludes accidental damage, improper handling, and normal wear and tear. During the warranty period, defective items will be repaired or replaced at our option at no charge.

GF, Graham-Field, and Lumex are registered trademarks of GF Health Products, Inc. Packaging, warranties, products, and specifications are subject to change without notice. GF Health Products, Inc. is not responsible for typographical errors.

© GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0601 • www.grahamfield.com

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