

LIFETIME LIMITED WARRANTY

Quality is the hallmark of all Lumex® products. Trusted by healthcare professionals and end-users for over 50 years, Lumex features a full line of patient aids and specialty healthcare seating.

Lumex products are designed and built to precise standards and are carefully inspected prior to shipment. We are dedicated to providing products that will give our customers years of dependable service.

This Lumex product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This warranty does not cover product failures due to normal wear and tear or device misuse. The warranty does not extend to non-durable components, such as rubber accessories, which are subject to normal wear and require periodic replacement.

Should a product defect that is covered by this warranty arise, we will, at our option, repair or replace the product.

For additional information on this Lumex product or this warranty, please contact an authorized Graham-Field Health Products Dealer.



All illustrations and specifications in this manual are based on the latest product information available at the time of publication. Graham-Field reserves the right to make changes at anytime, without notice.
© 2005, Graham-Field Health Products, 2935 Northeast Parkway, Atlanta, Georgia 30360.
Printed in China. GF0500033RevB05

LUMEX®

For The Quality Of Life®



INSTRUCTIONS & WARRANTY INFORMATION

Bath Stool
2060A

IMPORTANT: READ BEFORE USE

INSTRUCTIONS FOR USE – PLEASE READ ALL INSTRUCTIONS BEFORE USE.

SET-UP

1. Remove seat from carton

ADJUST HEIGHT

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
2. Repeat for other 3 legs.

⚠ **CAUTION:** Ensure that all four legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.



MAINTENANCE

- Use a mild soap and water solution, rinse with clear water, then dry with a soft cloth. Ensure that product is well rinsed before using.
- Never use organic solvents, abrasive cleaners or scouring pads on any part of product.
- Check fit and tightness of all nuts, bolts and knobs weekly to ensure that product is safe to use.
- To guarantee the best service from your product, always specify Lumex replacement parts, available from your Graham-Field Health Products dealer.

SAFETY GUIDELINES – PLEASE READ BEFORE USE

- ⚠ **CAUTION:** This product should not be used without proper instruction from a healthcare professional.
- ⚠ **CAUTION:** This product has a maximum user weight limitation of 300 lbs.
- ⚠ **CAUTION:** Graham-Field Health Products assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- ⚠ **CAUTION:** If components are damaged or missing, contact your Graham-Field Health Products dealer immediately. DO NOT use substitute parts.