

Note: The most current version of these instructions can be found online at www.grahamfield.com

- ⚠ WARNING: Important! Read and understand these instructions before installing or using the Versaframe Toilet Safety Frame. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the Versaframe is not properly assembled and installed, personal injury and damage to the Versaframe could result.**
- ⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and damage to the Versaframe.**
- ⚠ WARNING: This product has a weight limit of 250 lb.**
- ⚠ WARNING: Users who have limited physical capabilities must be assisted or supervised when using the Versaframe.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly, installation or use of this product.**

INSTALLATION

Unpacking

Remove Versaframe from carton and protective packaging. Check for obvious damage to the carton or its contents. If damage is evident, do not use the product; please notify the carrier and your Graham-Field authorized distributor for further instruction.

Installation

1. Remove existing toilet seat.
2. Place mounting bracket on flat surface of bowl and align holes. Ensure that the tube on the bracket is to the rear, closest to the tank.
3. Replace toilet seat, inserting bolts through mounting bracket holes and into toilet seat mounting holes. Replace original nuts and tighten.
4. Depress push buttons and install arms in mounting bracket as shown.
5. Adjust width of Versaframe arms by engaging push buttons in desired holes in mounting bracket tube.

- ⚠ WARNING: Ensure that mounting bracket push buttons are locked in place before use.**



Versaframe toilet safety frame

Height Adjustment

1. Adjust height by depressing the lower leg height adjustment button, twisting the lower leg, and moving leg to desired height.
2. When the desired height is reached, twist the lower leg until you hear the height adjustment button click into place.

⚠ WARNING: Ensure that height adjustment push buttons are locked in place, and both legs are adjusted to the same height, before use.

⚠ WARNING: Ensure that the Versaframe is installed as described. If the Versaframe is not properly assembled and installed, personal injury and damage to the Versaframe could result.

OPERATION

⚠ WARNING: Always ensure the Versaframe is securely installed before use.

MAINTENANCE

⚠ WARNING: Inspect the Versaframe weekly and tighten it as needed to ensure secure installation. Check all rubber tips for tears, rips or wear. If any of these conditions exist, replace tips IMMEDIATELY.

▲ Caution: Never use abrasive cleaners or scouring pads on any part of the Versaframe.

To clean the Versaframe, wash it with soap and water or a mild detergent, then rinse it with water.

WARRANTY

GF Health Products, Inc. warrants the Lumex 6460A / 6460R / 6465A-1 Versaframe Toilet Safety Frame as follows:

Limited lifetime warranty against manufacturer's defects. If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.



GRAHAM-FIELD.

GF Health Products, Inc. is not responsible for typographical errors. Packaging, warranties, products and specifications are subject to change without notice. Graham-Field and Lumex are registered trademarks of GF Health Products, Inc.

© March 2009 GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0601 • www.grahamfield.com