




User Guide 8805

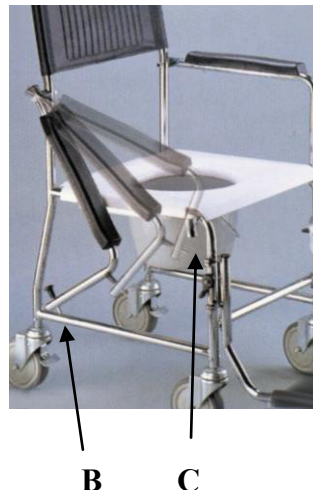

1. Inset both sides of backrest into tubing (A)
2. Make sure both arms are in downward position
3. Pull up on arm until the push button (B) clicks into the designated hole (C)
4. Pull on black knob to release armrest into downward position
5. To remove the commode bucket, lift off the white commode seat completely and slide out the bucket towards the rear of the seat
6. To put on the footrests, place both top and bottom pin holes over the designated pins towards the side (90 degrees from the front of the chair), swing the footrest towards the front until it clicks into place
7. To take off the footrests, push the small lever and allow the footrests to swing out 90 degrees, then lift directly upwards to remove
8. Before sitting, set the locking castors (wheels) by pushing down the silver metal tab on both sides
9. When the person seated is stable, unlock the castors by popping up the silver metal tab and roll the commode to the desired location - be especially careful over any threshold, mat, or uneven surface to not tip the person forward by moving too quickly
10. When you have reached your location, lock the castors again before the person attempts to stand up
11. Black seat pad (not shown) can be placed on top of white commode seat

LIMITED LIFETIME WARRANTY

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product. NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent. This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to nondurable components, including but not limited to rubber tips, handgrips, casters, and other parts subject to normal "wear and tear". THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.