



ORTHO-MED, INC.  
Commitment to Caring

## User Guide – 8450 Bedside Commode

**Weight Capacity: 350 lbs**



### Instructions Prior to Use

- 1) Attach commode seat and lid to the back bar. The clamps should snap onto the back bar. Lift the toilet seat and lid up and down to ensure they are properly attached.
- 2) Adjust the legs to the desired height. Make sure all four legs are set at the same height.
- 3) Make sure the Bedside Commode is set on a flat and level floor.
- 4) Remove the pail lid prior to use.

### How to Use

Over a toilet -

- 1) Remove bucket and lid from the Bedside Commode.
- 2) Place splash guard under the commode lid.
- 3) Lift up the toilet seat and lid.
- 4) Place Bedside Commode over the toilet.

As the toilet -

- 1) Make sure the bucket is placed under the commode lid.
- 2) Remove the bucket lid.
- 3) Make sure toilet seat is lowered.

### Care and Maintenance

- 1) Check regularly to make sure the back and legs are securely set.
- 2) Clean regularly with a toilet sanitizer or cleanser.
- 3) Discard bucket contents immediately after use.
- 4) Clean bucket immediately after use.

### LIFETIME LIMITED WARRANTY

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product. NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent. This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, handgrips, casters, and other parts subject to normal "wear and tear".

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.