

# 9016A

## Shower Bench with Arms Assembly Instructions



### Parts out of the box:

#### 1. Leg posts and Back Tubes



#### 2. Seat




#### 3. Legs



#### 4. Screws




1

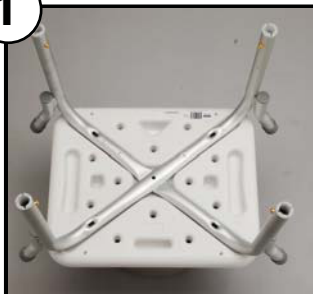

Set the seat on an elevated surface face down and place the leg marked 2 into the diagonal inset on the bottom of the seat. (The #2 on the seat should line up with the #2 label on the tube. Repeat for tube labeled #1 so that tube #2 is under tube #1.

2

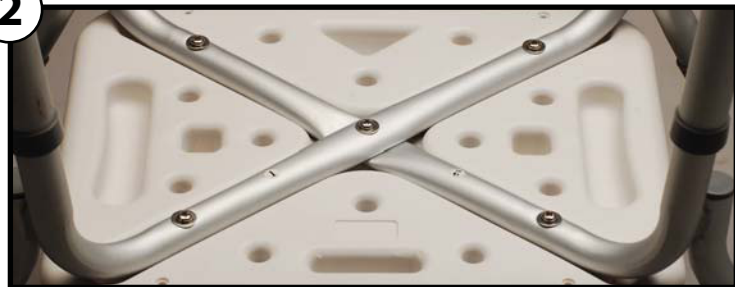

Hand tighten 5 screws with washers into the legs so that the screw goes through the tube and into the seat bottom then tighten all screws securely with a screwdriver.

3



Attach the legs by telescoping the adjustment leg over the leg stem and pushing the push pin in. Be sure to adjust all four legs to the same adjustment holes.

4



Set the bench on the floor and insert the arms by depressing the brass push button and inserting each side into the arm stems. Be sure the pins are fully engaged by pulling the arm.

# 9016A

## Shower Bench with Arms Assembly Instructions *(cont.)*



Products for an *independent* life



Use the push buttons on the legs to adjust the bench to the desired height.

**BE SURE THE BENCH IS LEVEL AND STABLE BEFORE USING.**



### **IMPORTANT**

**READ THESE DIRECTIONS CAREFULLY BEFORE USE.  
FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**WEIGHT CAPACITY NOT TO EXCEED 275LB.**

### **LIFETIME LIMITED WARRANTY**

Nova Ortho-Med, Inc. warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If within the specified warranty period this product is proven to be defective, it will be repaired or replaced at NOVA O.M.'s option. This warranty does not cover shipping costs incurred in replacement or repair of this product.

NOVA O.M. stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products. Do not return any products to our office without prior consent.

This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components, including but not limited to rubber tips, hand grips, casters, and other parts subject to normal "wear and tear."

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA O.M. SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.