SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Dealer/Carrier for further instructions.

SAFETY SUMMARY

The following recommendations are made for the safe installation and use of the WALKER TRAY on Single Release or Dual Release walkers:

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

DO NOT carry or place weight exceeding five (5) pounds in or on the walker tray when attached to walker. This may cause the walker to tip forward.

NEVER use the walker tray, walker basket and the walker pouch at the same time.

NEVER use the walker tray as a means of support.

DO NOT sit or push down on the walker tray when in use. This may cause the tray to collapse.

Always test to see that the walker and attachments are properly and securely locked in place before using.

NOTE: Not recommended for use with heavy duty walkers.

INSTALLING THE TRAY SIDE BRACES (FIGURE 1)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

NOTE: There is NO right or left tray side brace.

NOTE: BEFORE installing tray side braces onto the walker for the first time, soak them in hot water for five (5) minutes.

1. Make sure the openings in the tray side braces are pointing in towards the center of the walker.

NOTE: The tray side braces attach differently on dual release walkers than single release walkers.

2. Perform one (1) of the following:

DUAL RELEASE WALKERS - Position the openings in the tray side braces ABOVE and BELOW the pivot brace.

SINGLE RELEASE WALKERS - Position both openings in the tray side braces above the pivot brace.

3. Snap the tray side braces onto the walker at the points determined in STEP 2.


NOTE: If necessary, use a rubber mallet to snap the tray side braces onto the walker.

4. Perform one (1) of the following:

DUAL RELEASE WALKERS - Proceed to INSTALLING THE O-RINGS ONTO THE TRAY LINKS in this instruction sheet.

SINGLE RELEASE WALKERS - Proceed to INSTALLING SNAP SUPPORTS - SINGLE RELEASE WALKERS ONLY in this instruction sheet.

STEP 1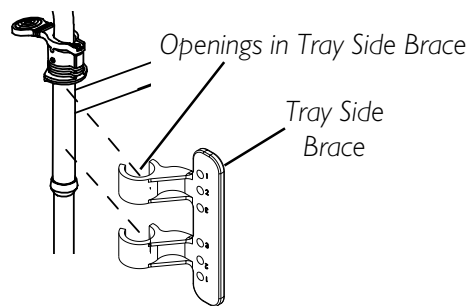

STEP 2 - POSITIONING THE SIDE BRACES

DUAL RELEASE WALKERS

SINGLE RELEASE WALKERS

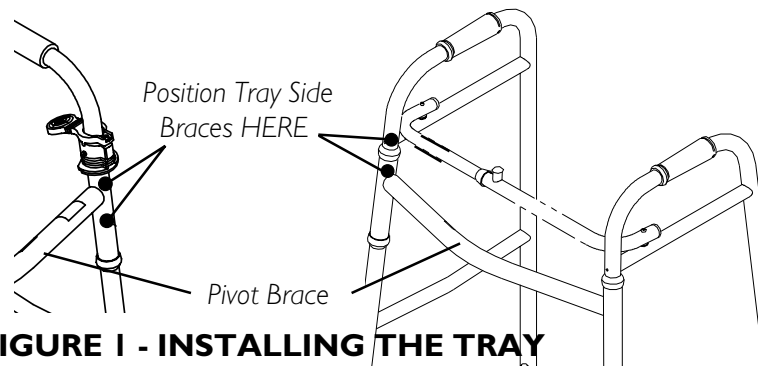
FIGURE 1 - INSTALLING THE TRAY SIDE BRACES

INSTALLING SNAP SUPPORTS - SINGLE RELEASE WALKERS ONLY (FIGURE 2)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Locate the upper hole in the tray side brace that is marked "2".
2. On the **INSIDE** of the tray side brace, attach the snap support into the location determined in STEP 1. Refer to FIGURE 2.
3. Repeat for opposite tray side brace.
4. Proceed to INSTALLING THE O-RINGS ONTO THE TRAY LINKS in this instruction sheet.

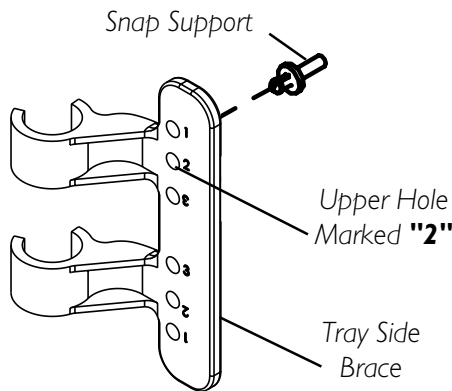
FIGURE 2 - INSTALLING SNAP SUPPORTS - SINGLE RELEASE WALKERS ONLY

INSTALLING THE O-RINGS ONTO THE TRAY LINKS (FIGURE 3)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Install the four (4) O-rings over the ends of the walker tray links as shown in FIGURE 3.

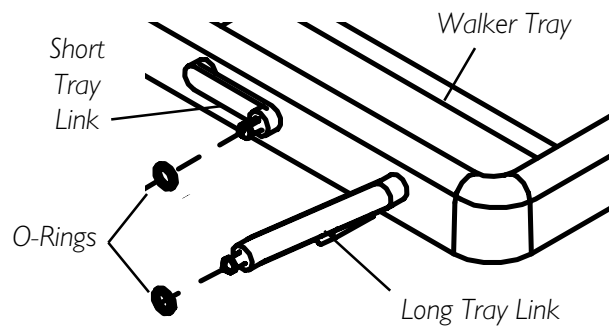
FIGURE 3 - INSTALLING O-RINGS ONTO THE TRAY LINKS

ATTACHING THE WALKER TRAY TO THE TRAY SIDE BRACES (FIGURE 4)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Place the walker tray onto the walker with the walker tray links positioned towards the front of the walker as shown in FIGURE 4.
2. Insert and lock the tray links into the following holes of the tray side brace as shown in FIGURE 4.

NOTE: There are two (2) sizes of tray links. Refer to the chart below and FIGURE 4 to determine tray link size.


SHORT TRAY LINK POSITIONING	
FOR DUAL RELEASE WALKERS	FOR SINGLE RELEASE WALKERS
SECOND HOLE FROM TOP (MARKED "2")	TOP HOLE (MARKED "1")

LONG TRAY LINK POSITIONING	
FOR DUAL RELEASE WALKERS	FOR SINGLE RELEASE WALKERS
FIFTH HOLE FROM TOP (MARKED "7")	FOURTH HOLE FROM TOP (MARKED "3")


3. Repeat STEP 2 for the opposite tray links.

STEP 1

NOTE: Single Release Walker is not shown for clarity.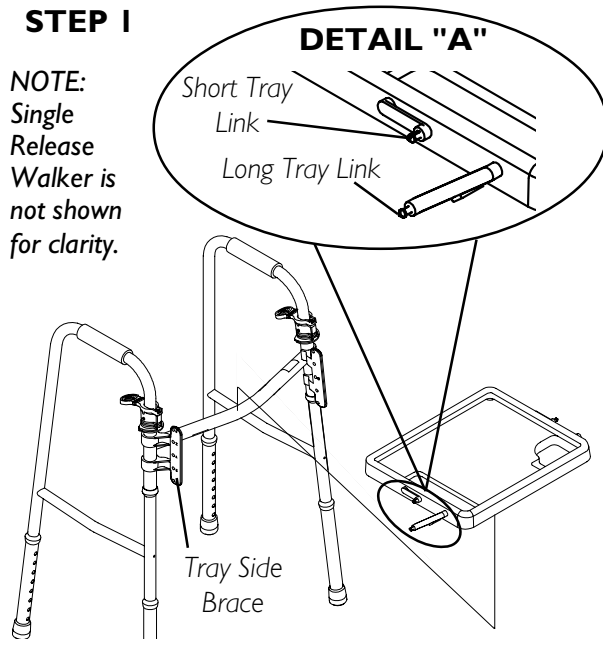

STEP 2 - DUAL RELEASE WALKERS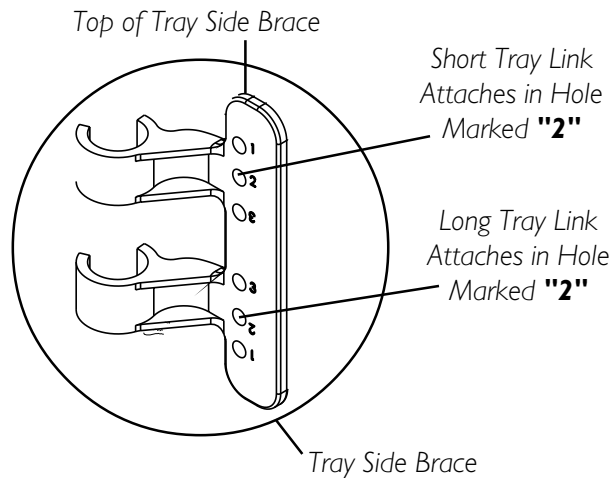

STEP 2 - SINGLE RELEASE WALKERS

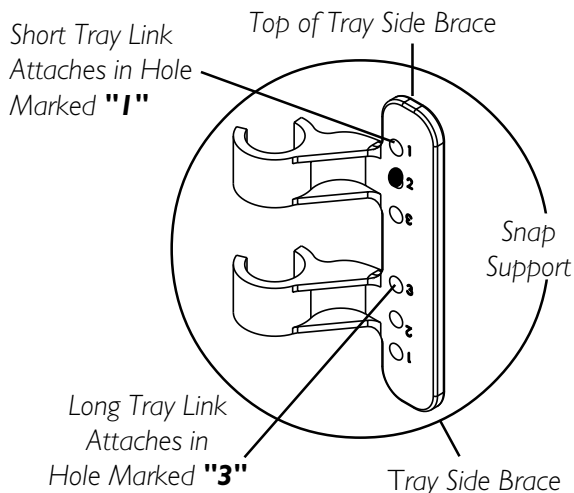
FIGURE 4 - INSERTING TRAY LINKS INTO TRAY SIDE BRACES

ADJUSTING THE WALKER TRAY (FIGURE 5)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

STORING THE WALKER TRAY

1. Pull the back of the walker tray UP and then push forward and down on the back of the walker tray to store.

NOTE: The walker tray will lie flat against the walker pivot brace when storing.

PREPARING TO USE THE WALKER TRAY

1. Pull the back of the tray UP and pull IN on the back of the tray until the tray is level.

REMOVING THE WALKER TRAY FROM THE WALKER

1. Grasp the BOTTOM of the tray side braces and pull up and out.

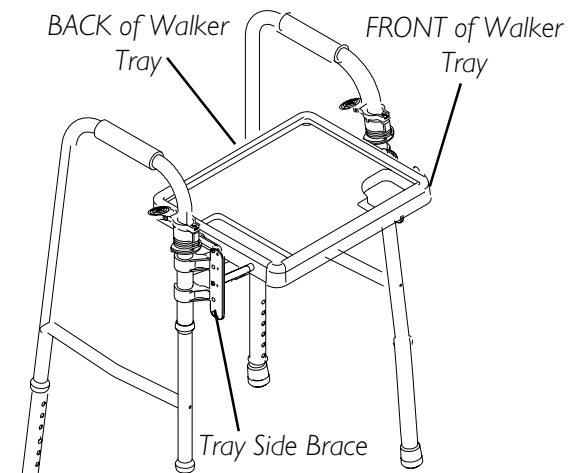
FIGURE 5 - ADJUSTING THE WALKER TRAY

CLEANING THE WALKER TRAY

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

NOTE: DO NOT use an abrasive detergent/cleaner or cloth to clean the walker tray.

1. Use only non-abrasive detergent/cleaner with warm water.
2. Rinse with warm water and wipe dry.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the lifetime of the product for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2003 Invacare Corporation
Invacare is a registered trademark of Invacare Corporation.
Yes, you can. is a trademark of Invacare Corporation.
Invacare Corporation www.invacare.com