drive  
WWW.DRIVEMEDICAL.COM

universal aluminum  
folding crutches

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EU Authorized Representative  
Drive Medical LTD  
Aimley's Industrial Estate  
Elland, West Yorkshire  
United Kingdom HX5 9JP

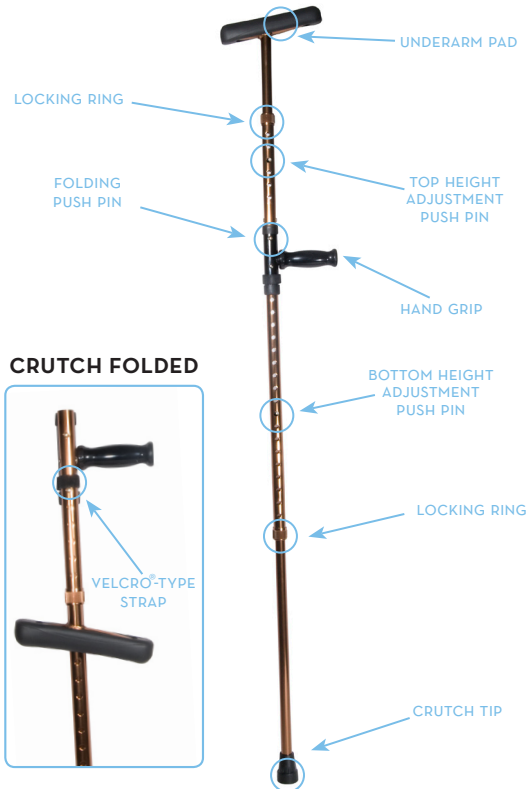
drive  
WWW.DRIVEMEDICAL.COM

## assembly instructions

For maximum fit and proper assembly, follow these instructions:

1. Remove the velcro®-type strap.
  2. Insert the top tube (with underarm pad) into the bottom tube (with black hand grip) by depressing the two (2) brass push pins.
- Insert first leg into the leg receptacle by depressing the two brass push pins. When brass push pins “snap” through the adjustment hole, the tube is locked into position.

**NOTE:** *There are two (2) height adjustments*

## height adjustments

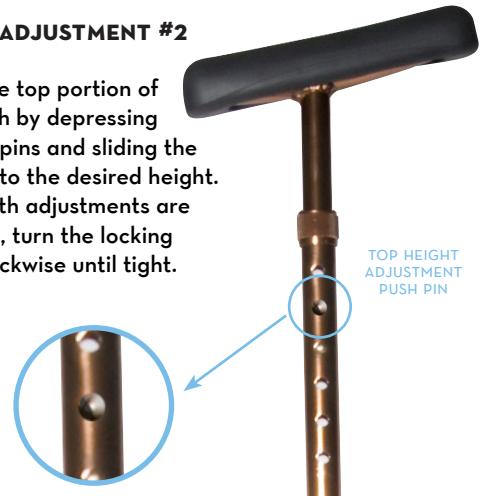
### HEIGHT ADJUSTMENT #1

Adjust the bottom tube of the crutch by depressing both push pins and pull the bottom tube to the desired height (the correct height is when your arm has an approximate 10 degree bend below the elbows where grasping the hand grip).

### HEIGHT ADJUSTMENT #2

Adjust the top portion of the crutch by depressing the push pins and sliding the top tube to the desired height. When both adjustments are complete, turn the locking collar clockwise until tight.

**NOTE:**

*Store crutches in carry bag (INCLUDED) when not in use*

## cautions

### CAUTIONS:

- Use after proper training under the direction of your Doctor or Physical Therapist.
- Replace Crutch Tips if worn.
- Use extra caution on wet and slippery surfaces.

### SAFETY CHECK:

Confirm patient can lean forward comfortably while resting on the underarm supports.

*Patient should not have to stand uncomfortably erect.*

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

**drive**  
[WWW.DRIVEMEDICAL.COM](http://WWW.DRIVEMEDICAL.COM)

© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

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