

# lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*WenzeliteRe/hab*

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

## posterior safety roller

item #'s

CE 1200 BK, PE 1200 and PE TYKE 1200



EU Authorized Representative  
Drive Medical LTD  
Ainley's Industrial Estate  
Elland, West Yorkshire,  
United Kingdom HX5 9JP

*WenzeliteRe/hab*

[www.drivemedical.com](http://www.drivemedical.com)

drive  
MEDICAL DESIGN & MANUFACTURING

## set up instructions

1. Remove Safety Roller from carton.
2. Unfold Safety Roller by lifting ring from left upright and open frame. Lift handlebar and position over left upright. Make sure spring button snaps into hole on the handlebar underside.

## height adjustment

Recommended handlebar height is at the wrist with arms fully extended. With your hand placed on the hand grip, your elbow should be flexed 20° - 30°.

1. Open both height adjustment knobs.
2. Push in spring button, one side at a time and raise or lower uprights to desired height.
3. Tighten knobs.

## folding instructions

1. Press spring button on left side of handlebar and lift handlebar.
2. Hold handlebar in right hand and with left hand, swing left upright outward and squeeze into Z position until fully folded.
3. Drop handlebar over folded Safety Roller.
4. Secure both the uprights by putting the ring over the left upright and under the spring button.

## accessories

Adjustable forearm platforms, vertical hand grips, variable resistance tabs on rear wheels, swivel wheel locking brackets, seat harness, ankle prompts, posterior positioning bar, pelvic stabilizer, trunk support, extended uprights for additional height, IV pole and oxygen tank holder.

Call an authorized Drive dealer for detailed information about our accessories.

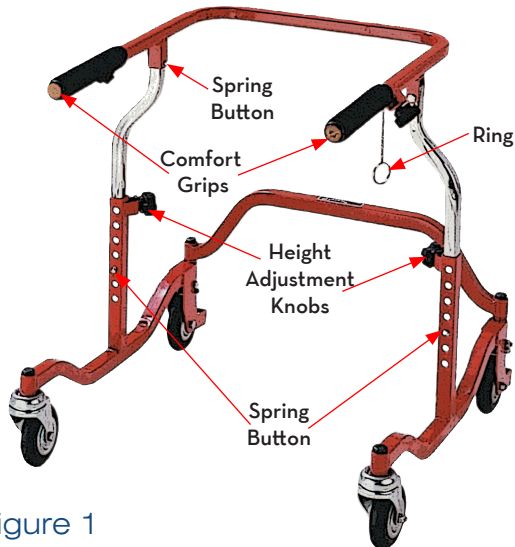


figure 1



figure 2  
folded position