

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2012 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

Wenzelite
nimbo lightweight
posterior posture walker



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP


www.drivemedical.com

WenzeliteRe/habTM
drive

user instructions

1. Remove Nimbo from carton.
2. To unfold the Nimbo, pull handgrips and cross brace in opposite directions. Continue to unfold until the push pins “click” into place.
3. To install the legs, match the leg numbers with the corresponding frame numbers. Depress the push pins on the legs and slide into the frame. When locked into position, the push pins will “CLICK” through the frame.
4. Adjust height of walker by pressing push pins on each leg and slide to desired position.


FIGURE A

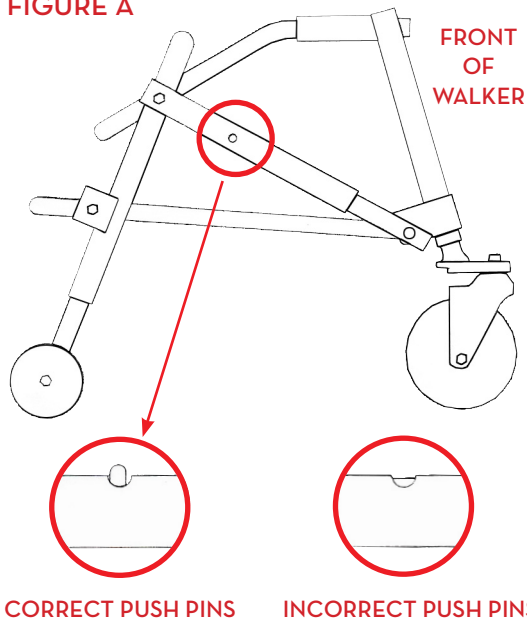

user information


1. Lift and position knob (A) to set front wheels in either the swivel or locked position.

FIGURE B

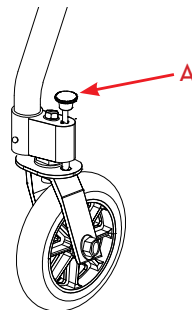

2. To fold the Nimbo, face it from the open end, depress the push pins, tilt walker so that it rests on its rear legs only and push the front legs toward the rear legs.
3. To override the one-directional feature of the rear wheels, lift up aluminum pin and insert into override bracket

FIGURE C

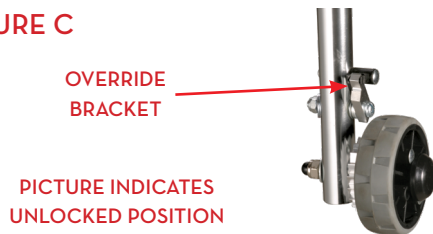

maintenance and care

1. Inspect the Roller regularly. Tighten the nuts and bolts as necessary. If a product needs service or replacement parts, contact an authorized Drive dealer.
2. To clean the Roller, use warm water with a mild detergent. Wipe clean with a soft, dry cloth.