

three year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for three years from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This Three Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



*Wenzel*ite*Re/hab*

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

seat

item # KA 1285

for use with nimbo, tyke
(item # KA 1200)



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

*Wenzel*ite*Re/hab*

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

installation

1. Remove bolts (A).
2. Drop outside frame connecting tube (B)
3. Lift horizontal U-tube (E).
4. Slide seat mounting brackets (C) onto horizontal U-tube and push towards the rear.
5. Tighten knobs (D) under seat mounting brackets.
6. Reinstall bolts (A).

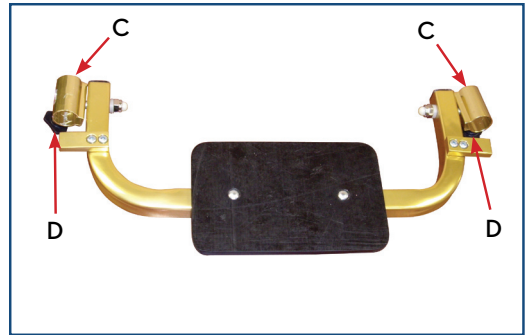


figure 2

operation

Flip up seat when standing and walking and fold down seat for sitting and resting. Seat should be in the up position when folding walker.



figure 3

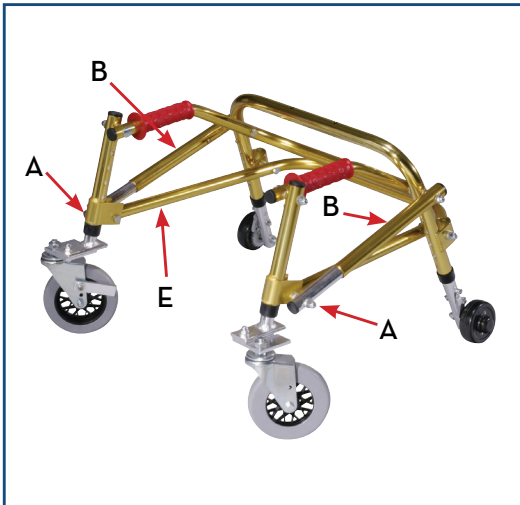


figure 1