

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2012 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

wheeled, drop arm
commode



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JD

www.drivemedical.com

driveTM

commode assembly instructions and precautions



FIGURE 1

assembling commode

1. Remove contents from box.
2. Attach front cross brace to top of wheeled leg section by depressing push button and sliding into place. A snap will be heard when cross brace is in place. Repeat for back cross brace.
3. Attach second wheeled leg section to cross braces.
4. Insert back support and arm section into frame. Tighten knobs.
5. To drop arm, depress release button on arm and gently allow to drop until reaching arm stop.
6. Attach lower frame support and tighten knob.
7. Attach footrests to front of frame using nuts and bolts provided.
8. Install commode bucket by placing it in grooves in bucket frame.
9. Align the toilet seat clamps onto the back frame rail. (Both clamps should sit outside the rail brackets). Push down on seat until clamps snap firmly into place.
10. When removing the seat grasp the back of the seat and lift.

additional precautions

1. Toilet seat must be in down position before use. Failure to do so may result in breakage of commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
3. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
4. Ensure ALL screws, nuts and/or bolts are tightened.
5. Users with limited physical strength should be supervised or assisted while using commode.
6. Use the arm supports only for assistance. DO NOT attempt to use the arms to support full body weight.
7. Arms should be unlatched and dropped to assist in transfer onto/off the device