Basic Back

Jay® Basic Back

Owner's Manual

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
Le Dossier Jay® Basic

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JAY®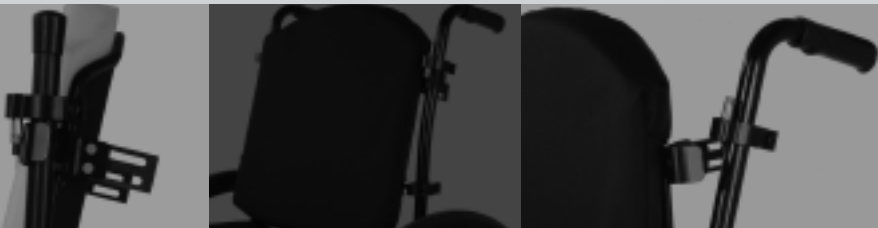

 **SUNRISE
MEDICAL**
www.sunrisemedical.com

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HOW TO FIT THE JAY BASIC BACK

The Jay Basic Back is a versatile back designed to replace the sling upholstery when ordered with either strap or clamp style mounting hardware. When properly utilized the back should:

- Offer you support and comfort
- Promote trunk alignment
- When used as a replacement for your existing sling upholstery offer moderate shear reduction.
- Maximum user weight: 250 lbs. (113 kg.)

CAUTION– Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury. Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

Determining Back Height

Measure your wheelchair post width from the outside of one post to the outside of the other post. Verify that you have the proper width Jay Basic Back for your chair. Begin by removing armrests and any upholstery from your wheelchair's upright posts. Place whatever cushion you plan to use into your wheelchair (preferably a Jay® Basic or Jay® Soft Combi® Cushion).

TOP HOOK INSTALLATION AND SELECTION

INSTALLING THE TOP HOOK TO THE BACK SHELL


The Jay Basic Back ships with two mounting hooks for the top portion of the hardware, a 0° and 8° recline. Some clients may find that being seated in 8° of recline increases their comfort and balance.

The Jay Basic Back is designed to replace the wheelchairs sling upholstery. The initial installation should be done without the foam insert on the back shell.

INSTALLING THE TOP HOOKS TO THE SHELL OF THE JAY BASIC BACK (FIGURE 1)

Once a hook is selected you will need to attach the hook to the back.

1. Place the mounting plate through the front section of the shell (A). Count the number of holes exposed to ensure the back is adjusted evenly on both sides.
2. Align the slots of the hooks with the mounting plate you have just placed through the front of the shell (B).
3. Adjust width to match the wheelchair post width. Ensure that the back is adjusted evenly on both sides.
4. Secure the hook by using the washer and nut provided in the hardware kit contents (C and D).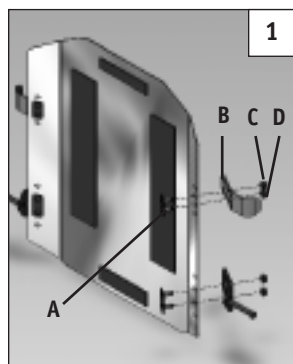


If you have selected a Jay Basic Back with clamp-style hardware to mount the back shell to the wheelchair please see instructions CLAMP HARDWARE INSTALLATION. If you have selected a Jay Basic Back with strap-style hardware to mount the back shell to the wheelchair please see instructions STRAP HARDWARE INSTALLATION.

STRAP HARDWARE INSTALLATION

INSTALLING THE BOTTOM POST STRAP (Figure 2)

Attach one bottom post bracket to each upright post of the wheelchair.


1. Determine the proper holes for different wheelchair post diameters. (The bracket comes assembled to attach to 7/8" - 1" (2.2cm - 2.5cm) posts.)
 - For 7/8" - 1" (2.2cm - 2.5cm) posts, use outside holes (A).
 - For 5/8"-3/4" (1.6cm-1.9cm) posts, use inside holes (B). Remove caps (C) and insert screws (D) into both inside holes (B).
2. Wrap the band around the wheelchair post. Screw "E" may have to be loosened to align strap hole with barrel nut. Place a washer on the screw and insert through the proper hole.
3. Insert screw "D" into the side of the bracket and tighten firmly. Ensure that the screw engages the insert inside the bracket.
4. Place caps (C) over both screw heads.
5. To adjust the height of the bracket, loosen screw "E" until bracket slides freely on wheelchair post. Once proper height of the back is determined, tighten screw "E". All future adjustments of the back height are done with this screw.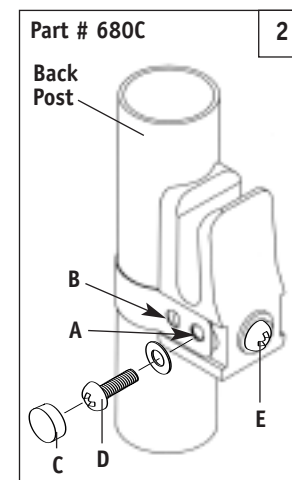


INSTALLING THE TOP QUICK RELEASE QUARTER TURN BRACKET HARDWARE (Figure 3)

Attach one top quick release quarter turn bracket to each upright post of the wheelchair.

1. The top quick release quarter turn bracket is designed to fit on 5/8", 3/4", 7/8" and 1" (1.6, 1.9, 2.2 and 2.5 cm) diameter back posts. Determine the proper holes to use for different wheelchair posts diameters. (The bracket comes assembled to attach to 7/8" or 1" (2.2 or 2.5 cm) posts.)
 - For 7/8" - 1" (2.2cm - 2.5cm) posts, use outside holes (A).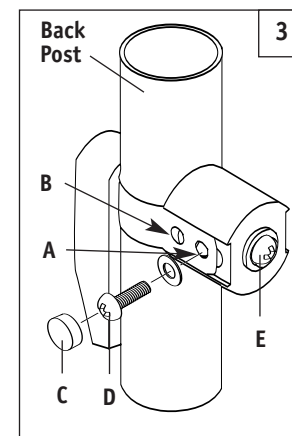

- For 5/8"-3/4" (1.6cm-1.9cm) posts, use inside holes (B). Remove caps (C) and insert screws (D) into both inside holes (B).
2. Wrap the strap around the wheelchair post. Screw "E" may have to be loosened to align the strap hole with the barrel nut located inside the bracket body. Place a washer on screw "D" and insert the screw through the appropriate hole in the strap.
 3. Tighten screw "D" into the barrel nut through the side of the bracket body. Tighten the opposite screw if necessary.
 4. Snap the screw caps over both screw heads.
 5. To adjust the height of the bracket, loosen screw "E" until the bracket slides freely on the back post. Once the proper height of the back is determined, tighten the screw. All future adjustments of the back height are done with screw "E".
 6. Always turn the quick release quarter turn bracket to the "DOWN" position (Figure 4) to secure the top hook in place.

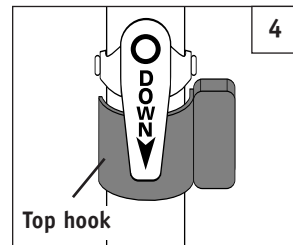CAUTION- The top quick release quarter turn brackets must be in their downward position when the back is in use. Improper use may cause the back to unexpectedly detach from the wheelchair.


NOTE- The back is properly secured when the bottom pins are resting in the lower post brackets and the upper quick release quarter turn brackets are installed above the back support's top hook brackets with the quick release mechanism pointing down.

CLAMP-STYLE HARDWARE INSTALLATION

INSTALLING THE BOTTOM POST CLAMP (Figure 5)

Required tools: 3/16" Allen wrench


1. Remove the screw and nut (E) securing the clamp.
2. Two spacers (F) are provided for installing the clamp. If required, install the spacer onto the backrest tube.
 - If installing the backrest on 1.125" (2.9 cm) diameter backrest tube, no spacer is required.
 - If installing the backrest on 1" (2.5 cm) diameter backrest tubes, the 1" spacers are required.
 - If installing backrest on 7/8" (2.2cm) diameter backrest tubes, the 7/8" spacers are required.
3. Install the clamp onto the backrest tube with the post cradle (G) facing upward, as shown in the figure 2. Install the clamp (around the spacer if required), so that the holes on the spacer are aligned with the holes on the clamp.
4. Install the screw through the hole on the clamp. Thread the nut onto the screw.
5. Orient the clamp as shown in the diagram, so that the post cradle faces the front of the backrest tube.
6. Tighten the screw using a 3/16" Allen wrench. Tighten the clamp just enough to prevent it from sliding on the tube. Do not completely tighten until the correct height location of the back is established (see instructions for height adjustment).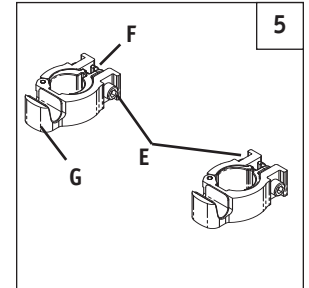

INSTALLING THE TOP DROP PIN CLAMP (Figure 6)

Required tools: 3/16" Allen wrench

1. Follow steps one through four above.
2. Orient the clamp as shown in figure 6, so that the drop pin faces the front of the backrest tube.
3. Tighten the screw (H) using a 3/16" Allen wrench. Tighten the clamp just enough to prevent it from sliding on the tube. Do not completely tighten until the correct height location of the back is established (see instructions for height adjustment).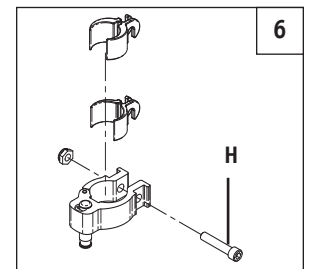


ESTABLISHING THE HEIGHT OF THE BACKREST

Required tools: 7/16" wrench, tape measure.

The Jay Basic Back must be mounted at the correct height on the backrest tubes. Sunrise Medical recommends that the initial fitting and positioning of the backrest take place under the supervision of a clinician, or a rehab technology supplier. The height of the backrest is altered by loosening the screws that secure the drop pin clamp/quick release bracket and the bottom post clamp/strap.

1. Establish the height of the backrest by loosening the screws (A) that secure the bottom post clamps/straps. Raise or lower the clamps/straps and retighten, checking that the backrest pan is at the proper height when the bottom posts are installed into the clamp/strap receiver saddles. Make certain that the left and right hardware is installed at the same height by measuring the height of each clamp.
2. Tighten the screws securing the bottom post clamps firmly.
3. Establish the correct height of the drop pin clamps/strap. Slide the pin in the receiver to the unlocked (raised) position. Slide the mounting hardware on the backrest tubes so that they reside just above the top hooks when the hooks cradle the backrest tubes. Allow for a small 1/16" (2mm) clearance between the top hooks and the drop pin clamps/quick release bracket hardware.
4. Tighten the screws firmly.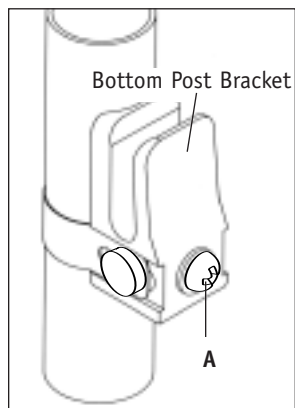

CHANGING THE HEIGHT POSITION OF THE BOTTOM POST AND TOP HOOK

Required tools: 7/16" wrench.

It is possible to change the height position of the bottom post or top hook on the backrest pan. Such adjustment is needed when either of these parts is located in a position that interferes with other components mounted to the backrest tube.


1. Remove the back foam from the backrest pan.
2. If the width of the bottom post brackets or top hook bracket has already been established (see instructions for width adjustment), mark their location using a pencil. This will assist in maintaining the correct width position when the brackets are moved to a different height.
3. Loosen and remove the nuts and washers securing the bracket and remove the bracket from the backrest pan.

4. Remove the mounting plate from the holes of the backrest pan, and reposition the mounting plate in the desired set of holes.
5. Install the bracket to the mounting plate, and secure the nuts and washers to the mounting plate.
6. Follow the instructions for "CHANGING THE WIDTH OF THE BOTTOM POST AND TOP HOOK".
7. Tighten the nuts firmly.

CHANGING THE WIDTH OF THE BOTTOM POST AND TOP HOOK

Required tools: 7/16" wrench.

The width of the bottom post and top hook brackets are adjustable, to accommodate various widths of backrest tubes, over a certain range.

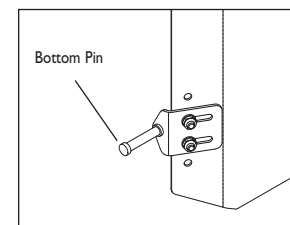Adjusting the bottom posts.

1. Loosen the nuts securing the left and right bottom posts with respect to the backrest pan. The nuts should be loose enough to allow the posts to slide along the slots in the post brackets.
2. Place the backrest against the wheelchair backrest tubes. Slide the posts inboard or outboard so that the posts will easily drop into the cradle of the bottom post clamps. Allow for a slight 1/16" (2 mm) clearance between the head of the post and the edge of the cradle. Make certain that the position of the left and right posts with respect to the backrest pan are the same, so that the pan is centered between the backrest tubes.
3. Tighten the nuts firmly.

Adjusting the top hooks.

1. Loosen the nuts securing the left and right top hooks with respect to the backrest pan. The nuts should be loose enough to allow the top hooks to slide along their slots.
2. Place the backrest against the wheelchair backrest tubes. Slide the hooks inboard or outboard so that the hooks cradle and are centered on the tubes. Make certain that the position of the left and right hooks with respect to the backrest pan are the same, so that the pan is centered between the backrest tubes.
3. Tighten the nuts firmly.

RELEASING THE BACKREST FROM THE WHEELCHAIR

The Basic Back is designed so that the backrest can easily be removed from the backrest tubes. The drop pins/quick release brackets must always be in their locked (lowered) position when the chair is in use. Improper use may cause the backrest to unexpectedly detach from the wheelchair. Never attempt to lift the wheelchair by lifting on the backrest pan or supporting hardware.

To remove the back

1. Slide the drop pins/quick release brackets to the unlocked (raised) position.
2. Remove the back by pulling forward and upward on the backrest pan.

To install the back

1. Insert the bottom posts into the post cradles. Rotate the backrest so that the top hooks cradle the backrest tube.
2. Slide the drop pins to the locked (lowered) position, / twist the quick release brackets down over the top so that the hooks are captured behind the pins/bracket.

USE AND MAINTENANCE INSTRUCTIONS

Your Jay Basic Back should require little or no maintenance. The aluminum shell may be wiped clean with soap and water. The cover should be machine washed and dried (gentle to moderate cycle) on a regular basis.

It is advisable to purchase a backup cover for use while your primary cover is being washed.

24 MONTH LIMITED WARRANTY

Each Jay Basic Back is carefully inspected and tested to provide peak performance. Every Jay Basic Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the Back's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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© 2005, Sunrise Medical 10.05
Sunrise Medical, Inc.