

To Clean the Cover

Remove the cover from the seat cushion. Machine wash in lukewarm water and air dry. No ironing is required. We do not recommend machine drying in order to prevent the breakdown of the moisture proof urethane laminate on the inside of the cover.

To Clean the Gravity Free™ Core

To clean the inner protective foam core simply spot clean or wipe the inner cover with moist warm cloth. Avoid submerging the base in water.

The Gravity Free™ Cushion Warranty

Under normal use from date of purchase, Hudson Medical guarantees The Gravity Free™ Cushion to be free from defects in material and workmanship for a period of 18 months on the foam core and 90 days on the cover. Should a defect occur, the cushion should be returned by the customer to the place of purchase with original sales receipt. Hudson Medical will determine if it is a manufacturing defect and replace or repair the cushion without charge. This warranty does not apply to damages due to misuse, punctures, tears, burns, etc. to the cushion or removable cover.

HUDSON INDUSTRIES, INC.
MEDICAL PRODUCTS DIVISION
Richmond, Virginia 23231 • 800-343-8112
www.hudsonindustries.com
1248863/1209

PRESSURE+EEZ®

GRAVITY FREE

3" Wheelchair Cushion

*For Moderate Pressure Sore
Management & Long-Term Care*



*Gel Foam Core
Protected from Incontinence*

User Instruction Manual & Warranty

HUDSON
MEDICAL PRODUCTS

SKIN PROTECTION – E2603

HUDSON

PRESSURE+EEZ®

GRAVITY FREE™

3" Wheelchair Cushion

A Uniquely Designed Lite Weight Gel Bladder Encased in Foam. Designed for Moderate Pressure Sore Management, Incontinency and Long-Term Care to Equalize Weight Pressure Distribution for Skin Protection.

The Gravity Free™ Gel Foam Core

The Gravity Free™ Cushion has a 3" thick resilient foam core embedded with a gel bladder constructed with a 1" thick foam insert. The air is vacuumed from the foam and replaced with a water based polymer in a 10-gauge vinyl bladder casing. The Gravity Free™ gel foam core provides weight and pressure distribution in the ischial and coccyx area for maximum skin protection.


The Gravity Free™ Foam Core Protective Liner

The Gravity Free™ gel foam core and bladder is protected from incontinence by a waterproof electronically sealed soft durable 10-gauge vinyl protective inner cover. Air vents are incorporated to reduce pressure and increase seating comfort.

The Gravity Free™ Cover

The Gravity Free™ gel foam core and bladder is further protected by a 2-way nylon stretch knit cover that reduces friction and shearing. The bottom of the cover is made of a non-skid vinyl.

PST SIZE RANGES	
Firmness	Cushion Width
Soft	10" to 14"
Medium	16" to 18"
Firm	20" to 24"

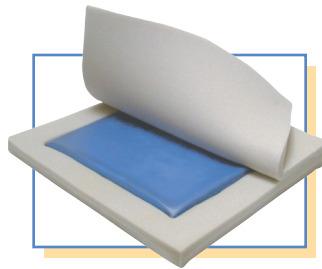

Use and Application of the Pressure EEZ® Gravity Free™ Wheelchair Cushion

The Gravity Free™ Cushion is ideal for long term care, moderate pressure sore management and incontinence. The foam core with inner gel bladder is protected by an electronically sealed inner vinyl cover for long term protection.

Remember, no cushion can eliminate seating pressure or totally prevent pressure sores, therefore, you should regularly check for skin redness as a noted indicator of tissue breakdown. The Gravity Free™ Cushion may be used in a sling seat wheelchair or on a solid seat base.

The Gravity Free™ Cushion has a moisture resistant cover designed to repel fluids as it is intended as a long-term incontinence cushion. However it should not be submerged in water or used as a bathtub seating product. Replacement covers are available from your Hudson dealer.



Placement of the Gravity Free™ Wheelchair Cushion

The Gravity Free™ Wheelchair Cushion is available in the most popular adult size (18" x 16" x 3"). The cushion should match the width of the hips of the client not the width of the chair. The cushion should be placed at the back of the wheelchair and the length should extend to 1" to 2" from the back of the knees. The ischial tuberosities (sitting bones) should be centered in the seat in the center of the back seating area.

The footrest should be adjusted properly in order to enhance seating comfort and help to lower peak seating pressures.

The Gravity Free™ Wheelchair Cushion is Available in One Size:

Item No.	Size	Units/Case	Medicare Code	Weight Capacity	Warranty
248863	18" x 16" x 3"	1	E2603	250	18 Months

Cover Warranty: 3 Months