Assembly Instructions

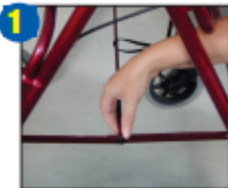
Instrucciones de Asamblea

Remove all parts from box and verify that you have received two (2) Handlebars with Loop-Lock Brakes, one (1) Basket, one (1) Backrest.

NOTE: If any parts are missing, do not return to the place of purchase. Contact Duro-Med Industries at 877-505-6190.

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- 1. To open your rollator, push the folding hinge down until it locks securely in a straight position.**

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- 2. Place both loose handgrip pieces, into the handlebar openings and adjust to the desired height. Remove the knob, washer and screw from the handlebar. Insert the screw, with the washer on it, through the desired hole aligned in the rollator bar and screw the knob back on securely.**

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- 3. Insert the backrest into rollator frame securely making sure the retainer pins are locked into position through the frame holes on both sides.**

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- 4. Test and adjust the tension on the braking system mechanism to ensure proper and secure braking before use.**

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- 5. Secure the basket in the proper location. Flip up the seat and place the basket on the rollator crossbars or position the basket hooks over the front bar (not shown).**

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