

LIMITATIONS AND EXCLUSIONS

Products which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond reasonable control are not covered by this warranty.

Parts or materials which are subject to normal wear resulting from the use of this product, and which must be replaced or repaired due to such normal wear, are the owner's responsibility and are not covered by this warranty.

This warranty is exclusive and is in lieu of all other express warranties, implied warranty, including but not limited to the implied warranty of merchantability and fitness for a particular purpose, and shall not extend beyond the duration of the express warranty provided herein. DMI will not be liable for consequential or incidental damages whatsoever.

This warranty gives you specific legal rights which may vary from state to state. Some states do not allow the exclusions or limitations of incidental or consequential damages, or how long an implied warranty lasts, so the exclusions and limitations may not apply to you.

WARNING! READ BEFORE USING

This unit is not intended to be used as a wheelchair. At regular intervals, check and tighten all fasteners and hinged parts. Check all plastic parts for wear often. Units are not designed to assist users from a sitting to a standing position. When stopping, do not lean excessively on the unit. The brakes are not designed to withstand excessive pressure. Excessive use causes the brake cables to stretch or fray and adjustment or replacement of the brake cable may be required. DMI rollators are not suggested for people with severely limited ambulation for whom a rigid walker may be recommended. Failure to observe this warning may result in injury.

CAUTION! CHECK BRAKES BEFORE USING

Always check your brakes before using your rollator. Keep your brake handles in excellent condition at all times. Always check for loose parts, tightness of the cables and cleanliness. It is best to keep your rollator fully assembled at all times.

Distributed by:

Duro-Med Industries
Div. of DMS Holdings Inc.
1931 Norman Drive South
Waukegan, IL 60085 USA
www.duromed.com

Made in China

91-032-3012 02/07

©2007 Duro-Med Industries