


# Shower Transport Chair

## WITH REAR WHEEL LOCKS

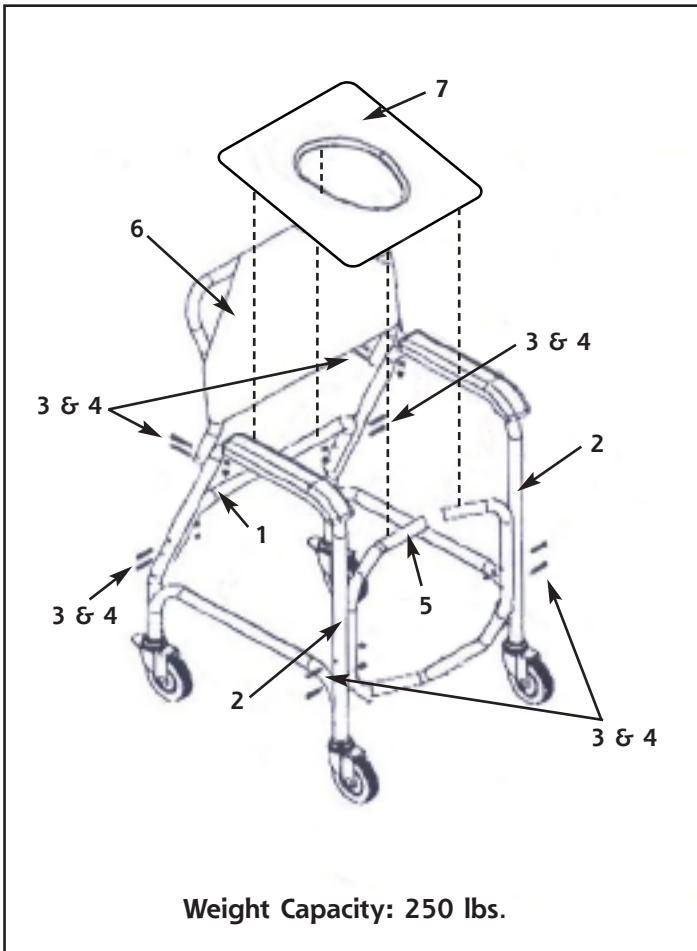

### Assembly Instructions

- ▶ 1. Secure the rear brace (1) to both side supports (right & left) (2) using two screws (3) and two nuts (4) on each side as shown.
- ▶ 2. Attach the front brace (5) by aligning the holes with those on the side supports (2). Insert two screws (3) and tighten with two nuts (4) on each side.
- ▶ 3. Align the holes on the backrest (6) with the holes on both side supports (2). Insert two screws (3) and secure with nuts (4) on each side.
- ▶ 4. Firmly push down on the seat (7) until it locks into place on both rear (1) and front (5) braces.
- ▶ 5. Tighten all screws securely using a screwdriver and pliers or wrench. **DO NOT OVER TIGHTEN AS IT COULD STRIP THE THREADS.**

**TOOLS NEEDED FOR INSTALLATION:** Screwdriver and Pliers or Wrench.

**CAUTION:** Make sure all screws and wing nuts are tightened securely. For your safety, the four extension legs must always be locked into the same height position. All pins must be pushed completely through the pinholes before using.

**NOTE:** If any parts are missing, do not return to the place of purchase. Contact us at 877-505-6190.

**Distributed by:**

Duro-Med Industries  
Div. of DMS Holdings Inc.  
1931 Norman Drive South  
Waukegan, IL 60085 USA  
www.duromed.com

Made in China

Toll-Free Customer  
Care Help Line  
877-505-6190

Monday - Friday, 8:00 am - 4:30 pm CST

#910-1702-1900 06/06

©2006 Duro-Med Industries

### Assembly Code

- ▶ 1- Rear Brace
- ▶ 2- Side Supports (Right & Left)
- ▶ 3- Screw
- ▶ 4- Nut
- ▶ 5- Front Brace
- ▶ 6- Backrest
- ▶ 7- Seat

### CAUTION

This product contains natural rubber latex which may cause allergic reactions.

**Safety**

**Security**

**Durability**