


Premium TUB GRAB BAR

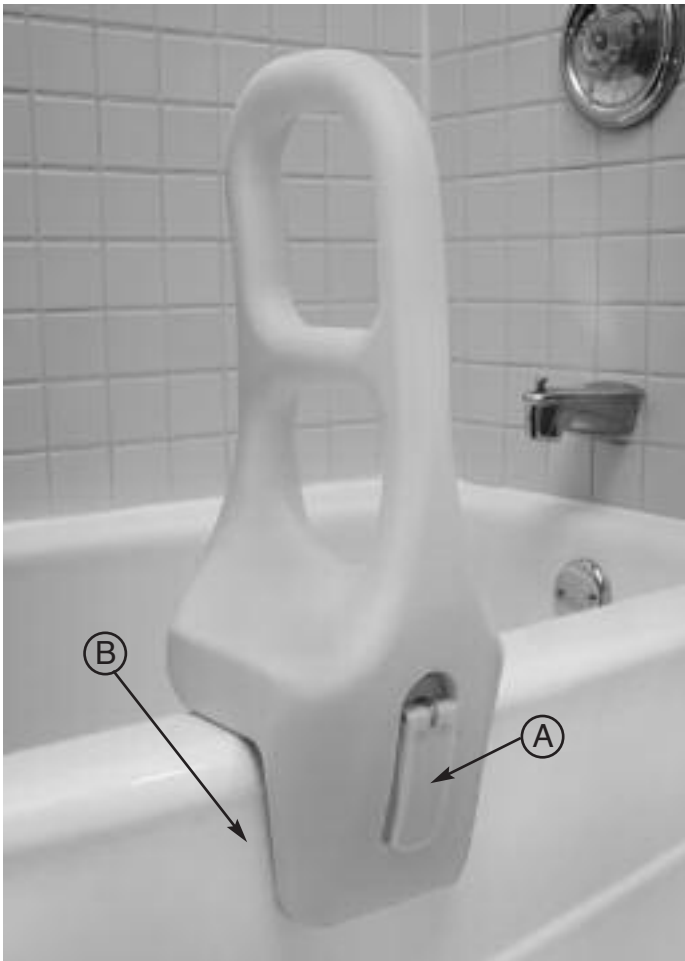

Weight Capacity: 250 lbs.

Installation Instructions

This tub grab bar fits 2-1/2" - 6-1/4" thick bathtub walls. Locate a section of the bathtub where the inner and outer walls are straight (B). **Do not mount the tub bar on a curved portion of the bathtub.** Be sure the desired location for mounting is dry, clean, and free of dirt before installing the grab bar.

- ▶ 1. To begin, fold out the adjustment handle (A) from the recess on tub grab bar.
- ▶ 2. Loosen the adjustment handle (A) to fit over the bathtub wall (B) by turning the handle counterclockwise. Place the tub grab bar onto the bathtub wall ensuring it is fully seated. The adjustment handle should be facing the outside of the bathtub.
- ▶ 3. Secure the tub grab bar by turning the adjustment handle clockwise until the bar is firmly locked onto the bathtub wall.

DO NOT OVER TIGHTEN AS IT COULD STRIP THE THREADS.

- ▶ 4. Once the tub grab bar is secure, fold the adjustment handle down until it locks back into the recess.
- ▶ 5. Before using, be sure to test it first by pushing and pulling on the bar. If it seems loose or moves, repeat steps 1-4.

Care and Maintenance

To clean, wash the tub grab bar with soap and water or a mild detergent. Be sure the tub grab bar is rinsed well before use.

WARNING: Periodically check assembly for tightness. **NEVER USE TUB GRAB BAR IF ANY PARTS APPEAR TO BE LOOSE OR DAMAGED, AS PERSONAL INJURY MAY RESULT.**

NOTE: If any parts are missing, do not return to the place of purchase. Contact us at 877-505-6190.

For fiberglass or plastic bathtubs, we recommend using a wall-mounted grab bar.

Distributed by:

Duro-Med Industries
Div. of DMS Holdings Inc.
1931 Norman Drive South
Waukegan, IL 60085-6715 USA
www.duromed.com

Made in China

Toll-Free Customer
Care Help Line
877-505-6190
Monday - Friday, 8:00 am - 4:30 pm CST

#910-1607-1900 10/06
©2006 Duro-Med Industries

Safety

Security

Durability