
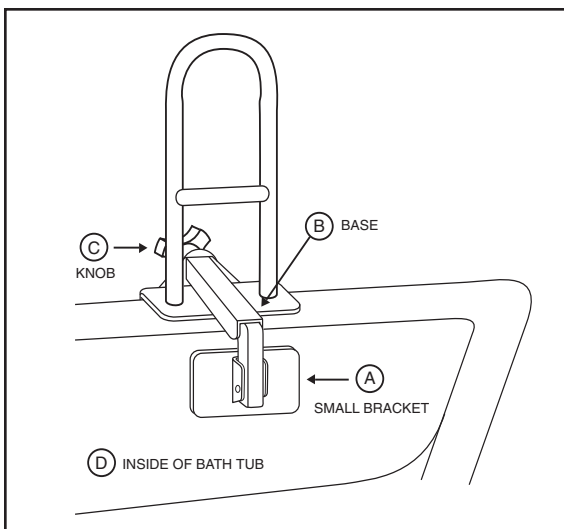
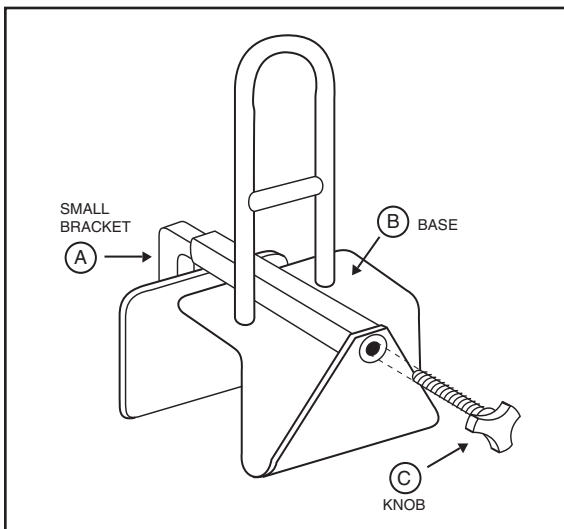


Easy Grip **ADJUSTABLE TUB BAR**

(NO TOOLS REQUIRED FOR INSTALLATION)

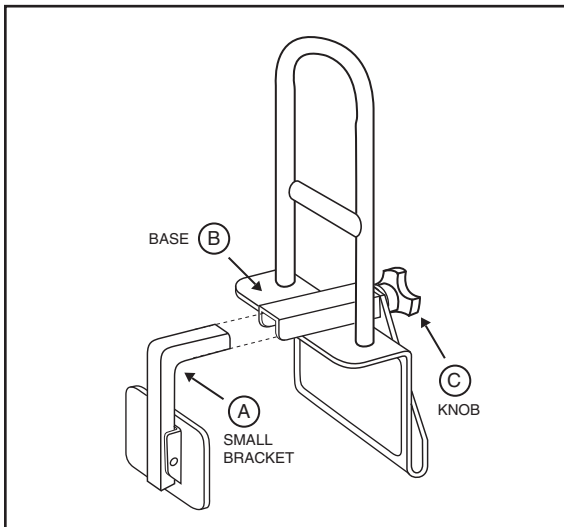

Installation and operation

- ▶ This item is designed to provide support and increase stability for a person weighing up to 250 lbs. This bar is NOT designed to support the total weight of an individual.
- ▶ Use the tub bar for assistance when entering or exiting the bathtub only. Use extreme caution on wet tub or floor surfaces. Users with limited physical abilities should be supervised or assisted when using this tub bar.
- ▶ 1. Thoroughly clean area of tub (D) where tub bar is to be attached.
- ▶ 2. Slide the small bracket (A) into the base (B) of the bathtub bar so that the padded area of the small bracket (A) is angled down.
- ▶ 3. Insert the knob (C) through the base (B) and fasten to the small bracket (A) by turning the knob (C) clockwise.
- ▶ 4. Lower the safety tub bar over the external wall of the bathtub so that the knob (C) is outside the bathtub. Secure in place by tightening the knob (C) (turn clockwise).
- ▶ 5. Try moving the safety tub bar to ensure it is securely fastened. If necessary, continue to tighten, but do not over tighten.

CAUTION: DO NOT USE UNTIL IT IS SECURELY FASTENED TO THE BATHTUB.

Care and maintenance

- ▶ Clean the safety bar periodically using a mild soap and water or a non-abrasive cleaner.
- ▶ Inspect the bar for looseness prior to each use. If loose, tighten the knob (C) until the tub bar is once again secure.
- ▶ Do not use if any damage to the tub bar is detected.
- ▶ **Note:** If any parts are missing, do not return to the place of purchase. Contact us at 877-505-6190.

Distributed by:

Duro-Med Industries
Div. of DMS Holdings Inc.
1931 Norman Drive South
Waukegan, IL 60085 USA
www.duromed.com

Made in China

Toll-Free Customer
Care Help Line
877-505-6190

#910-1609-1900 09/07
©2007 Duro-Med Industries