Instructions

Adjustable Shower Stool

Item # 522-9801-1900

Assembly Instructions

1. Place the round seat (4) upside down on a flat surface.
2. Align the support legs (1 and 2) to the seat. Using the screw knob (a) and a bent washer (b), hand tighten securely in the center of the seat.
3. Align the remaining holes on the seat with the remaining holes on the legs. Use the screw knobs (a) and straight washers (c) to secure the legs to the seat.
4. Adjust the extension legs in 1" increments to the desired height, and lock the pins into place.


Caution: Make sure all screw knobs are tightened securely. Do not overtighten as it could strip the threads. For your safety, the four extension legs must always be locked in the same height position. All pins must be pushed completely through the pinholes before using.

Note: If any parts are missing, do not return to the place of purchase, call 800.622.4714.

Warning: Do not use if any parts appear to be loose or damaged as personal injury may result.

Assembly Codes

- 1 – Support Leg (1)
- 2 – Support Leg (1)
- 3 – Round Seat (1)
- a – Screw Knobs (5)
- b – Bent Washers (1)
- c – Straight Washers (4)

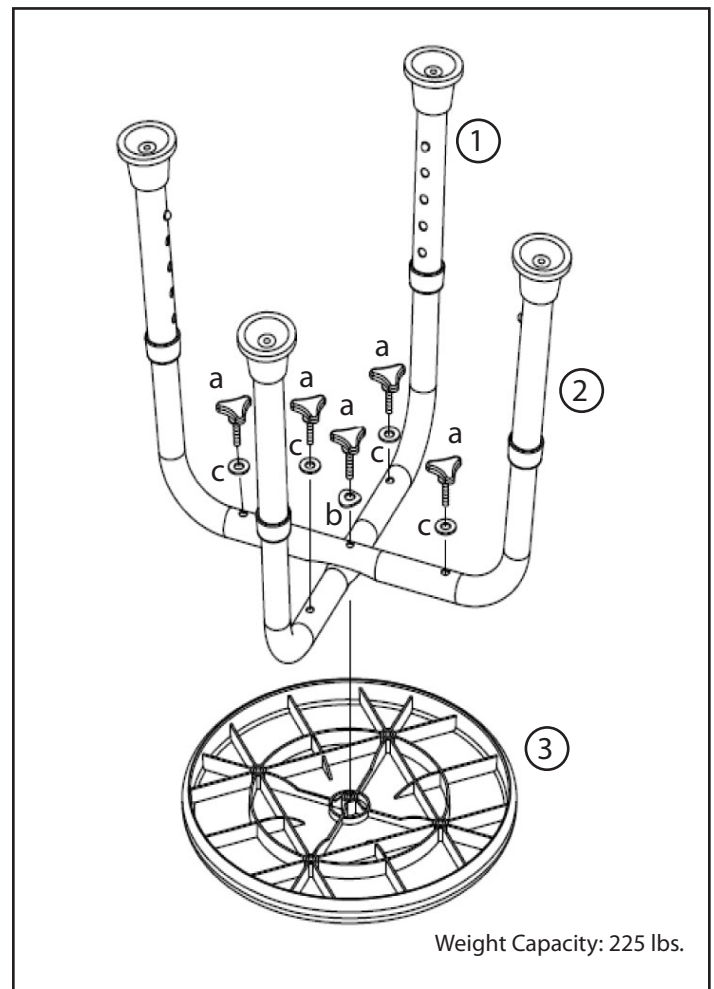CAUTION

This product contains natural rubber latex, which may cause allergic reactions.

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