# Instructions

## Adjustable Transfer Bath Bench

Item # 522-9821-1900


### Assembly Instructions

1. Align the cross-brace (I) between the two leg frames (H). Align the two holes on either side of the cross-brace (I) with the holes on the leg frames (H). Thread a bent washer (F) over a 2-1/8" short-thread bolt (C) and insert through the hole. Place a wingnut (G) on the end of the bolt and hand tighten loosely. Repeat for the remaining holes.
2. Repeat step one to attach the remaining cross-brace (I).
3. Lay the three bench panels (L) next to each other on the floor with the textured surface facing down. Position the assembled frame on top of the panels and align the holes in the panels with the holes in the frame. For the first panel (the one farthest from where the handrail will be), thread a flat washer (E) over a 1-3/8" screw knob (A) and insert the screw knob through the hole in the frame. Hand tighten. Repeat for the remaining hole on this panel.
4. Lay the long section of the backrest support (J) above the holes in the frame on the two remaining panels. Thread a flat washer (E) over a 2-1/4" screw knob (B) and inset the screw knob into the hole in the backrest support brace (J), then through the frame. Hand tighten. Repeat for the remaining holes on the two panels.
5. To attach the handrail (K), align the holes to the exterior surface of the frame. Note: The handrail should be mounted on the side of the frame closest to the backrest. Thread a bent washer (F) over a 2-1/4" long-threaded bolt (D) and insert through the hole. Fasten a wingnut (G) on the end of the bolt by hand. Repeat for the three remaining holes.
6. Hand tighten the wingnuts on all four bolts. To ensure proper placement, the handrail should be mounted on the exterior of the frame and should extend above the plastic bench seat panels.

**Caution:** Make sure all screw knobs are tightened securely. Do not overtighten as it could strip the threads. For your safety, the four extension legs must always be locked in the same height position. All pins must be pushed completely through the pinholes before using.

**Note:** If any parts are missing, do not return to the place of purchase, call 800.622.4714.

**Warning:** Do not use if any parts appear to be loose or damaged as personal injury may result.

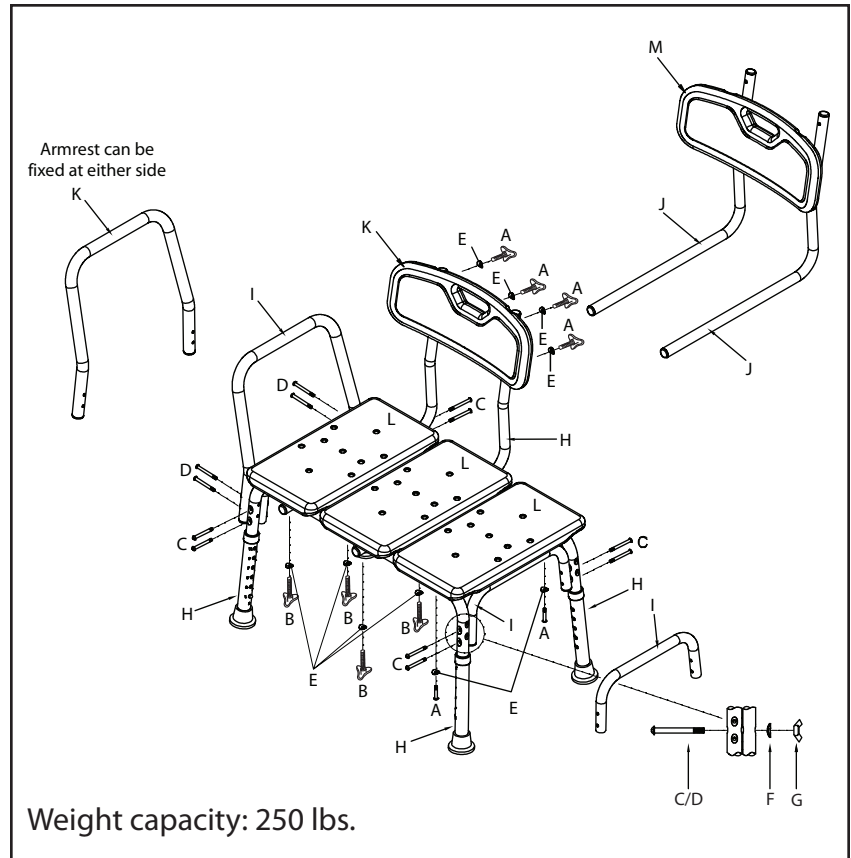**CAUTION**  
This product contains natural rubber latex,  
which may cause allergic reactions.

### Assembly Code

- A – 1-3/8" screw knobs (6)
- B – 2-1/4" screw knobs (4)
- C – 2-1/8" short-thread bolts (8)
- D – 2-1/4" long-thread bolts (4)
- E – Flat washers (10)
- F – Bent washers (16)
- G – Wingnuts (12)
- H – Leg frames (2)
- I – Cross-braces (2)
- J – Backrest support braces (2)
- K – Handrail (1)
- L – Plastic molded bench panels (3)
- M – Plastic molded backrest (1)

Manufactured for:  
Briggs Medical Service Company  
1931 Norman Drive  
Waukegan, IL 60085  
Made in China

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