GUARDIAN[®]
. S I G N A T U R E[™] .

LIFETIME
LIMITED
WARRANTY

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**ENVOY[®] 380,
THREE-WHEELED
ROLLING WALKER
USER INSTRUCTIONS
& WARRANTY**

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REF G07981T

WARNING!

FAILURE TO FOLLOW THESE INSTRUCTIONS MAY RESULT IN INJURY.

- Do not use your walker until the handles are properly adjusted and securely tightened.
- Press folding mechanism past center line for full engagement prior to use.
- Do not use this walker on hills. A wheeled walker may not provide adequate support on slopes.
- Do not exceed the maximum weight capacity listed on the walker.

ASSEMBLY:

INSTALL HANDLE BARS

1. Loosen the adjustment knobs.
2. Insert the handle into walker frame (Figure 1).
3. Stand inside the walker and move the handle up or down until it is in line with the bend in your wrist.
4. Tighten knobs securely.

INSTALL BASKET (Sold Separately)

1. Attach basket to the frame by placing the basket hooks toward the back inside walker frame (Figure 2).
2. Pull the basket towards the front of the walker until the hooks engage around the body frame and the basket rests on the folding mechanism.

USING YOUR WALKER

FOLDING YOUR WALKER


1. Push down on the center folding mechanism bar until the bar is past center line and fully engaged (Figure 3).
2. To unfold, place all wheels on the ground. With one hand, pull-up on the center folding mechanism bar.

LOCKING AND UNLOCKING YOUR WALKER'S BRAKES

1. To lock brakes, push down on brake loop handle until it "clicks" into the locked position (Figure 4).
2. To unlock, pull up on brake loop handle.

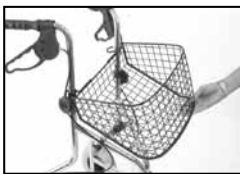(Figure 1)

(Figure 2)

(Figure 3)

(Figure 4)

Safety Instructions

- **DO NOT** use this product without first reading and understanding the instructions contained in this booklet. If you are unable to understand the warnings, cautions, or instructions, contact a healthcare professional, dealer, or technical personnel before use - otherwise serious bodily injury and damage may occur.
- Consult with your doctor, physical, or occupational therapist or other qualified professional to assure the proper height adjustment.
- **DO NOT** use the rollator as a wheelchair.
- **DO NOT** attempt to move the rollator while you or anyone is sitting on the seat.
- **DO NOT** self-propel the rollator while seated.
- Serious injury to you and/or damage to the rollator walker frame or wheels may result from improper use.
- **DO NOT** use the seat to transport people or objects. **DO NOT** use the seat to carry or move anything.
- **DO NOT** sit on the rollator while it is being loaded into a vehicle or while it is being transported in a vehicle.
- **DO NOT** exceed the maximum weight capacity (including bag contents) or serious injury could result: **Maximum weight capacity: 250 lbs (113 kg)**
- Before each use make sure the rollator frame is fully opened.
- Check the brakes for proper operation. Make sure that all parts are secure and that moving parts are in good working order. Wheels should roll and not wobble.
- All wheels must be in contact with the floor and/or ground at **ALL TIMES**. This will ensure the rollator is properly balanced.
- The brakes **MUST** be in the locked position **BEFORE** sitting on the seat.
- **DO NOT** sit on the walker if the ground is sloped or uneven.
- **DO NOT** use the rollator to walk backwards, down gradients or to climb stairs, curbs, or to go over obstacles. Serious risk of fall or injury may occur.
- **DO NOT** perform any adjustments to the rollator while it is in use.
- Only use accessories and spare parts authorized by Medline Industries, Inc.
- **DO NOT** push rollator while brakes are enabled. Brakes are used to keep the wheels from rolling. But it may not prevent the rollator from moving if it is pushed.
- **DO NOT** use the rollator to provide assistance while rising from a lower position. If you use it to pull yourself up to a standing position, the rollator may move even if the brakes are in the locked position and this could result in a fall.
- Inspect the handle bar grips prior to use. Assure the plastic parts are still functional if they are exposed to extreme temperatures (above 100°F (38°C) or below 32°F (0°C) high humidity and/or become wet.
- **DO NOT** twist hand grips as they are not meant to be twisted. Failure to do this could result in a fall.
- **DO NOT** hang anything from the frame of the rolling walker other than the basket included or the tote bag accessory. The basket has a weight limitation of 11 lbs (5 kg). The tote bag has a weight limitation of 10 lbs (4.5 kg). Items should be placed completely in the basket or tote bag and should not protrude. This may cause the rollator to tip, resulting in bodily injury or rollator damage.
- The backrest is intended to provide back support as you sit on the seat. The backrest is not intended to support full weight. Avoid leaning while sitting as this may result in fall.

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LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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Medline Canada
1-800-396-6996
www.medline.ca | Canada@medline.com

Medline Mexico
01-800-831-0898
www.medlinemexico.com | mexico@medline.com

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2010 Medline industries, Inc. Envoy is a registered trademark and Guardian Signature is a trademark of Medline Industries, Inc.

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- Utilice únicamente accesorios y repuestos autorizados por Medline Industries, Inc.