# DN7065 HOME CARE® BY MOEN® TOOL-FREE TRANSFER BENCH INSTRUCTIONS

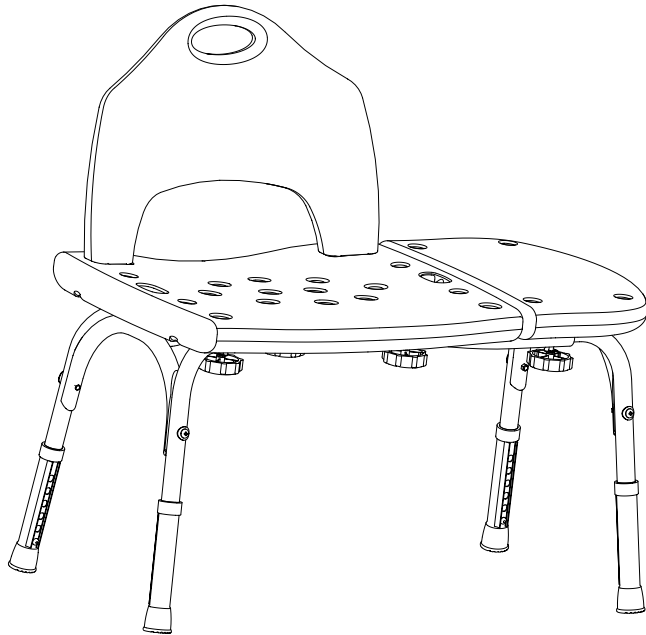
INSTRUCCIONES PARA EL BANCO DE TRANSFERENCIA SIN HERRAMIENTAS HOME CARE® DE MOEN®  
HOME CARE® DE MOEN® - DIRECTIVES POUR L'INSTALLATION DU BANC DE TRANSFERT SANS OUTILS

INNOVATIVE DESIGNS  
for independent living™

Thank you for purchasing this Home Care® by Moen® product. The entire Home Care by Moen family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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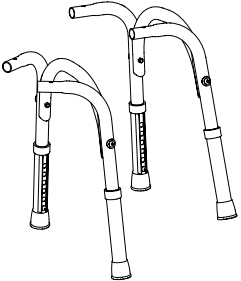
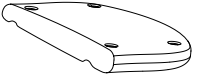
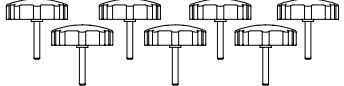
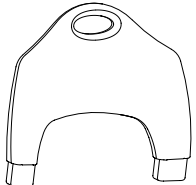
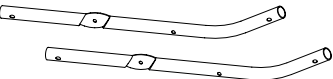
**WARNING: PLEASE FOLLOW THESE ASSEMBLY INSTRUCTIONS CAREFULLY. ASSEMBLER ASSUMES RESPONSIBILITY FOR PROPER ASSEMBLY. FAILURE TO FOLLOW CORRECT ASSEMBLY PROCEDURES CAN CAUSE THE SEAT TO BECOME LOOSE, WHICH CAN RESULT IN SERIOUS INJURY.**

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### Components Included/Componentes incluidos/Pièces incluses

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
INNOVATIVE DESIGNS  
for independent living™

**DN7065 HOME CARE® BY MOEN® TOOL-FREE TRANSFER BENCH INSTRUCTIONS**  
**INSTRUCCIONES PARA EL BANCO DE TRANSFERENCIA SIN HERRAMIENTAS HOME CARE® DE MOEN®**  
**HOME CARE® DE MOEN® - DIRECTIVES POUR L'INSTALLATION DU BANC DE TRANSFERT SANS OUTILS**


- 1.) Determine orientation of chair relative to water supply in order for transfer bench to be positioned outside of bathtub. This determines whether bench will be attached to right or left side of seat.
- 2.) Locate the two under seat support beams.
- 3.) Locate the flat, indented area of each beam. Place the two flat areas together, so they are touching, and the two beams form an "X" shape (as shown in Figure 1).

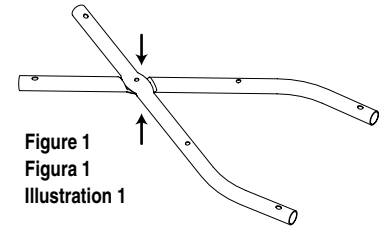**Figure 1**  
**Figura 1**  
**Illustration 1**

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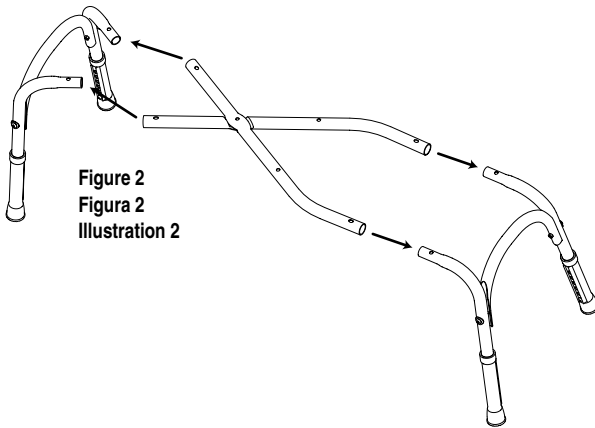
**Figure 2**  
**Figura 2**  
**Illustration 2**

- 4.) Locate the two leg units. Align the top posts with the ends of the crossed support beams (as shown in Figure 2).
- 5.) Slide the top posts of the leg units over the ends of the crossed support beams (as shown in Figure 3).

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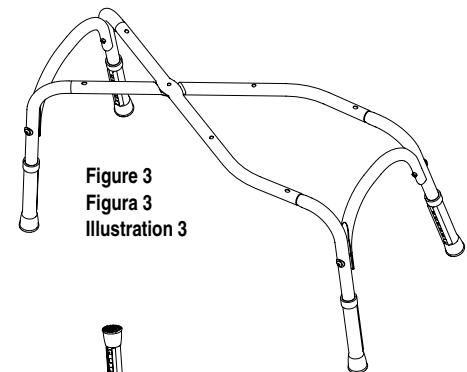
**Figure 3**  
**Figura 3**  
**Illustration 3**

- 6.) Locate the assembly knobs, lock washers and cupped washers. Thread the washers onto the bolt, small lock washer first, followed by the cup washer (as shown in Figure 4).
- 7.) Carefully flip frame so that legs point up. Slide the bolts of the assembly knobs into the holes on the bottom of the under seat support beams (as shown in Figure 5).

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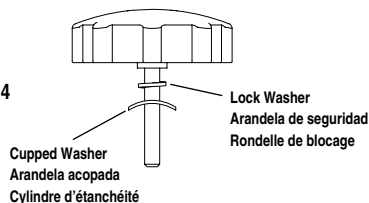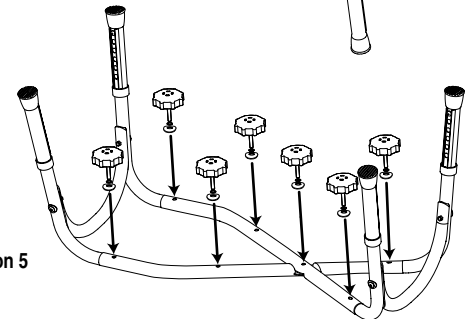
**Figure 5**  
**Figura 5**  
**Illustration 5**

- 8.) Locate the transfer bench. Place it seat side down.
- 9.) Place the curved ends of the crossed support beams into the grooves on the underside of the transfer bench, aligning the holes in the support beams with the holes inside the grooves (as shown in Figure 6).
- 10.) Use the assembly knobs to attach the transfer bench to the support beams.


**IMPORTANT: Hand tighten knobs firmly, but do not force. Tightening knobs too firmly can damage the unit.**

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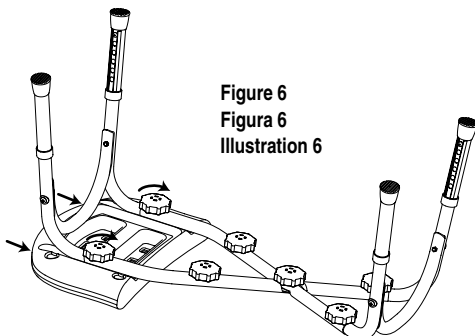**Figure 6**  
**Figura 6**  
**Illustration 6**

INNOVATIVE DESIGNS  
for independent living

**DN7065 HOME CARE® BY MOEN® TOOL-FREE TRANSFER BENCH INSTRUCTIONS**  
**INSTRUCCIONES PARA EL BANCO DE TRANSFERENCIA SIN HERRAMIENTAS HOME CARE® DE MOEN®**  
**HOME CARE® DE MOEN® - DIRECTIVES POUR L'INSTALLATION DU BANC DE TRANSFERT SANS OUTILS**

11.) Locate the large seat bottom. Place it under the support beams so that the center of the "X" slides into the center of the "X" shaped grooves on the large seat bottom (as shown in Figure 7).  
12.) Use the assembly knobs to attach the large seat bottom to the support beams.

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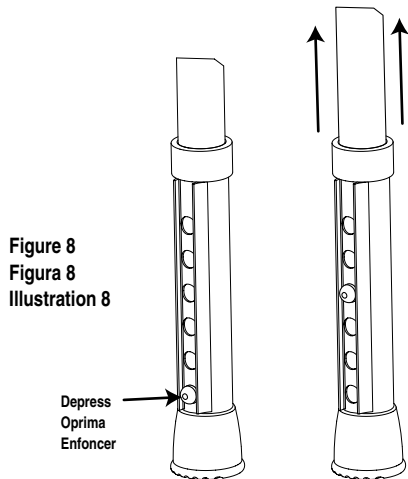
Figure 8  
Figura 8  
Illustration 8

Depress  
Oprima  
Enfoncer

To adjust length of legs/height of seat (see Figure 8).

FOR MAXIMUM STABILITY, CHAIR LEGS ON EACH END OF BENCH MUST BE ADJUSTED TO THE SAME LENGTH

16.) Test each leg to make sure that all studs have fully clicked into place.

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17.) Grasp seat back by handle and insert the tabs into large holes in seat bottom until they snap into place (as shown in Figure 9).

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
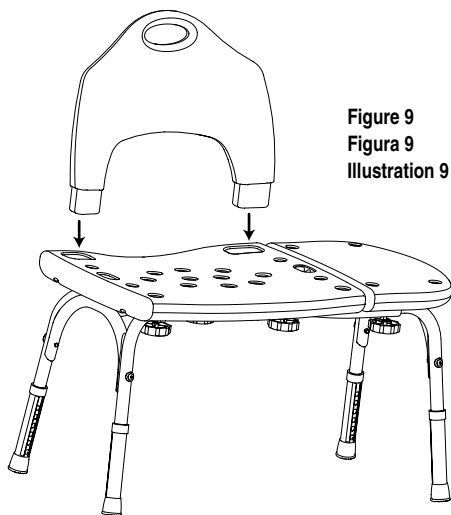
Figure 9  
Figura 9  
Illustration 9

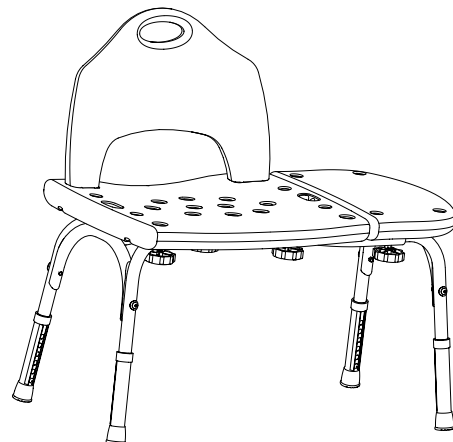INNOVATIVE DESIGNS  
for independent living™

**WARNING:**

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 300 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-882-0116.

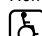
**LIMITED LIFETIME WARRANTY:**

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:** THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit [www.homecare.moen.com](http://www.homecare.moen.com)

Home Care by Moen Tool-Free Transfer Bench meets or exceeds the applicable requirements of the following standards/guidelines:

 The Americans with Disabilities Act ( ADA )

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Creative Specialties is a division of Moen Incorporated. © 2005
Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated.
Home Care and Creative Specialties are a registered trademarks of Moen Incorporated.