


LIFE SOURCE[®]

QUICK RESPONSE

Blood Pressure Monitor

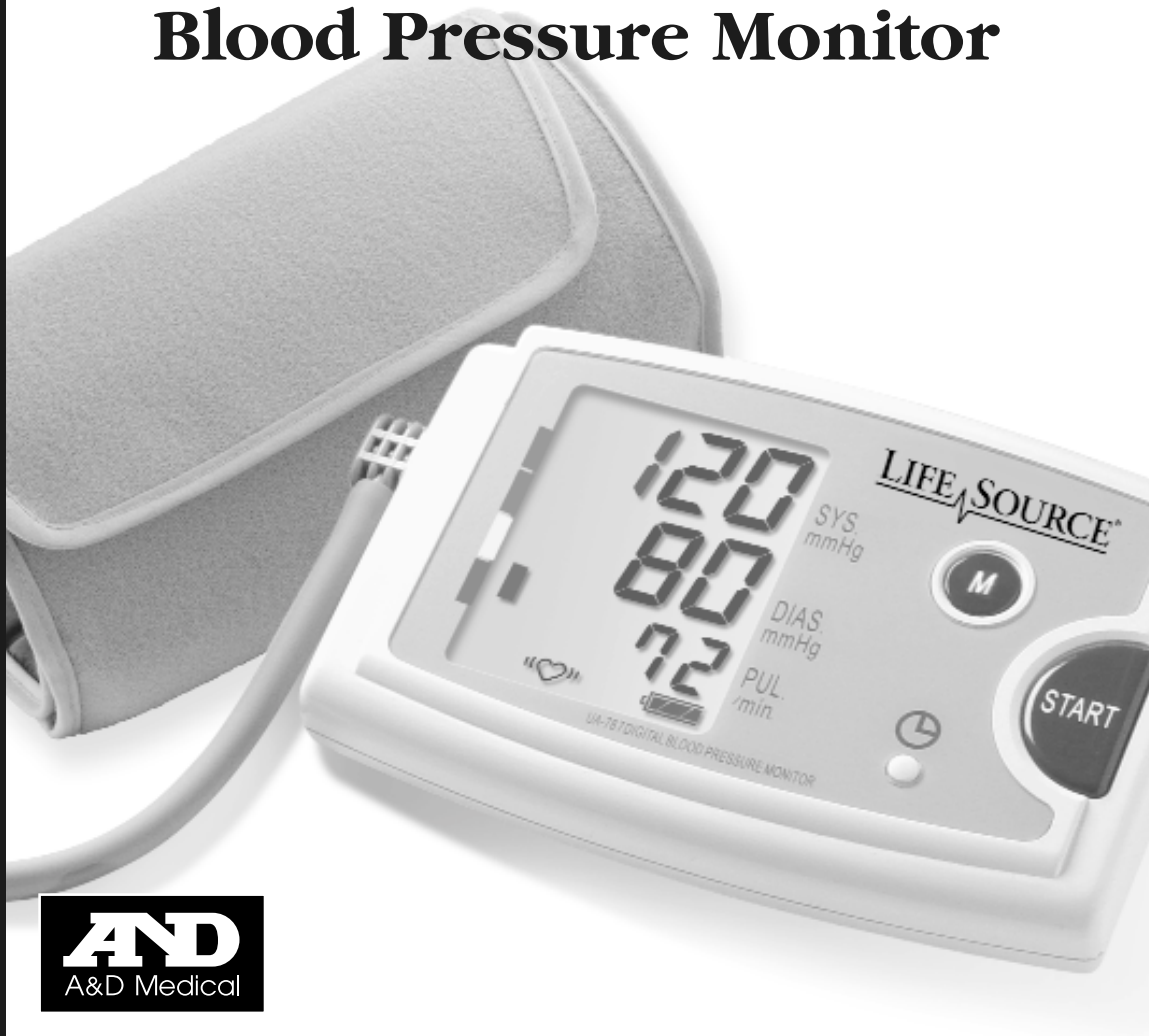

Trilingual Instruction Guide – MODEL UA-787
English ■ Français ■ Español










THIS MONITOR IS CLINICALLY VALIDATED

This LifeSource® blood pressure monitor has undergone and passed a Clinical Validation study to determine the performance and accuracy. This monitor is clinically validated according to the British Hypertension Society (BHS) standard. The BHS is the most prestigious international blood pressure organization. The published study was performed by a reviewing committee consisting of physicians and/or nurses. Among blood pressure manufacturers, A&D Medical is proud to have the highest number of Clinically Validated monitors with a positive recommendation from professionals that are available to consumers.

The Canadian Hypertension Society recognizes the accuracy of this blood pressure monitor.

IMPORTANT INFORMATION

Please read this important information before using your monitor.

-  Please remember that only a medical practitioner is qualified to interpret your blood pressure measurements. Use of this device should not replace regular medical examinations.
-  Have your physician review your procedure for using this device. He or she will want to verify blood pressure measurements before prescribing or adjusting medications.
-  Consult your physician if you have any doubt about your readings. Should a mechanical problem occur, contact A&D Medical.
-  Do not attempt to service, calibrate, or repair this monitor.
-  Because your UA-787 monitor contains delicate, high-precision parts, avoid exposing it to extremes in temperature or humidity or to direct sunlight, shock and dust. A&D Medical guarantees the accuracy of this monitor only when it is stored and used within the temperature and humidity ranges noted on page E-21.
-  Clean the monitor and cuff with a dry, soft cloth or a cloth dampened with water and a mild detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean monitor or cuff.
-  Remove and replace batteries if monitor is not used for more than **six months**. Alkaline batteries recommended.

PRECAUTIONS

The UA-787 is designed to be used at home, by those who are eighteen (18) years and older, to monitor blood pressure (systolic and diastolic) and pulse rate.

LIFE SOURCE[®]

Fast. Easy. Accurate.

Congratulations on purchasing a state-of-the-art LifeSource[®] blood pressure monitor—one of the most technologically advanced monitors available today. Designed for ease of use and accuracy, this monitor will facilitate your daily blood pressure regimen.

Physicians agree that daily self-monitoring of blood pressure is an important step individuals can take to maintain cardiovascular health and prevent the serious consequences associated with undetected and untreated hypertension.


A&D Medical has been manufacturing high quality healthcare products for over 20 years. We are committed to providing you and your family with monitoring devices that provide the utmost in accuracy and convenience.

LifeSource - Designed for Life.

TABLE OF CONTENTS




| | |
|--|------------|
| WHAT DISPLAY SYMBOLS MEAN | E-2 |
| MONITOR COMPONENTS | E-3 |
| HOW UA-787 WORKS | E-4 |

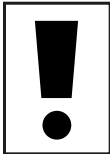


| | |
|--|------------|
| BEFORE YOU START | E-4 |
| SELECTING THE CORRECT CUFF SIZE | E-5 |



| | |
|---|-------------------|
| TAKING YOUR BLOOD PRESSURE | E-6 — E-10 |
| WHAT IS AN IRREGULAR HEARTBEAT | E-11 |
| ABOUT CUFF INFLATION METER | E-11 |
| ABOUT AVERAGE READING | E-12 |
| ABOUT PRESSURE RATING INDICATOR™ | E-12 |
| ABOUT MEMORY | E-13 |

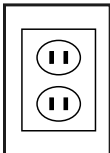

| | |
|--|-------------|
| USING THE MONITOR WITH AC ADAPTER | E-14 |
|--|-------------|

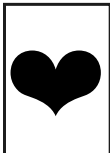

| | |
|-----------------------------------|--------------------|
| ABOUT BLOOD PRESSURE | E-15 — E-19 |
|-----------------------------------|--------------------|



| | |
|------------------------------|-------------|
| TROUBLESHOOTING | E-20 |
| SPECIFICATIONS | E-21 |






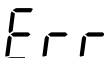

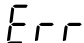



| | |
|----------------------------------|-------------|
| CONTACT INFORMATION | E-22 |
|----------------------------------|-------------|


| | |
|-----------------------------------|------------|
| FRENCH INSTRUCTIONS | F-1 |
| SPANISH INSTRUCTIONS | S-1 |



WHAT DISPLAY SYMBOLS MEAN

| Display Symbol | Condition/Cause | Recommended Action |
|--|---|---|
|  | The symbol shown while measurement is in progress. It blinks while detecting the pulse. | Measurement is in progress, remain as still as possible. |
|  | An irregular heartbeat or body movement may have occurred. Refer to page E-11 for more information on irregular heartbeats. | Take measurement again and consult with your physician. |
|  | Previous measurement stored in MEMORY. | |
|  Battery Full | The battery power indicator during measurement. | Replace all batteries with new ones when the symbol blinks. Alkaline batteries recommended. |
|  Battery Low | The batteries are low when it blinks. | |
|  | Unstable blood pressure due to excessive body movement. | Try the measurement again. Remain very still during the measurement. |
| | The systolic and diastolic values are within 10 mmHg of each other. | Fasten the cuff correctly, and try the measurement again. |
| | The pressure value did not increase during inflation. | Check for air leaks along the tube and around the air socket. |
|  | The cuff is not fastened correctly. | Refasten the cuff and retake measurement. |
| | There is an air leak in the cuff or monitor. | Make sure tube is properly connected to cuff and monitor. |
|  PULSE DISPLAY ERROR | The pulse is not detected correctly. | Try the measurement again. Remain very still during the measurement. |
|  | Cuff Inflation Meter | Measurement is in progress, remain as still as possible. |
| | Pressure Rating Indicator™ | Refer to the section “About Pressure Rating Indicator” for further explanation. |

MONITOR COMPONENTS ?

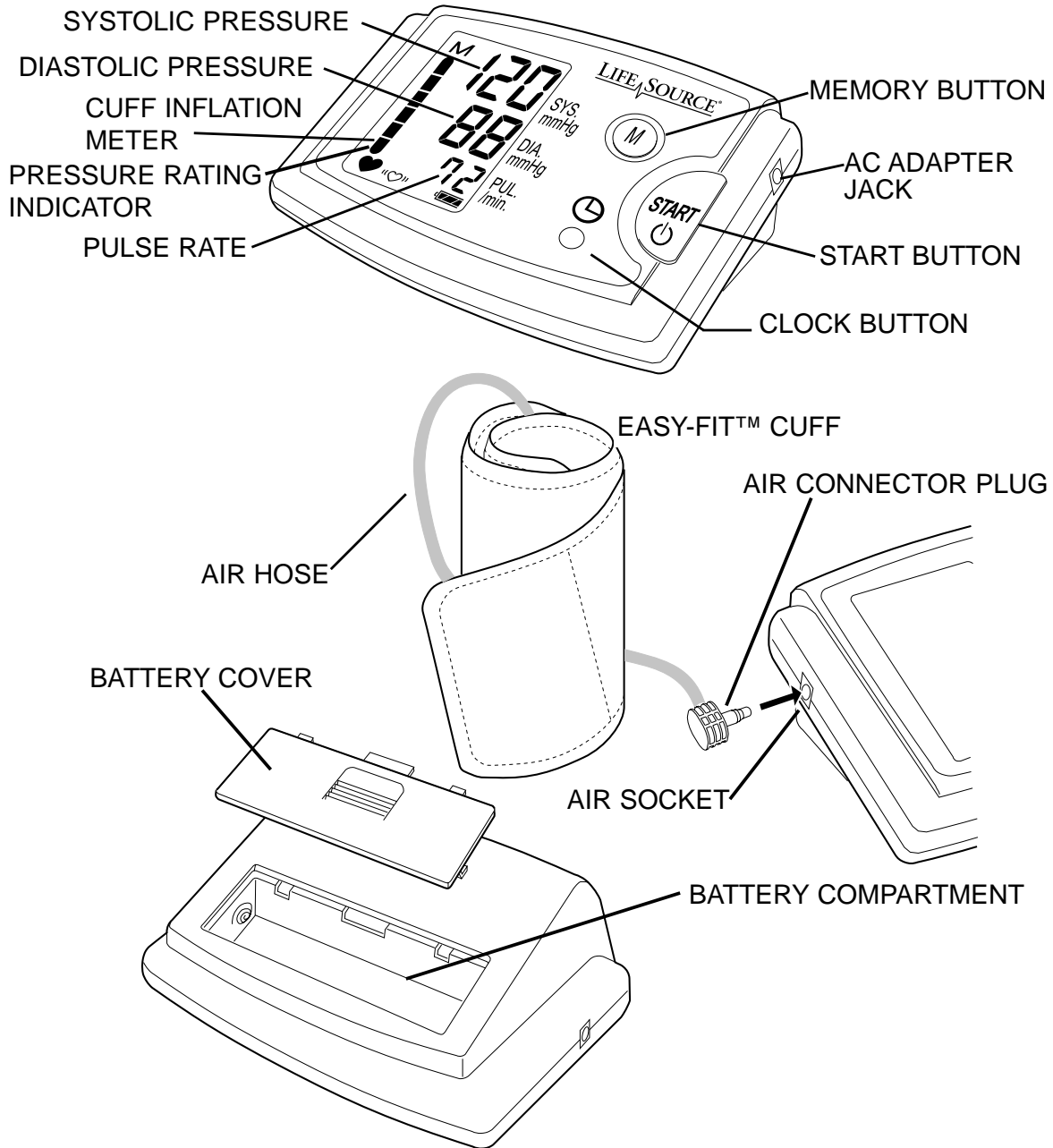



HOW UA-787 WORKS

LifeSource® blood pressure monitors are easy to use, accurate and digitally display full measurement readouts. Our technology is based on the “oscillometric method” – a noninvasive blood pressure determination. The term “oscillation” refers to any measure of vibrations caused by the arterial pulse. Our monitor examines the pulsatile pressure generated by the arterial wall as it expands and contracts against the cuff with each heartbeat.

The cuff is inflated until the artery is fully blocked. The inflation speed is maximized and the pressure level is optimized by the device. The monitor takes measurements while the cuff is both inflating and deflating. This results in a faster measurement providing greater comfort to the user.




BEFORE YOU START

You must install 4 type AA (1.5 volt) batteries (alkaline batteries recommended), or use the AC Adapter (see page E-14 for using the Monitor with AC Adapter) and attach the cuff to the monitor before using it. To install batteries (or replace them if the “Low Battery” symbol appears on display), proceed as follows:

1. Remove battery compartment cover by gently pushing down on arrow and sliding cover forward.
2. Put in bottom row of batteries first. Place the batteries in compartment with positive (+) and negative (–) terminals matching those indicated in the compartment. Be sure batteries make contact with compartment terminals.
3. Replace cover by sliding it into the compartment and gently pressing into place.

NOTE: Rechargeable batteries will not work with this monitor.

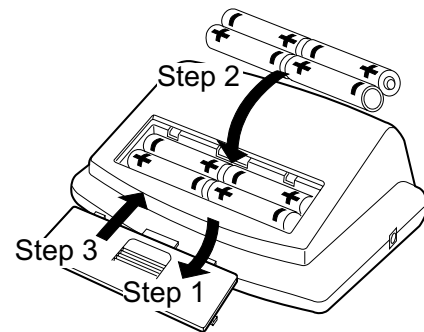

SELECTING THE CORRECT CUFF SIZE



Using the correct cuff size is important for an accurate reading. A cuff that is too large will produce a reading that is lower than the correct blood pressure; a cuff that is too small will produce a reading that is higher than the correct blood pressure. With your arm hanging at the side of your body, measure the circumference of your upper arm at the midpoint between the shoulder and elbow.

| ARM SIZE | RECOMMENDED CUFF SIZE | REPLACEMENT CUFF MODEL# |
|-------------------------|-----------------------|-------------------------|
| 9" - 17" (23-43 cm) | Easy-Fit™ Cuff | |
| 6.3" - 9.4" (16-24cm) | Small Cuff | UA-279 |
| 9.4" - 14.2" (24-36cm) | Medium Cuff | UA-280 |
| 14.2" - 17.7" (36-45cm) | Large Cuff | UA-281 |

Using the right sized cuff is essential to an accurate measurement. Our unique Easy-Fit™ cuff is designed to accommodate most arm sizes, from 9-17" (23-43 cm). The Easy-Fit™ cuff is contoured to provide you with a quick and comfortable fit. Just slide the cuff onto your upper arm and secure – it's that simple! To place the cuff on properly, please read the section Taking Your Blood Pressure (see pg. E-8).

If your arm circumference is greater than 17" (43 cm) or smaller than 9" (23 cm), you will need to purchase a different sized cuff than the one that is included in this box. Please call A&D Medical at 1-888-726-9966 (in Canada – 1-800-461-0991) for more information on our alternate cuff sizes.



TAKING YOUR BLOOD PRESSURE

Adjusting the Built-in Clock

The UA-787 Quick Response monitor has a built-in clock that gives you the time and date and can tell you when a blood pressure reading stored in memory was taken. We recommend that you adjust the clock prior to use. To adjust the clock, follow these simple steps:

- 1.** Press and hold the clock button until the digits start blinking.
- 2.** To select the year, press and release the START button (to fast forward, press and hold the START button). When the current year is selected, set the year by pressing the clock button.
- 3.** After the year is set, follow step 2 to set the month, day, hour (AM/PM) and minute on the display screen.

TAKING YOUR BLOOD PRESSURE




Setting the Alarm

The UA-787 Quick Response monitor allows you to set up to three alarms per day to notify you when to take your blood pressure. To set the alarm(s):

1. Press and hold the clock button until the digits start blinking. Then press and release the memory button. "P-1" will be shown at the middle of the display screen indicating that this is the first alarm.


NOTE: If you do not wish to set an alarm, press the memory button several times to return to the main screen or allow the display to disappear after 30 seconds.

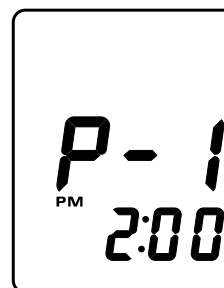

2. Select the hour using the START button and press the clock button to set the hour and move to the minute selection. After selecting and setting the minute, press the memory button to complete the alarm setting.
3. The display will move onto the second alarm setting and conclude with the third alarm setting. Repeat the previous step if you need a second or third alarm.

When the alarm is set, a "P" will appear on the main screen. If you do not see a "P" on the main screen, press the clock button to activate the alarm function. When the alarm sounds, it will last for one minute. To turn the alarm off immediately, press the START button.



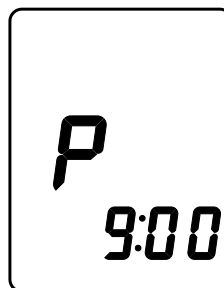



NOTE: To deactivate the alarm function, simply press the clock button once. You can reactivate the alarm function by pressing the clock button again.







TAKING YOUR BLOOD PRESSURE

Tips for Blood Pressure Monitoring:

-  Relax for about 5 minutes before measurement.
-  Do not smoke or ingest caffeine at least 30 minutes prior to measurement.
-  Remove constricting clothing and place cuff on bare arm.
-  Unless your physician recommends otherwise, use left arm to measure pressure.
-  Do not talk during measurement.
-  Do not cross legs and keep your feet flat on floor during measurement.


Now you are ready. Follow these simple steps:

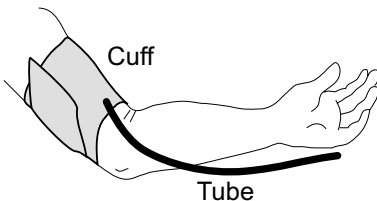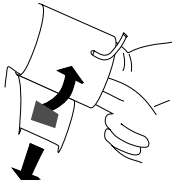- 1.** Sit comfortably with your left arm resting on a flat surface so that the center of your upper arm is at the same height as your heart.
- 2.** Lay left arm on the table with palm up. Place the Easy-Fit cuff on your upper arm. Align the white marker on the cuff over the brachial artery on the inside of the arm. The tube should be facing downward and toward the inside of your arm. Fasten the cuff securely. The bottom of the cuff should be about 1" above your elbow.
- 3.** The cuff should be snug but not too tight. You should be able to insert two fingers between the cuff and your arm.

TAKING YOUR BLOOD PRESSURE




4. Press the START button. Your average blood pressure reading will appear with the number of measurements stored in memory (i.e. A25 = total of 25 measurements stored in memory). As the cuff pressurizes, measurement will begin and the Cuff Inflation Meter will show on the display screen. It is normal for the cuff to feel very tight.

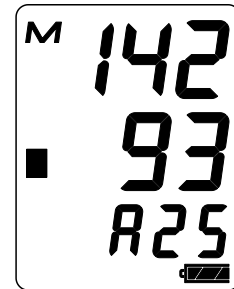

NOTE: If an appropriate pressure is not obtained, the device automatically starts to inflate again.

NOTE: If you wish to stop inflation at any time, press the START button again.

5. When the measurement is complete, the systolic and diastolic pressure readings and pulse rate are displayed. The cuff deflates and the monitor automatically shuts off after 45 seconds, or you can turn it off by pressing the START button.

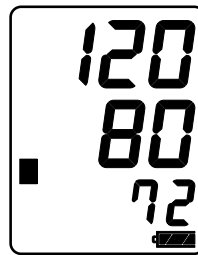

SYSTOLIC

DIASTOLIC

PULSE

6. Remove cuff and make note of your blood pressure and pulse rate on the Blood Pressure Record chart in the back of the manual, indicating the date and time of measurement.

IMPORTANT: Measure blood pressure at the same time each day.





TAKING YOUR BLOOD PRESSURE

Measurement with the desired systolic pressure:

If your systolic pressure is expected to exceed 230 mmHg follow these steps:

- 1.** Place the cuff on the left arm, unless your physician tells you otherwise.
- 2.** Press and hold the START button until a number 30 to 40 mmHg higher than your expected systolic pressure appears.
- 3.** Release the START button when the desired number is reached. 280 mmHg is the highest pressure level the monitor can reach. Measurement will then begin. Continue to measure your blood pressure as described on Page E-8.

WHAT IS AN IRREGULAR HEARTBEAT



The UA-787 Blood Pressure Monitor provides a blood pressure and pulse rate measurement even when an irregular heartbeat occurs. The Irregular Heartbeat symbol «♥» will appear in the display window in the event an Irregular Heartbeat has occurred during measurement. An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeat intervals during the blood pressure measurement. It is important that you relax, remain still and refrain from talking during measurements.

NOTE: We recommend contacting your physician if you see this «♥» symbol frequently.

ABOUT CUFF INFLATION METER




The Cuff Inflation Meter is located on the left side of the display screen to tell you when the blood pressure monitor is inflating and deflating the cuff. The Cuff Inflation Meter moves up during inflation and moves down during deflation.

| Inflation in Progress | Inflation Complete | Deflation/Measurement in Progress |
|-----------------------|--------------------|-----------------------------------|
| | | |



ABOUT AVERAGE READING

The UA-787 provides an average blood pressure reading based on the total measurements stored in memory. When taking a measurement, the average blood pressure reading will appear and the number of measurements stored in memory will flash several times before the cuff begins to inflate. The average blood pressure reading will also appear briefly when you recall the measurements stored in memory.

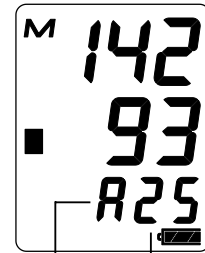



AVERAGE
NUMBER OF MEASUREMENTS STORED
IN MEMORY



ABOUT PRESSURE RATING INDICATOR™

The Pressure Rating Indicator™ is a feature which provides a snapshot of your blood pressure classification based on your measurements. This will let you quickly know what your blood pressure readings mean. Each segment of the bar indicator corresponds to the Seventh Report of the Joint National Committee (JNC7) on Prevention, Evaluation and Treatment of High Blood Pressure from the National Heart, Lung and Blood Institute - May 2003. For a more detailed look at this blood pressure classification, please refer to “Assessing High Blood Pressure” on page E-16.

Pressure Rating Indicator™

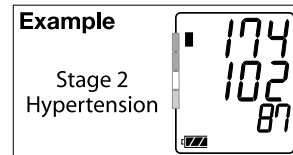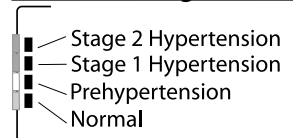

■ :The indicator displays a segment, based on the current measurement, corresponding to the JNC7 Classification.

NOTE: Due to other risk factors (e.g. diabetes, obesity, smoking, etc.) in addition to your blood pressure measurement, the Pressure Rating Indicator is approximate. Please consult with your physician for interpretation and diagnosis of your blood pressure measurements.

NOTE: Residents outside of the United States (e.g., Canada and Mexico) should refer to the WHO Classification Table on page E-16 for assessment of their blood pressure measurement.



ABOUT MEMORY



This monitor automatically stores up to sixty (60) blood pressure and pulse measurements in memory. Measurements stored in memory are assigned an index number in the order of the newest to the oldest.

To retrieve readings, follow these simple steps:


1. When the display screen is showing only the time, press and quickly release the Memory ("M") button. You will see the date and time of the most recent blood pressure that was taken followed by the measurement.
2. If you want to retrieve other measurements in memory, press and release the Memory button repeatedly until the desired date and time of the measurement are reached. The measurement will then be displayed.
3. The display will shut off automatically a few seconds after the readings are displayed.

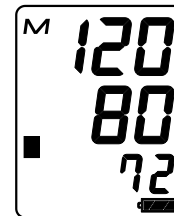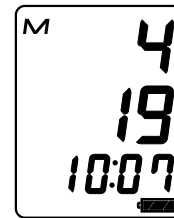

To **clear** the measurement history from memory, press and hold the Memory button for at least five seconds while the display screen is showing only the time. Release the Memory button when you see the M symbol in upper left hand corner of the display flash and disappears. This indicates that the memory has been cleared.

If there are no measurements stored in memory, you will see dashes in the date and time fields followed by a measurement of three 0s displayed vertically.

NOTE: Stored measurements will be lost if batteries become low or are removed.

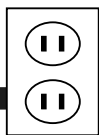

USING THE MONITOR WITH AC ADAPTER

The UA-787 has an AC adapter jack to allow you to supply power from an outlet in your home. We recommend you use only the LifeSource AC adapter to avoid potential damage to the monitor.


Connecting the AC adapter to the monitor:

1. Gently insert the AC adapter plug into a 120V AC outlet.
2. Connect the AC adapter plug into the jack on the right side of the monitor.

Disconnecting the AC adapter from the monitor:

1. Turn the unit off by pressing the START button.
2. Disconnect the plug from the jack of the monitor quickly.
3. Gently remove the AC adapter from the outlet.

NOTE: If monitor has batteries and an AC adapter connected, the measurements stored in memory will be lost if AC adapter is unplugged from the wall first. To ensure that measurements remain in memory, make sure the unit is off and that the jack plugged into the monitor is removed quickly before unplugging adapter from the wall.

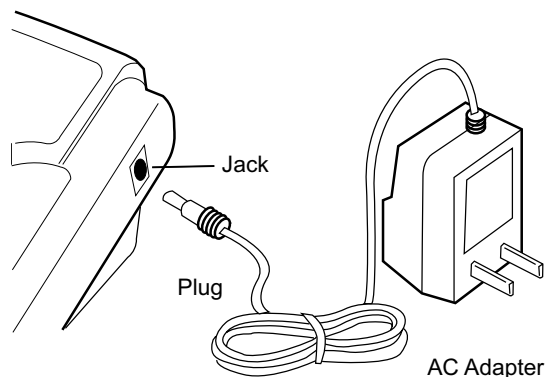

ABOUT BLOOD PRESSURE

■ What Is Blood Pressure?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts; diastolic pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg).

■ What Affects Blood Pressure?


Blood pressure is affected by many factors: age, weight, time of day, activity level, climate, altitude and season. Certain activities can significantly alter one's blood pressure. Walking can raise systolic pressure by 12 mmHg and diastolic pressure by 5.5 mmHg. Sleeping can decrease systolic blood pressure by as much as 10 mmHg. Taking your blood pressure repeatedly without waiting an interval of 5 minutes between readings, or without raising your arm to allow blood to flow back to the heart, can also affect it.

In addition to these factors, beverages containing caffeine or alcohol, certain medications, emotional stress and even tight-fitting clothes can make a difference in the readings.

■ What Causes Variations In Blood Pressure?

An individual's blood pressure varies greatly from day to day and season to season. For hypersensitive individuals, these variations are even more pronounced. Normally, blood pressure rises during work or play and falls to its lowest levels during sleep.

Fluctuation within a day (case: 35 year old male)

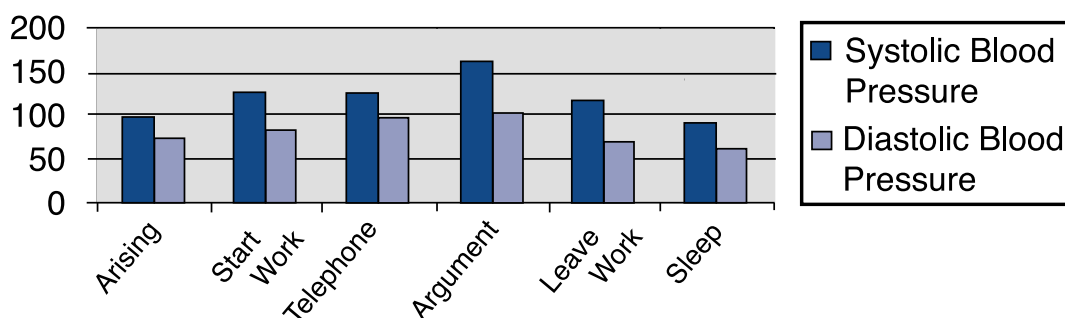



ABOUT BLOOD PRESSURE

■ Assessing High Blood Pressure for Adults

The following standards for assessing high blood pressure (without regard to age or gender) have been established as a guideline. Please note that other risk factors (e.g. diabetes, obesity, smoking, etc.) need to be taken into consideration and may affect these figures. Consult with your physician for an accurate assessment.

JNC7 Classification Table – for adults within the U.S.

| BP Classification | Systolic (mmHg) | Diastolic (mmHg) |
|----------------------|------------------------------|---------------------------------|
| Normal | less than 120 | and less than 80 |
| Prehypertension | 120-139 | or 80-89 |
| Stage 1 Hypertension | 140-159 | or 90-99 |
| Stage 2 Hypertension | greater than or equal to 160 | or greater than or equal to 100 |

SOURCE: The Seventh Report of the Joint National Committee on Prevention, Evaluation and Treatment of High Blood Pressure for Adults. National Heart, Lung and Blood Institute - May 2003.

WHO Classification Table – for adults outside of the U.S. (e.g. Canada, Mexico)

| BP Classification | Systolic (mmHg) | Diastolic (mmHg) |
|----------------------|------------------------------|---------------------------------|
| Optimal | less than 120 | and less than 80 |
| Normal | less than 130 | or less than 85 |
| High-Normal | 130-139 | or 85-89 |
| Stage 1 Hypertension | 140-159 | or 90-99 |
| Stage 2 Hypertension | 160-179 | or 100-109 |
| Stage 3 Hypertension | greater than or equal to 180 | or greater than or equal to 110 |

SOURCE: Standards to assess high blood pressure, without regard to age or gender, have been established by the World Health Organization (WHO).



■ What Is Hypertension?

Hypertension (high blood pressure) is the diagnosis given when readings consistently rise above normal. It is well known that hypertension can lead to stroke, heart attack or other illness if left untreated. Referred to as a “silent killer” because it does not always produce symptoms that alert you to the problem, hypertension is treatable when diagnosed early.

■ Can Hypertension Be Controlled?

In many individuals, hypertension can be controlled by altering lifestyle and minimizing stress, and by appropriate medication prescribed and monitored by your doctor. The American Heart Association recommends the following lifestyle suggestions to prevent or control hypertension:

- **Don't smoke.**
- **Reduce salt and fat intake.**
- **Maintain proper weight.**
- **Exercise routinely.**
- **Have regular physical checkups.**
- **Monitor your blood pressure at periodic intervals.**

■ Why Measure Blood Pressure at Home?

It is now well known that, for many individuals, blood pressure readings taken in a doctor's office or hospital setting might be elevated as a result of apprehension and anxiety. This response is commonly called “white coat hypertension.” In any case, self-measurement at home supplements your doctor's readings and provides a more accurate, complete blood pressure history. In addition, clinical studies have shown that the detection and treatment of hypertension is improved when patients both consult their physicians and monitor their own blood pressure at home.



ABOUT BLOOD PRESSURE

■ **Answers to Why Your Readings Are Different Between Home and at the Doctor's Office**

Why are my readings different between home and at a doctor's office?

Your blood pressure readings taken in a doctor's office or hospital setting may be elevated as a result of apprehension and anxiety. This response is known as white coat hypertension.

When I bring my monitor to a doctor's office, why do I get a different measurement from my monitor to that taken by a doctor or nurse?

The healthcare professional may be using a different sized cuff. The size of the bladder inside the cuff is critical for the accuracy of the measurement. This may give you a different reading. A cuff too large will produce a reading that is lower than the correct blood pressure; a cuff that is too small will produce a reading that is higher than the correct blood pressure. There may also be other factors that can cause the difference in measurements.

■ **Keys to Successful Monitoring:**

Blood pressure fluctuates throughout the day. We recommend that you are consistent in your daily measurement routine:

- Measure at the same time every day.
- Sit in the same chair/position.
- Do not cross legs and keep your feet flat on the floor.
- Relax for 5 minutes before measurement.
- Use the correct cuff size to get an accurate reading.
- Sit still during measurement – no talking, eating or sudden movements.
- Record your measurement in a logbook.



■ Establishing Baseline Measurements

The most important method to get an accurate blood pressure measurement is consistency. To get the most benefit out of your monitor, it is important to establish a “baseline measurement.” This helps build a foundation of measurements that you can use to compare against future readings. To build this baseline measurement, devote two weeks for consistent blood pressure monitoring. This involves doing everything the same way when you measure (e.g. measuring during the same time of day, in the same location, sitting in the same chair, using the same cuff, etc.). Once you establish your baseline measurement, you can start evaluating if your measurement has been affected based on things lifestyle changes or medication treatment.

■ How Do I Record My Blood Pressure?

Blood pressure readings are typically recorded with the systolic pressure written first, followed by a slash mark and the diastolic pressure. For example, 120 mmHg systolic and 80 mmHg diastolic measurements are written as 120/80. Pulse is simply written with the letter “P” followed by the pulse rate—P 72, for example. Please see the back of the manual for the blood pressure tracking record.

■ Customer Support Tools Online

Additional tools are available on www.LifeSourceOnline.com (in Canada - www.autocontrol.com) to help you get the most out of your blood pressure monitoring. These include:

- Large Print Instruction Manuals
- Animated Operating Instructions for select models
- Additional Logbook Sheets

TROUBLESHOOTING

| Problem | Probable cause | Corrective action |
|---|---|---|
| Nothing appears in the display, after I press the START button. | Batteries are drained. | Replace all batteries with new ones. Alkaline batteries recommended. |
| | Battery terminals are not in the correct position. | Reinstall the batteries with negative and positive terminals matching those indicated in the battery compartment. |
| The cuff does not inflate. | Battery voltage is too low. Low battery symbol blinks. [If the batteries are drained completely, the mark does not appear.] | Replace all batteries with new ones. Alkaline batteries recommended. |
| The unit does not measure. Readings are too high or too low. | The cuff is not fastened properly. | Fasten the cuff correctly. |
| | You moved your arm or body during the measurement. | Make sure you remain very still and quiet during the measurement. |
| | The cuff position is not correct. | Sit comfortably and still. Make sure the cuff is at the same level as your heart. |
| | You are using the wrong size cuff. | See Pg. E-5 "Selecting the Correct Cuff." |
| | Taking too many readings on the same arm in a short period of time. | Relax five minutes before each measurement. |
| The value is different from that measured at a clinic or doctor's office. | The healthcare professional may be using a different sized cuff. | See Pg. E-18 "Answers to Why Your Readings Are Different Between Home and the Doctor's Office." |
| | Your measurements may be elevated by white coat hypertension | See Pg. E-17 "Why Measure Blood Pressure at Home." |

NOTE: If the actions described above do not solve the problem, call 1-888-726-9966 (In Canada - 1-800-461-0991). Do not attempt to repair the device yourself.

SPECIFICATIONS

| | |
|------------------------------------|---|
| Model | UA-787 |
| Type | Oscillometric |
| Display | Digital, 20-mm character height Pressure/pulse displayed simultaneously |
| Time/Date | 12 hour format (AM/PM), year (2005-2054), month, and day with automatic adjustment for leap years |
| Memory | 60 readings |
| Measurement range | Pressure: 20 mmHg to 280 mmHg Pulse: 40 pulses to 200 pulses |
| Accuracy | Pressure: ± 3 mmHg or $\pm 2\%$, whichever is greater Pulse: $\pm 5\%$ |
| Pressurization | Automatic, using micropump |
| Depressurization | Active exhaust valve (ECEVII) |
| Power source | 4 type "AA" (1.5 volt) alkaline batteries (not included) or 120 V AC Adapter |
| Battery life | Approximately 4 months with one daily measurement |
| Operating environment | 50°F to 104°F (10°C to 40°C) Less than 85% relative humidity |
| Storage environment | -4°F to 140°F (-20°C to 60°C) Less than 95% relative humidity |
| Dimensions | Length: 4.4"(112 mm) Width: 6.4"(163 mm) Height: 2.4" (62 mm) |
| Weight | 12.3 oz. (350 g) without batteries |

Blood pressure measurements determined by the UA-787 are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method within the limits prescribed by the American National Standards Institute for electronic or automated sphygmomanometers.



CONTACT INFORMATION

Our products are designed and manufactured using the latest scientific and technological methods, and offer accurate, easy to use, home monitoring and treatment options. Our full product line includes:

- **Digital Blood Pressure Monitors**
- **Manual Blood Pressure Kits**
- **Blood Pressure Cuffs**
- **Stethoscopes**
- **Personal Health Scales**
- **Nebulizers**
- **Digital Thermometers**

This LifeSource blood pressure product is covered by a Lifetime Warranty. See warranty card for details.

Visit our web site at www.LifeSourceOnline.com for warranty registration.

For more information regarding use, care or servicing of your blood pressure monitor, contact:

A&D Medical

A division of A&D Engineering, Inc.

1555 McCandless Drive

Milpitas, CA 95035

LifeSource Health Line (Toll-Free): 1-888-726-9966

www.LifeSourceOnline.com

For Canadian Residents, please contact:

Auto Control Medical

6695 Millcreek Drive, Unit 5

Mississauga, Ontario

L5N 5R8 Canada

Auto Control (Toll-Free): 1-800-461-0991

www.autocontrol.com

IMPORTANT!

If You Need Assistance with the Set-Up or Operation

We Can Help!

Please call us FIRST before contacting your retailer at

LifeSource Health Line

1-888-726-9966 - Toll Free

Auto Control Medical

1-800-461-0991 - In Canada

A specially trained representative will assist you



A division of A&D Engineering, Inc.

1555 McCandless Drive

Milpitas, CA 95035

www.LifeSourceOnline.com

Pressure Rating Indicator and Easy-Fit™ Cuff are trademarks and LifeSource is a registered trademark of A&D Medical. Pressure Rating Indicator is patent pending.

©2005 A&D Medical. All rights reserved.

Specifications subject to change without notice.

I-MAN:787 Printed in China 4/05