



549 SERIES SUPPORTO TRUSS



INSTRUCTIONS FOR USE

CAUTION: This product contains natural rubber latex which may cause allergic reactions.

Read these instructions before using the Pipeer Supporto Truss. Save these instructions for future reference.

The most current version of these instructions can be found online at www.grahamfield.com.

Safety

- ⚠ WARNING: DO NOT use this product without proper instruction from a healthcare professional.**
- ⚠ WARNING: Notice for California Customers- California Proposition 65 WARNING: This product contains a chemical known to the State of California to cause cancer and reproductive or developmental harm.**
- ⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.**

Intended use

The intended use of the Supporto Truss is the treatment of reducible inguinal hernia and athletic support. Another undergarment is not needed while wearing Supporto Truss.

Application

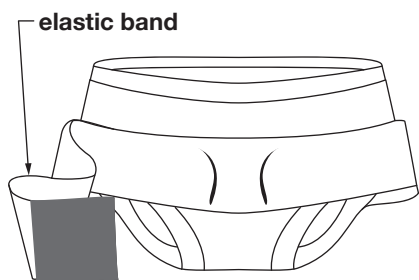


fig. 1

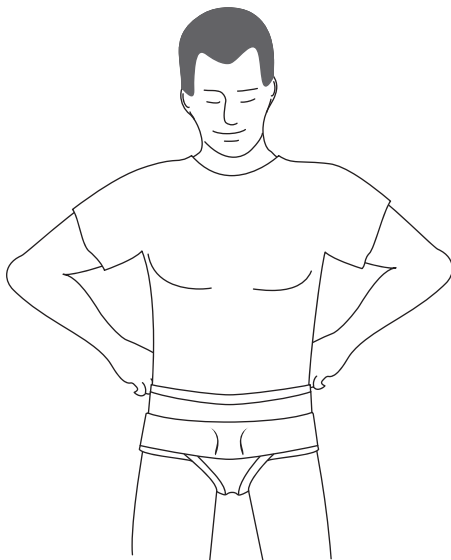


fig. 2

1. Pull apart the elastic band. Leave unattached.
2. If rupture is protruding, lie down. Reduce rupture size to normal by using the palm of your hand to gently push rupture upward and in. If necessary, cough to locate exact position of rupture opening.

3. Step into the Supporto Truss as you would any briefs or undershorts. Pull up into proper fitting position. There are two pads to provide balanced support for reducible inguinal hernia. Center pads on exact rupture opening.
4. Stand up straight. Take one end of the elastic band in each hand and pull out and around your back to attach the hook and loop closure firmly. Adjust until the fit is comfortably snug. ENSURE THE PAD IS PLACED AT THE POINT OF RUPTURE.

Removal

To remove Supporto Truss, unfasten the elastic band hook and loop closure and step out of Supporto Truss.

Cleaning and Maintenance

Machine wash Supporto Truss in warm water, gentle cycle, use mild soap. Tumble dry low, slow, short cycle.

▲ **NOTICE: DO NOT use detergent or bleach to clean Supporto Truss. DO NOT iron Supporto Truss.**

Limited Warranty

GF Health Products, Inc. ("Graham-Field") guarantees the Pipeer 549 Series Supporto Truss free from defects in materials and workmanship under normal use for thirty days from the date of purchase.

If a product is deemed to be under warranty, GF Health Products, Inc. shall provide, at its option, (1) replacement of any defective part or product or (2) a credit of the original selling price made to GF Health Products, Inc.'s initial customer. The warranty does not include any labor charges incurred in replacement part(s) installation or any associated freight or shipping charges to GF Health Products, Inc.

The warranties contained herein contain all the representations and warranties with respect to the subject matter of this document, and supersede all prior negotiations, agreements and understandings with respect thereto. The recipient of this document hereby acknowledges and represents that it has not relied on any representation, assertion, guarantee, warranty, collateral contract or other assurance, except those set out in this document.

GF Health Products, Inc.
2935 Northeast Parkway
Atlanta, GA 30360
telephone: 770-368-4700
fax: 770-368-2386



GRAHAM-FIELD
www.grahamfield.com



© 2012 GF Health Products, Inc.

GF Health Products, Inc. is not responsible for typographical errors. For the most updated and current information on packaging, warranties, products and specifications, please visit our website at **www.grahamfield.com**. **Graham-Field** and **Pipeer** are registered trademarks of GF Health Products, Inc.