

- ⚠ WARNING:** Important! Read and understand these instructions before using the Gowllands 1226 Otoscope/Ophthalmoscope Set. If you do not understand any part of these instructions, contact your medical supply distributor for direction in the use of this product. If the Otoscope/Ophthalmoscope Set is not properly operated, personal injury and/or damage to the device could result.
- ⚠ WARNING:** If components are damaged or missing, contact your dealer immediately. **DO NOT** use substitute parts.
- ⚠ WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

### OTOSCOPE/OPHTHALMOSCOPE SET OPERATION

The set includes a chrome-plated otoscope head designed with integral and swivel lens, and three aural (small, medium, large) and one nasal specula. The pre-focused ophthalmoscope head has five different apertures: slit small circle, large circle, green filter and graticule. Corrective lens can be selected from +40 to -25 D. Lamp bulb #12 included. Two "C" batteries not included.



#### To attach otoscope to power source:

Align otoscope with base, push in slightly, and turn.  
To remove it, reverse this procedure.

#### To attach ophthalmoscope to power source:

Align ophthalmoscope with base, push in slightly and turn. To remove it, reverse this procedure.

#### Cleaning instructions

Do not apply alcohol, chemicals or water. Clean with a dry, clean cloth. The window may be cleaned with a cotton swab or lens-cleaning paper.



[www.grahamfield.com](http://www.grahamfield.com)

GF and Graham-Field are registered trademarks of GF Health Products, Inc.

© GF Health Products, Inc. • 2935 Northeast Parkway, Atlanta, GA 30360 • tel 800-347-5678 • fax 800-721-0601