

CorPak™

Hot & Cold Therapy for Every Budget

HOW TO BUY



● **Dual Comfort CorPak™**
Two different fabrics provide your choice of rapid-release or slow-release therapy

● **Soft Comfort CorPak™**
Fabric cover feels gentle against the skin

● **Basic Comfort CorPak™**
Same great fill as the other CorPaks with an economical plastic covering

Pain Management System

Great for strains, sprains, tendinitis and other injuries, these packs are available in various sizes to treat virtually any spot on your body.

Non-toxic, biodegradable gel retains therapeutic temperatures longer. Store in the freezer until needed; stays soft and pliable. Heat in microwave or hot towel cabinet.

Dual Comfort CorPak™

The Dual Comfort Packs feature two "frost-free" surfaces to provide a full-range of treatment. The plush surface provides slow-release treatment while the rapid release surface provides quicker results for more acute situations. Unique elastic strapping system fastens anywhere on the plush, high-

nap surface (dark blue surface) to keep them securely in place almost anywhere on the body.

Soft Comfort CorPak™

The Soft Comfort Packs have a "frost-free" surface that protects your skin while still allowing therapeutic temperatures to penetrate.

Basic Comfort CorPak™

The Basic Comfort Packs feature a cost-saving plastic cover instead of fabric making it perfect for the cost-conscious consumer. Packs can be used as promotional giveaways or sold as the perfect travel hot/cold pack. Basic Comfort CorPaks should be used with a towel or other protective covering.



Specifications

Options: (weights are approximate)

Dual Comfort CorPak

• #530	3x5"	0.50 lbs	12/box	5 lbs
• #531	6x10"	1.75 lbs	12/box	21 lbs
• #532	10x13"	4.00 lbs	6/box	25 lbs
• #533	6x20"	2.25 lbs.	12/box	27 lbs
• #534	9x16" Tri-Sectional	4.50 lbs	6/box	27 lbs

Soft Comfort CorPak

• #549	3x5"	0.25 lbs	12/box	4 lbs
• #550	6x10"	1.25 lbs	12/box	15 lbs
• #551	10x13"	3.00 lbs	6/box	18 lbs
• #552	6x20" Cervical	1.75 lbs	12/box	21 lbs
• #554	9x16" Tri-Sectional	4.25 lbs	6/box	26 lbs

Basic Comfort CorPak

• #590	6x9"	1 lb	24/box	25 lbs
• #592	9x12"	2 lbs	12/box	25 lbs

Colors

- Blue



Core Products
American Manufacturing Company



Available From:

Hot & Cold Comfort Packs

Instructions

These hot & cold Comfort Packs feature a "frost-free" surface that protects the skin while allowing therapeutic temperatures to penetrate. For best results, store the packs in your freezer. Store flat. Freeze for 1 to 2 hours to chill. To heat, place flat in your microwave and heat as instructed. Do not overheat.

If you have questions about the use of these hot & cold packs, consult your doctor. Do not use on broken skin. Individuals with circulation problems, the disabled and children should use these products only with supervision of a doctor.

Heating

All values are based on a cold pack. Knead after each heating. Microwave ovens vary; Use care when heating and removing pack from microwave. Overheating your pack may cause it to rupture and leak. Carefully check the pack with the back of your hand before applying. Discard pack if punctured or broken.

3x5" Pack: Heat 15 seconds on high heat, knead, then knead again at 5 second intervals until desired temperature. Knead after each heating.

6x10" Pack: Heat 30 seconds on high heat, knead, then knead again at 10 second intervals until desired temperature. Knead after each heating.

10x13" Pack: Heat 30 seconds on high heat, knead, then knead again at 10 second intervals until desired temperature. Knead after each heating.

6x20" Pack: Heat 30 seconds on high heat, knead, then knead again at 10 second intervals until desired temperature. Knead after each heating.

Tri-Sectional Pack: Heat 30 seconds on high heat, knead, then knead again at 10 second intervals until desired temperature. Knead after each heating.

Care of Pack

Hand wash packs with warm, soapy water. Air dry flat.



Available from: